

Master BOARD[®]




The Original Scandinavian Human Handling System



- | | | | |
|----------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------|
| <input type="checkbox"/> Art. No. MC 005-1060
Master BOARD Medium
Dimensions: L 110 x B 40 cm | <input type="checkbox"/> Art. No. MC 005-1061
Master BOARD Medium
Cover (Glide) of nylon (40)
Dimensions: L 110 x B 44 cm | <input type="checkbox"/> Art. No. MC 005-1200
Hmi nr. 55311
Master BOARD Large
Dimensions: L 180 x B 40 cm | <input type="checkbox"/> Art. No. MC 005-1201
Hmi nr. 55312
Master BOARD Large
Cover (Glide) of nylon (40)
Dimensions: L 180 x B 44 cm |
| <input type="checkbox"/> Art. No. MC 005-1070
Hmi nr. 34170
Master BOARD Medium
Dimensions: L 110 x B 50 cm | <input type="checkbox"/> Art. No. MC 005-1071
Hmi nr. 34173
Master BOARD Medium
Cover (Glide) of nylon (50)
Dimensions: L 110 x B 54 cm | <input type="checkbox"/> Art. No. MC 005-1210
Hmi nr. 33236
Master BOARD Large
Dimensions: L 180 x B 50 cm | <input type="checkbox"/> Art. No. MC 005-1211
Hmi nr. 38948
Master BOARD Large
Cover (Glide) of nylon (50)
Dimensions: L 180 x B 54 cm |
| <input type="checkbox"/> Art. No. MC 005-1080
Master BOARD Medium
Dimensions: L 110 x B 60 cm | <input type="checkbox"/> Art. No. MC 005-1081
Master BOARD Medium
Cover (Glide) of nylon (60)
Dimensions: L 110 x B 64 cm | <input type="checkbox"/> Art. No. MC 005-1220
Master BOARD Large
Dimensions: L 180 x B 60 cm | <input type="checkbox"/> Art. No. MC 005-1221
Master BOARD Large
Cover (Glide) of nylon (60)
Dimensions: L 180 x B 64 cm |

When the mobility is impaired...

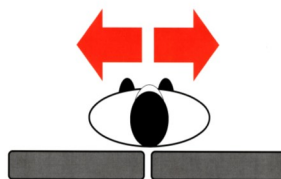

... we improve your quality of life!


Master BOARD[®]



The Original Scandinavian Human Handling System


USER GUIDE

A. preparation of Master BOARD[®] Medium

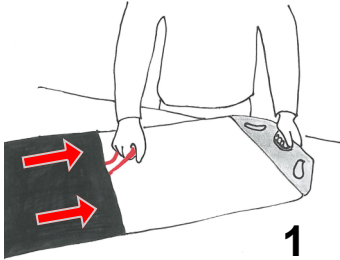

1

The enclosed nylon cover is placed over Master BOARD Medium.

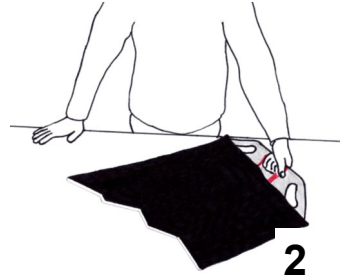

2

Master BOARD Medium is now ready for use.



3


Before transfer is attempted, the wheels on both trolleys **MUST** be locked.

B. Positioning of Master BOARD[®] Medium



1

One helper turns the user onto the desired side. You may use a mattress cover or bed sheet as an aid.

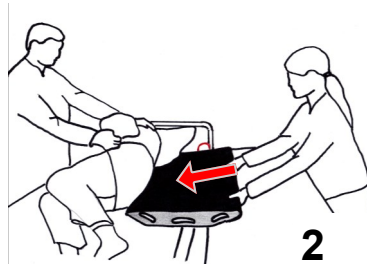


2

The second helper places the Master BOARD Medium so that it reaches from the head down to the hip area. Be careful that the board goes right up to the top of the user's head, as otherwise the head may be rotated sharply during movement.




3

The user is now rolled back-wards onto the Master BOARD Medium, so that it covers the upper half of the user's body.

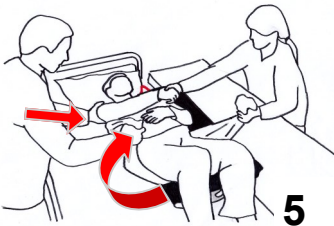


5

One helper grips the sheet near the hip area, pushes the shoulder, lifts the sheet up slightly and rolls the user's weight up and into the Master BOARD Medium.

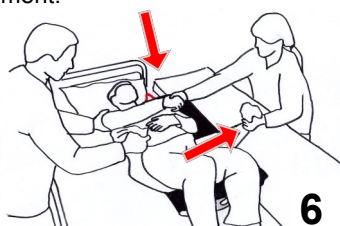

6


At the same time the helper on the opposite side pulls the sheet in the shoulder and hip areas. If the user is active, he or she can grip the helper's hand.

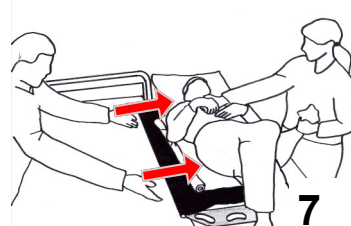

7

The surface that the user was originally lying on is lifted slightly higher than the one he or she is to be transferred to. The user is then very smoothly transferred onto the new surface, while the Master BOARD Medium rolls with the user.

Make sure that the user's legs are moved at the same time, either by the user or by the helpers.

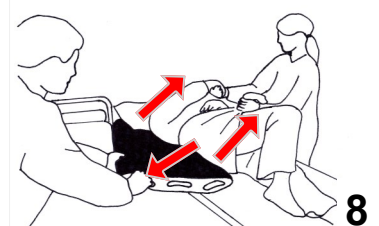

8

One of the helpers now rolls the user's weight onto the side, while the other helper takes hold of the lower handle of the Master BOARD Medium. The board is then pulled very smoothly away from under the user.


Master BOARD[®]




The Original Scandinavian Human Handling System

USER GUIDE


A. Preparation of Master BOARD[®] Large

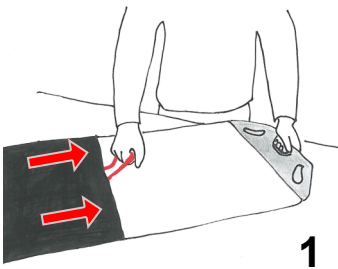

1
The enclosed nylon cover is placed over Master BOARD Large.

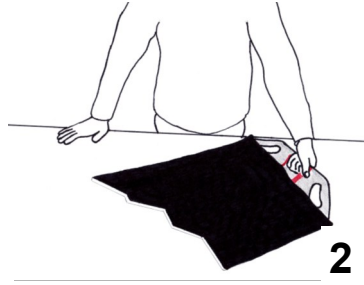

2
Master BOARD Large is now ready for use.

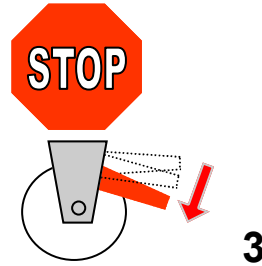

3
Before transfer is attempted, the wheels on both trolleys **MUST** be locked.


B. Positioning of Master BOARD[®] Large




1
Turn the user onto the desired side.




2
If possible you may use a sheet for turning.

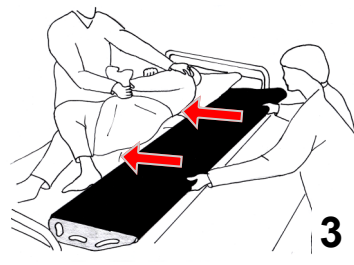


3
While one of the helpers holds the user in the side position, the other user positions the Master BOARD Large so that it reaches from the user's head down to the hips and legs.

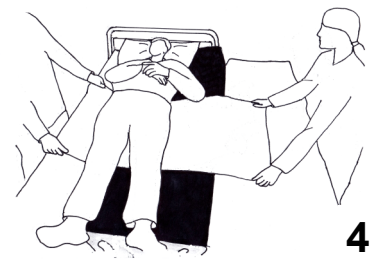


4
The user is now rolled backwards onto the Master BOARD Large. Make sure that the head and one leg are up on the Master BOARD Large.

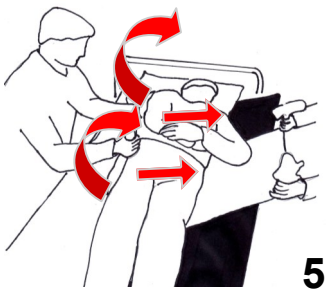


5
One helper grips the sheet near the hip area, pushes the shoulder, lifts the sheet up slightly and rolls the user's weight up and into the Master BOARD Large. The longer the sheet is, the easier the transfer will be.

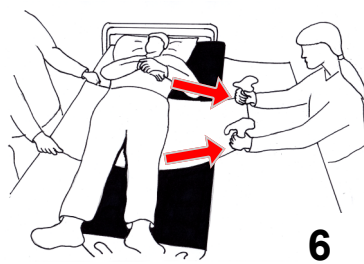

6
At the same time, the helper on the opposite side pulls the sheet in the shoulder and hip areas. Transfer to the Master BOARD Large begins.

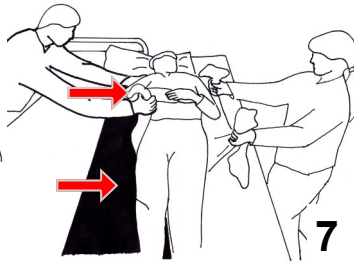

7
The trolley that the user was originally lying on is raised slightly higher than the one he or she is to be transferred to. The user is then transferred very smoothly onto the new trolley, while the Master BOARD Large rolls with the user. Make sure that the user's legs are moved at the same time, either by the user or by the helpers.

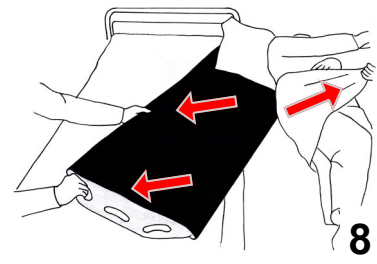

8
One of the helpers now rolls the user's weight onto his or her side, while the other takes hold of the lower handle of the Master BOARD Large. The board is then pulled very smoothly away from under the user.

Master BOARD®



The Original Scandinavian Human Handling System

Description of the models

Master BOARD Medium®

Consists of a padded stiffened board, that can be separated and be washed.

Master BOARD Medium is a two-point board, which covers the hip, shoulder and head area. Here the user can either move his or her legs, or a helper can move them.

Master BOARD Large®

Consists of a padded stiffened board, that can be separated and be washed.

Master BOARD Multi Large is a three-point board, which covers the whole length of the body.

Washing Instructions


Master BOARD® can be washed at 80°C.



Do not use fabric softener as this reduces the sliding effect.

Max. userweight

Body weight: 160 kg with a maximum 20 cm transfer distance.

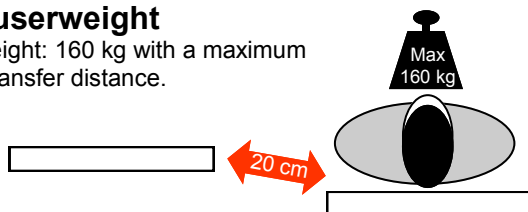

IMPORTANT

Always ensure that this Master CARE aid is intact by periodically inspecting the seams.

If this aid is severely worn, or has any other kind of damage, it should not be used.

If you are in doubt about the use of this aid, you should immediately contact Master CARE

LIABILITY

If this aid is used incorrectly or inappropriately, Master CARE will not be responsible for any accidents resulting in serious injury.

Product Description

Master BOARD® consists of a padded stiffened transfer board.

Over the Master BOARD® a nylon cover is used, which rolls and slides with the board. In this way a rolling movement under the user is created.

Master BOARD Multi® can be used for lying transfer between beds, trolleys, reclining arm-chairs and wheelchairs that can be put down. What is unique about Master BOARD® is that it creates a bridge between surfaces that are a small distance apart or have a small height difference.

For storage of Master BOARD® is used:

Master RACK Peg
Art. No: 002-1282



If there are stringent hygiene requirements, a disposable cover can be ordered:

Art. No. MC 002-1262 Medium (50)
Art. No. MC 002-1251 Large (40)
Art. No. MC 002-1252 Large (50)



For storage of Disposable Covers is used:

Master RACK Wallstand
Art. No. MC 002-1280



Master CARE
Sofienlystvej 3
DK-8340 Malling
Denmark



Tel. +45 86 93 85 85
Fax. +45 86 93 85 05
E-mail: info@master-care.dk
Web: www.master-care.dk