

# One Way MASTER<sup>Open</sup>®



The ORIGINAL Scandinavian Slide and Turn System

Art. no. MC 001-1365  
Hmi no. 42101  
One Way Master AIR Small  
Size: L 80 x W 35 cm  
Number of handles: 5

Art. no. MC 001-1370  
Hmi no. 42102  
One Way Master AIR Medium  
Size: L 100 x W 40 cm  
Number of handles: 5

Art. no. MC 001-1375  
Hmi no. 42103  
One Way Master AIR Large  
Size: L 120 x W 60 cm  
Number of handles: 5




Art. no. MC 001-1265  
Hmi no. 34162  
One Way Master Small  
Size: L 80 x W 35 cm  
Number of handles: 5

Art. no. MC 001-1270  
Hmi no. 33243  
One Way Master Medium  
Size: L 100 x W 40 cm  
Number of handles: 5

Art. no. MC 001-1275  
Hmi no. 34161  
One Way Master Large  
Size: L 120 x W 60 cm  
Number of handles: 5

## When the mobility is impaired...

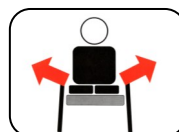

## ... we improve your quality of life!


# Open<sup>®</sup> One Way MASTER

The ORIGINAL Scandinavian Slide and Turn System




## USER GUIDE


### A. Positioning of One Way MASTER Open

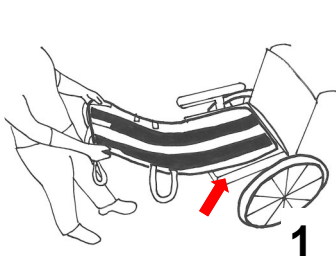

1  
Place One Way MASTER Open with its underside facing the seating surface so that the 3 adjustable white straps face upward.

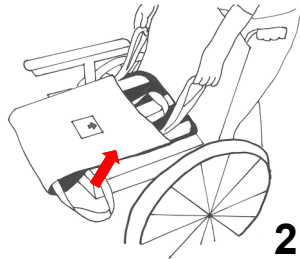


2  
Grasp One Way MASTER Open's front edge and fold it over toward the back of the chair.

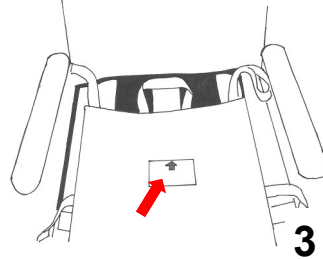

3  
One Way MASTER Open is positioned correctly when the "This Side up and against the back/headrest" label faces upward.


### B. Positioning of User

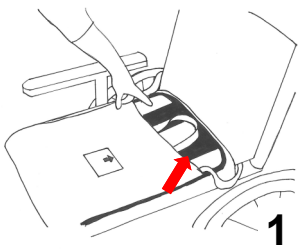

1  
Pull the top of One Way MASTER Open slightly out toward the front edge of the seat of the chair.




2  
Place user on One Way MASTER Open.

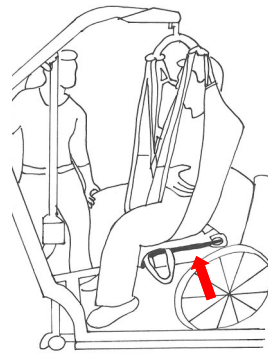


3  
User can also be positioned with the assistance of a sling.




### C. Transfers into Low-Backed Chairs

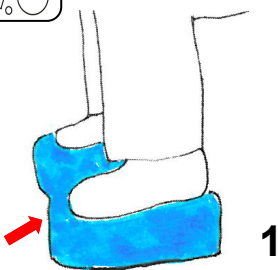

1  
Prevent user's legs from catching on the front edge of the chair of the seat by placing legs in foot supports or by using Master CARE Foot MASTER.

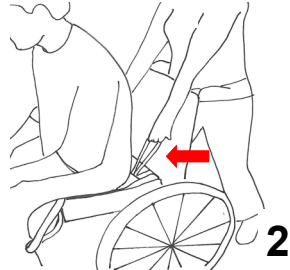

2  
Remove armrest from wheelchair. Grasp One Way MASTER Open's rear edge and pull to help shift weight. It is also possible to pull alternately on the left- and right-hand sides until user is positioned all the way back in the wheelchair.

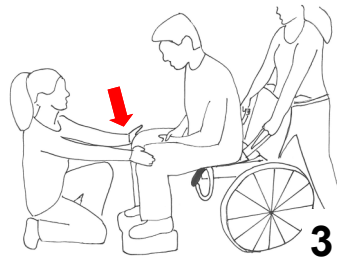

3  
One Way MASTER Open helps user slide back on the seating surface. Two helpers should be present to ensure user safety, with your colleague assisting in supporting user from the front.



4  
One Way MASTER Open helps move user farther back in the chair.

# Open<sup>®</sup> One Way MASTER


The ORIGINAL Scandinavian Slide and Turn System



## USER GUIDE



### D. Transfers into High-Backed Chairs

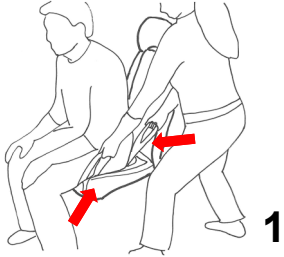

1  
User sits on One Way MASTER Open in a high-backed chair. Grasp the rear handle as well as a side handle, using both of your hands.




2  
Lean back to help shift weight. Use One Way MASTER Open to pull user back toward the side of the seating surface on which you are standing. Repeat process from the opposite side. Pull from alternate sides until user is positioned correctly on the seat of the chair.




3  
If you are unsure whether user has good balance, we recommend that there be two helpers, with your colleague steadying user from the front. There can also be helpers on either side of the wheelchair.




### E. Change Sitting Position

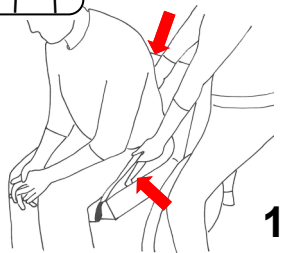


1  
Position yourself on the side of the wheelchair. Grasp the corner handle below the knee closest to you as well as either the opposite corner handle or the middle rear handle.

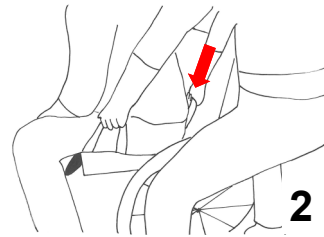

2  
Pull either the handle farthest from you or the middle rear handle. User's torso leans toward you.

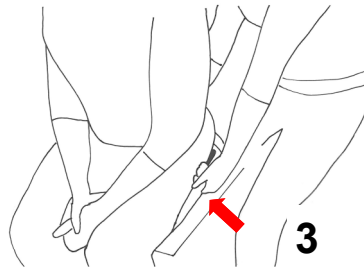


3  
Shift user by pulling the handle closest to you. One Way MASTER Open helps user glide sideways toward you on the seating surface. This relieves pressure on user's thigh that is closest to you.

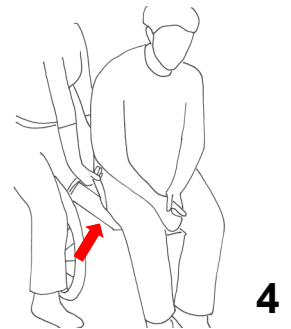


4  
To relieve pressure on the opposite thigh, simply go over to the other side of the chair and shift user utilising the same technique. **If user becomes too heavy or passive, we recommend switching to a solution that uses a sling.**



### F. Transfers into Mechanical Rotating Chairs

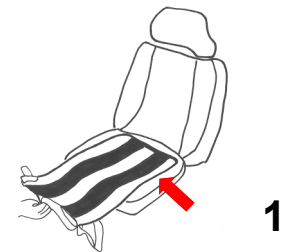

1  
Place One Way MASTER Open with its underside facing the seating surface so that the 3 adjustable white straps face upward. Grasp One Way MASTER Open's front edge and fold it over toward the back of the chair.

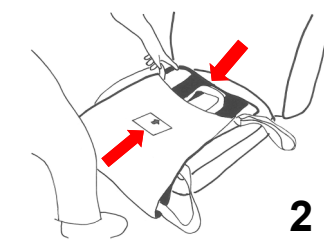

2  
One Way MASTER Open is positioned correctly when the "This Side up and against the back/headrest" label faces upward. Pull the top of One Way MASTER Open slightly out, and place user on One Way MASTER Open.



3  
Position yourself beside one of user's legs, and place your hand on user's knee. User leans slightly away from you. Push user's knee alternately from one side and the other so that user slides into the seat with the help of the buttocks tracks.



4  
Finally, turn the seat, and allow user to remain seated on One Way MASTER Open for the duration of the drive. **When driving with One Way MASTER Open, ALWAYS REMEMBER TO WEAR A SEAT-BELT.**

# Open One Way MASTER<sup>®</sup>



The ORIGINAL Scandinavian Slide and Turn System

## Descriptions of the Different Models

### (1) One Way MASTER AIR Open

Consists of a thin, open sliding pillow with a cellular structured netted top. This ensures optimal ventilation and prevents user from becoming too hot.


### (2) One Way MASTER Open

Consists of a thin, padded open sliding pillow with a velour top.

## Washing Instructions

One Way MASTER Open can be washed at 80° C.



 Do not use washing detergent as this may lessen the product's sliding effect.

## WARNING

When driving with One Way MASTER Open, ALWAYS REMEMBER TO WEAR A SEATBELT that is permanently fastened to the car.



## IMPORTANT

Make regular checks to ensure that this Master CARE assistive technology is intact.

If this assistive technology has undergone serious wear or exhibits other signs of damage, do not use it.

If in doubt about the usability of this assistive technology, contact Master CARE immediately.

## RESPONSIBILITY

In the event that this assistive technology is used incorrectly or inappropriately, Master CARE takes no responsibility for accidents resulting in serious personal injury.

## Product Description

One Way MASTER Open consists of thin, open sliding pillows.

One Way MASTER Open is used as a permanent sliding surface and is ideal for users who need to slide back and forth frequently, for example in a wheelchair.

One Way MASTER Open makes it easy for helpers and comfortable for users to attain the correct position on the wheelchair seat. The inner strap is placed in such a way that One Way MASTER Open slides exceptionally easily and possesses a uniquely effective fastening function.

Customers who buy One Way MASTER also buy:  
**Foot MASTER**



Art. no. MC 006-8090  
Hmi no. 52014  
Foot MASTER  
Size: L 38 x W 32 x H 10 cm



Master CARE  
Sofienlystvej 3  
DK-8340 Malling  
Denmark



Tel. +45 86 93 85 85  
Fax. +45 86 93 85 05  
info@master-care.dk  
www.master-care.dk