

Sling MASTER[®]



The ORIGINAL Scandinavian Human Handling System



Art. no. MC 003-9005
Hmi no. [55767](#)
Sling MASTER X-small
Dimensions: L 150 x W 60 cm
Number of handles: 10

Art. no. MC 003-9000
Hmi no. [46707](#)
Sling MASTER Small
Dimensions: L 200 x W 60 cm
Number of handles: 14

Art. no. MC 003-9010
Hmi no. [45043](#)
Sling MASTER Medium
Dimensions: L 200 x W 80 cm
Number of handles: 14

Art. no. MC 003-9020
Hmi no. [45044](#)
Sling MASTER Large
Dimensions: L 200 x W 100 cm
Number of handles: 14

When mobility is impaired ...



... We improve your quality of life !


Sling MASTER[®]




The Original Scandinavian Human Handling System

User Guide


A. Transfer back into the chair

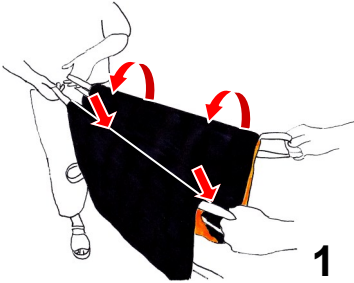


1
Hold the Sling MASTER by gripping the two grey handles in the middle of it. Then grip the two white handles and at the same time let the two grey handles drop down into the middle.

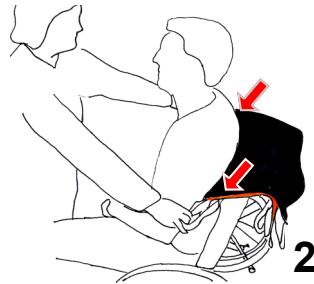


2
Then slide the Sling MASTER behind the user's back and down to the buttock area.

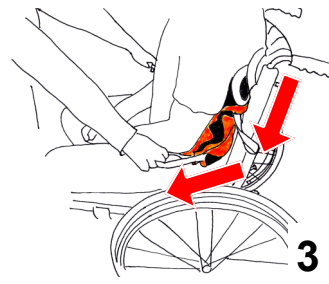

3
Tighten the Sling MASTER and allow it to slide in and under the user's buttock area with sawing movements.

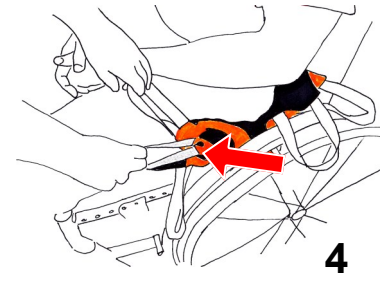


4
Finally unfold the Sling MASTER under the user's buttock area by gripping the grey handles and pulling them down first on one side and then on the other.

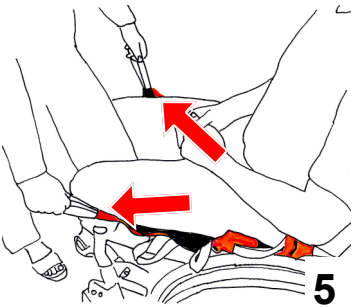


5
Always stand in front of the user to prevent the person from sliding forward in the chair.


If necessary be two helpers

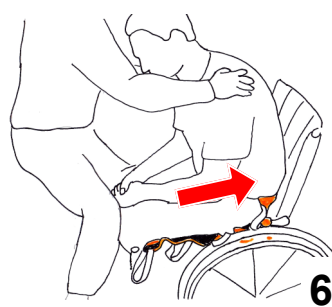

6
The user leans the upper body forward, while the helper pushes and allows the user to slide alternately from side to side back into the chair. It is important that the user places his or her feet on the footplates or on Foot MASTER

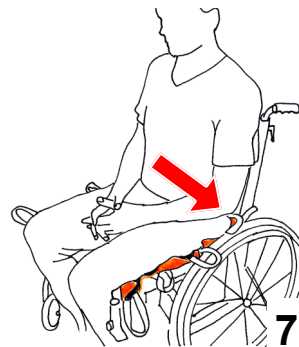


7
The user has now slid back into the chair using the sling MASTER.

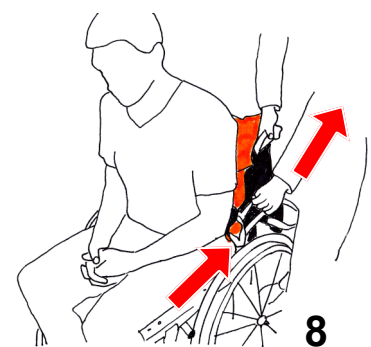


8
Sling MASTER is removed by gripping the handles and pulling it under the user's hip first in the one side and then in the other side. Now grip in the handles in the lower part of Sling MASTER and pull alternately in these until Sling MASTER has glided out under the user


B. Placing the Sling MASTER under the shoulders of a user lying in bed (Before placing a lifting sling)

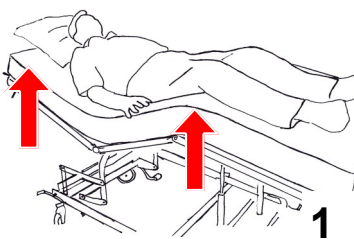


1
Raise the bed under the user's knees and elevate the head area slightly.

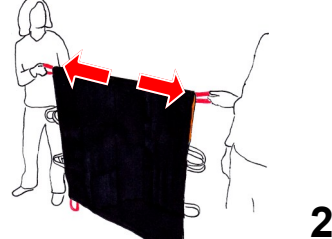

2
Hold the Sling MASTER by gripping the two grey handles in the middle of it.

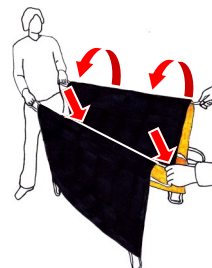

3
Then grip the two white handles and at the same time let the two grey handles drop down into the middle.

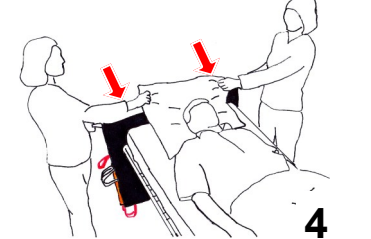

4
Hold both of the Sling MASTER's white handles in one hand. With the other hand, take of the pillow and the Sling MASTER will slide in under it.


Sling MASTER[®]



The Original Scandinavian Human Handling System


User Guide

C. Positioning the Sling MASTER under the lower back

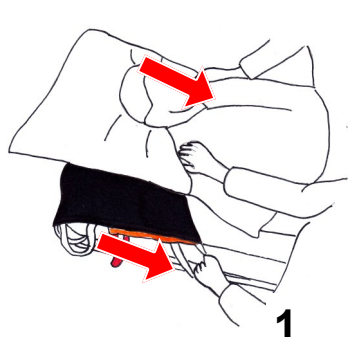


1

The helpers pull in turn on the white handles, making sawing movements, until the Sling MASTER has passed underneath the user's shoulder blades.

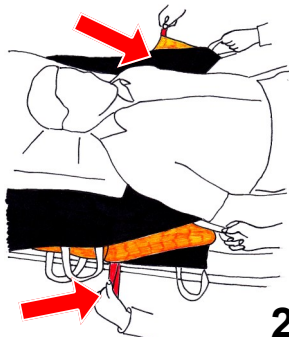

2

The helpers now grip the lowest grey handles on the Sling MASTER and this unfolds and glides down towards the user's lower back.

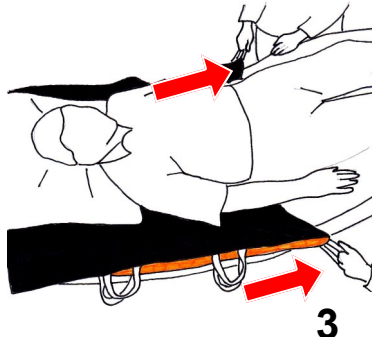

3


The Sling MASTER is now in place under the user from the lower back upwards.



4


If the user is already lying on the MC 001-1940 Silk MASTER Glide Sheet, it will be extremely easy to move the Sling MASTER into place.

D. Positioning the lifting sling

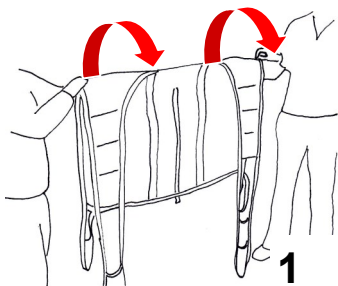


1

Fold the lifting sling up so that the back of the sling hangs down with its outer side against the headboard, with the foot straps hanging down to the side that is turned towards the user's headboard.

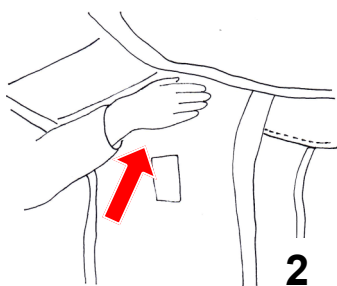


2

The helpers put the palms of their hands under the lifting sling where it is folded.

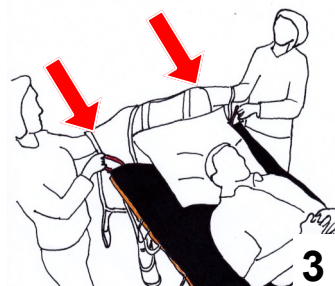


3

The helpers hold on to the two top handles of the Sling MASTER and push the sling a little between the two sliding surfaces.

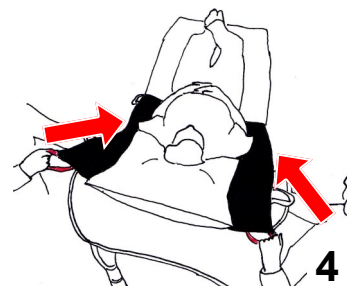


4

Then the helpers hold the two red handles on the Sling MASTER, at the same time moving the lifting sling down to the user's midriff, so that the sliding sling slips into the Sling MASTER.

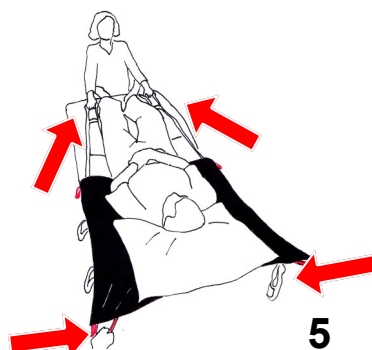


5

One helper pulls the leg straps on the lifting sling, while the other holds the handle on the opposite end of the Sling MASTER. The lifting sling now glides into place.

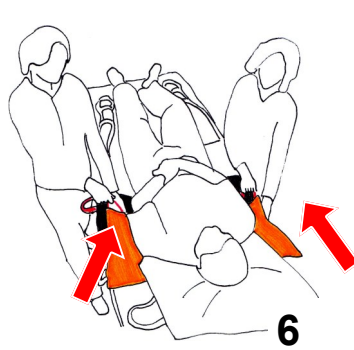

6

Take hold of the grey handles at the end of the Sling MASTER. Fold that part up, then pull and slip this part down towards the user's lower back.

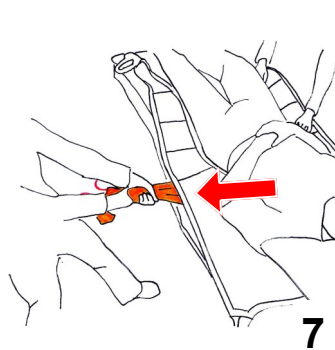

7

The Sling MASTER is removed simply by pulling on the lowest sliding piece and rolling this down under the user.

Sling MASTER[®]



The Original Scandinavian Human Handling System

Description of the different models

Sling MASTER[®] X-small

Sling MASTER X-small is ideal if the user is to be transferred back into a chair.

Sling MASTER[®] Small

Sling MASTER Small is ideal if users are to be transferred back into a chair or to place the lifting sling under users lying in bed.

Sling MASTER[®] Medium and Large

Sling MASTER Medium and large are used to place the lifting sling under users lying in bed.

Washing instructions

Sling MASTER can be washed at 80°C.



Do not use fabric conditioner, as this will reduce the sliding effect.

Max. Userweight

Body weight: 250 kg



WARNING



Never leave users sitting on the Sling MASTER. This is to avoid accidents that may result in serious injury.

IMPORTANT

Always ensure that this Master CARE aid is intact by periodically inspecting the seams.

If this aid is severely worn, or has any other kind of damage, it should not be used.

If you are in doubt about the use of this aid, you should immediately contact Master CARE.

LIABILITY

If this aid is used incorrectly or inappropriately, Master CARE will not be responsible for any accidents resulting in serious injury.

Product Description

Sling MASTER[®] consists of a thin sliding sheet. It is supplied with handles for easy handling. Sling MASTER is used to slide users back into a wheelchair or place a lifting sling under users lying in bed. If the user is lying on Silk MASTER or Arcus Vita Silk Sheets, it is amazingly easy to place the Sling MASTER[®] under the user. It can also be used to place an x-ray box or move the user higher up in bed.



Master CARE
Sofienlystvej 3
DK-8340 Malling
Denmark



Tel. +45 86 93 85 85
Fax. +45 86 93 85 05
E-mail: info@master-care.dk
www.master-care.dk