

# Sling VEST<sup>®</sup>



The ORIGINAL Slide cushion for slings

PAT/PEND/201000218/Europatent





- Art. no. MC 003-9040  
Hmi no. [48844](#)  
Sling VEST Medium Basic  
Dimensions: L 70 x W 50 cm



- Art. no. MC 003-9041  
Hmi no. [55719](#)  
Sling VEST Large Basic  
Dimensions: L 75 x W 60 cm

## When mobility is impaired

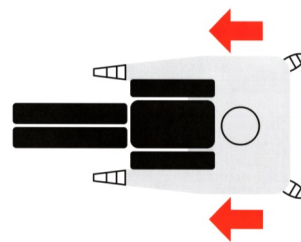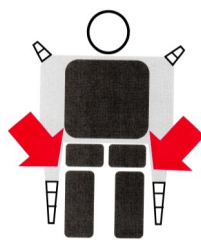

## ... we improve your quality of life!


# Sling VEST



The ORIGINAL Slide cushion for slings


## User guide

### A. Attaching the Sling VEST to the lifting sling

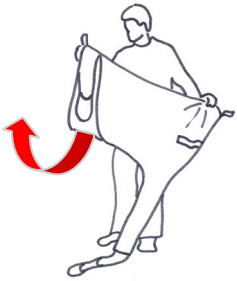


1

Lift up one of the foot straps of the lifting sling and hold it together with the sling.

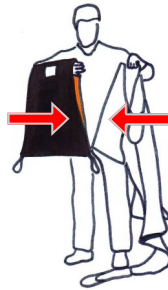

2

Put the Sling VEST in over the lifting sling. Make sure that the broad part with the handles is facing down.

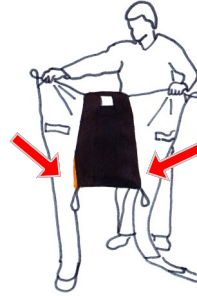

3

Then let go of the foot strap and the Sling VEST is now in place.

### B. Placing the lifting sling and Sling VEST into a chair



1

**ONE HELPER.**  
Stand behind the wheelchair. Hold the Sling VEST and the lifting sling together, making them tight, and push the sling down between the user's back and the back of the chair, first starting diagonally down towards one side.




2

Hold the sling straight out and then pull it down towards the other side and continue in turn until the sling is positioned down by the user's lower back.



3

Then place your hand into the Sling VEST and push it down and around towards the hips on both sides. If possible, remove the armrests in order to have better access.

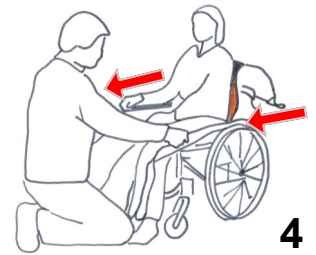


4

Finally, sitting in front of the user pull the leg straps of the lifting sling. It will then all slide into place.



5

**TWO HELPERS.**  
The helpers position themselves on each side of the wheelchair. The lifting sling and the Sling VEST are held tightly together.

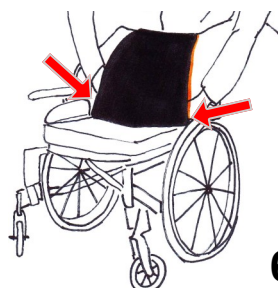

6

Hold the sides of the sling and put your hands into the Sling VEST from the sides.



7

The Sling VEST with the lifting sling is now pushed down between the user and the back of the chair.



8

If possible, take hold of the armrests of the wheelchair's, and finally push the lifting sling with the Sling VEST into place around the user's hips.

# Sling VEST




The ORIGINAL Slide cushion for slings

## User guide


### C. Removing the lifting sling and Sling VEST from the chair




1  
Pull the leg straps down and back at the sides.

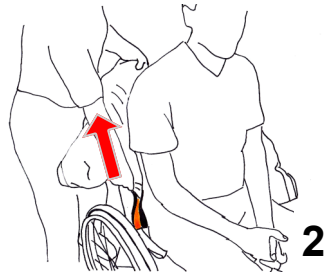

2  
Pull the lifting sling up. This will now slide into the Sling MASTER and up towards the back of the wheelchair.

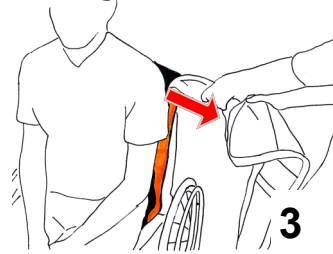


3  
Hold the sling folded together and let it slide out to one side. The other helper takes hold of the edge of the Sling VEST on the opposite side.

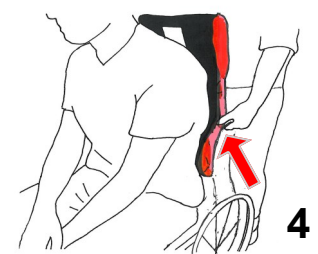


4  
Take hold of the lowest sides of the Sling VEST which are turned in towards the back of the wheelchair and roll them up, in the way one removes a pipe-shaped sliding cushion. Sling VEST is very easy to remove.


### D. Placing the Sling VEST using the Master TURNER DUO divisible Glide- and a turning sheet.

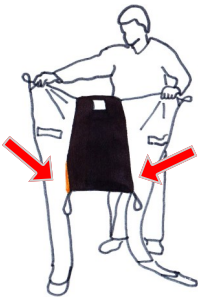


1  
Place the Sling VEST on the lifting sling as shown in Fig. A Points 1-3.

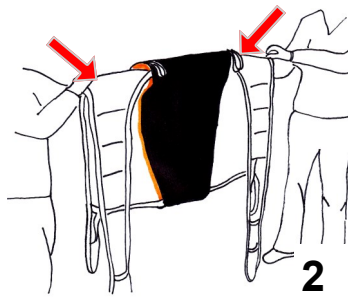

2  
Helpers put their palms into the sling by the hip parts and fold it so that the leg straps drop down on the other side.

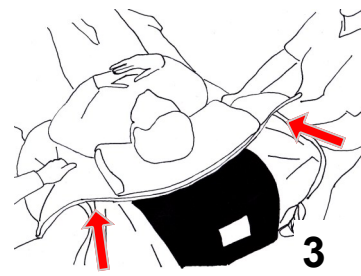


3  
Lift the upper part of the Master TURNER DUO "Top Piece" and hold it with one hand. Now start to position the lifting sling by pushing and letting the Sling VEST slide down between the sliding sections of the Master TURNER DUO "Top Piece" and the smooth upper surface of the corresponding mattress cover.

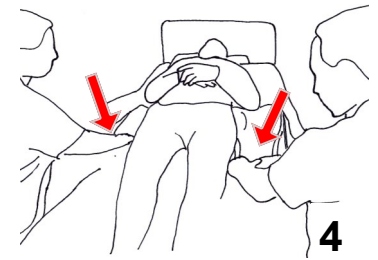

4  
Separate the "Bottom Piece" from the "Top Piece" of the Master TURNER DUO and finally roll the "Bottom Piece" down and under the knees of the user.




5  
Take the leg straps down along the sides and position the lifting sling by pulling it into place at the sides by the user's hips.

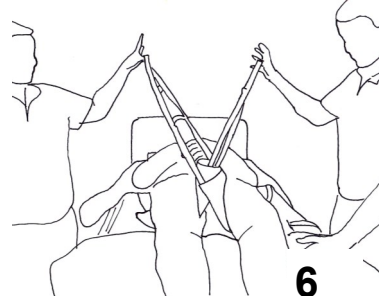

6  
You may use MC 003-9031 Sling SOCK to make it easier to positioning the lifting sling under the legs of the user and finally be ready for lifting. The "Top Piece" now go with and can easily be removed when the user is sitting in the chair.

### E. Fjerne Sling VEST

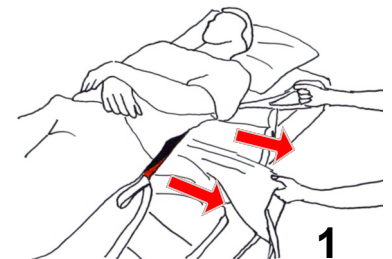

1  
The lifting sling can easily be removed from the Sling VEST by simply pulling it out to one side, while the other helper holds the edge of the Sling VEST on the opposite side. Finally, the Sling VEST is rolled or pulled out from under the user.

# Sling VEST



The ORIGINAL Slide cushion for slings

## Description of Sling VEST® BASIC

### Sling VEST® BASIC

Consists of a padded sleeve with smooth exterior and interior. The sliding material is a specially manufactured PU coated nylon HLF (Hyper Low Friction) Trademark, known for its unique sliding properties and durability.

### Washing Instructions

Sling MASTER can be washed at 80°C.



Do not use fabric conditioner as this reduces the sliding effect.

### Max. User Weight

Body weight: 250 kg



## Product Description

Sling VEST® consists of a padded slider in the form of a sleeve. The lifting sling is placed inside Sling VEST®, so that it is smooth on the back. This makes the sling very easy to position and remove under a user in lying or sitting position. This takes place without friction or any need for the helpers to lift. Sling VEST® gives a high level of comfort and furthermore prevents the lifting sling from cutting into the bare skin at the seat of the user.

## IMPORTANT

Always ensure that this Master CARE aid is intact by periodically inspecting the seams.

If this aid is severely worn, or has any other kind of damage, it should not be used.

If you are in doubt about the use of this aid, you should immediately contact Master CARE.

## LIABILITY

If this aid is used incorrectly or inappropriately, Master CARE will not be responsible for any accidents resulting in serious injury..



Master CARE  
Sofienlystvej 3  
DK-8340 Malling  
Denmark



Tel. +45 86 93 85 85  
Fax. +45 86 93 85 05  
E-mail: info@master-care.dk  
Web: www.master-care.dk