

Turn MASTER[®]

The ORIGINAL Scandinavian Slide and Turn System



Art. No. MC 001-9040
Hmi No. 39010
Turn MASTER Classic
Transfer Disc Small
Dimensions: Diam 40 cm
Number of Handles: 0

Art. No. MC 001-9050
Hmi No. 39011
Turn MASTER Classic
Transfer Disc Medium
Dimensions: Diam 50 cm
Number of Handles: 0

Turn MASTER[®] Classic



Art. No. MC 001-9060
Hmi No. 39012
Turn MASTER Classic
Transfer Disc Large
Dimensions: Diam 60 cm
Number of Handles: 0

Art. No. MC 001-9070
Hmi No. 45925
Turn MASTER Classic
Transfer Disc X-large
Dimensions: Diam 70 cm
Number of Handles: 0

Art. No. MC 001-9080
Hmi No. 45926
Turn MASTER Classic
Transfer Disc XX-large
Dimensions: Diam 80 cm
Number of Handles: 0

Art. No. MC 001-9041
Hmi No. 55563
Turn MASTER Multi
Transfer Disc X-small
Dimensions: Diam 40 cm
Number of Handles: 2

Art. No. MC 001-9045
Hmi No. 54608
Turn MASTER Multi
Transfer Disc Small Plus
Dimensions: Diam 45 cm
Number of Handles: 2

Art. No. MC 001-9051
Hmi No. 55564
Turn MASTER Multi
Transfer Disc Medium
Dimensions: Diam 50 cm
Number of Handles: 3

Turn MASTER[®] Multi





Art. No. MC 001-9061
Hmi No. 55565
Turn MASTER Multi
Transfer Disc Large
Dimensions: Diam 60 cm
Number of Handles: 3

Art. No. MC 001-9071
Hmi No. 55566
Turn MASTER Multi
Transfer Disc X-large
Dimensions: Diam 70 cm
Number of Handles: 3

Art. No. MC 001-9081
Hmi No. 55567
Turn MASTER Multi
Transfer Disc XX-large
Dimensions: Diam 80 cm
Number of Handles: 3

When mobility is impaired ...

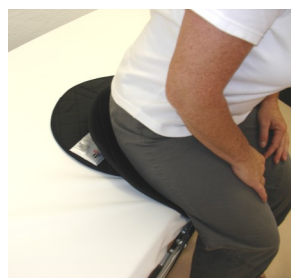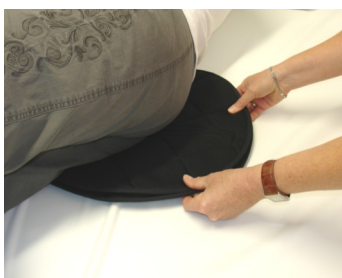

... we improve your quality of life !


Turn MASTER[®]

The ORIGINAL Scandinavian Slide and Turn System




USER GUIDE

A. Turn MASTER[®] (Turn on Chair)

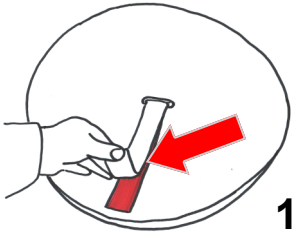

1
Tighten the Velcro band on the underside of the lower sliding part so that Turn MASTER[®] can do nothing but rotate. In this case, ignore the "This Side up and against the back/headrest" label.

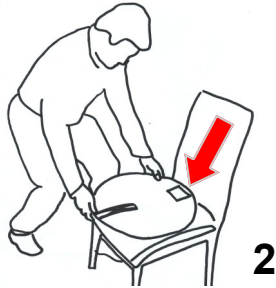


2
Place Turn MASTER[®] on the seat with the nonslip sliding part below and the black netted sliding part above.




3
Sit down on Turn MASTER[®], and you can now rotate on the seat.


B. Turn MASTER[®] (Turn in Bed)

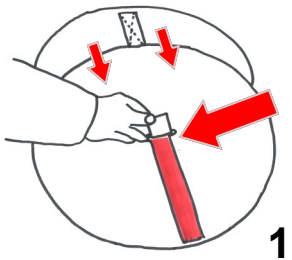

1
Lower the bed as far down as possible. Loosen the Velcro band on underside of the lower sliding part of Turn MASTER[®]. Pull the lower sliding part out to the desired place by adjusting the band.

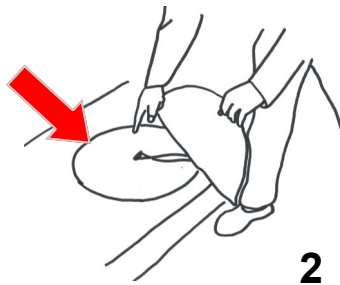

2
Place the lower sliding part in the centre of the bed.

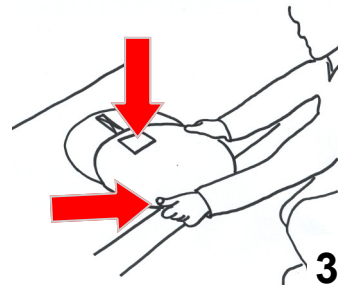


3
Move the upper sliding part out to the edge of the bed. The "This Side up and against the back/headrest" label now points away from the centre of the bed.


WARNING



In order to avoid accidents resulting in serious personal injury when using Turn MASTER[®], ensure that the bed is low and that user is not sitting too close to the edge of the bed.

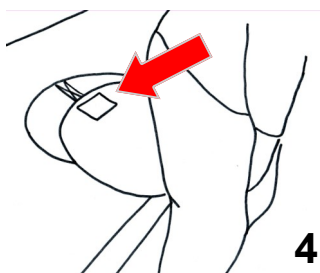

4
User sits on Turn MASTER[®] with buttocks facing the "This Side up and against the back/headrest" label.

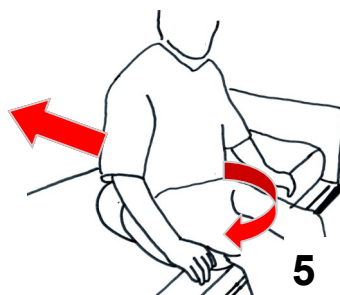

5
User now slides back toward the centre of the bed unassisted, rotates, swings legs onto the bed, and slides into the desired position.



6
Turn MASTER[®] "Multi" allows one helper to assist by taking the handles on the back portion of the underside of the upper sliding part while another helper moves user's legs.


Turn MASTER[®]

The ORIGINAL Scandinavian Slide and Turn System




USER GUIDE


D. Turn MASTER[®] (Remove Turn MASTER[®] from Bed)

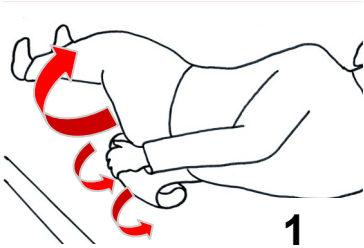

1
User rolls onto one side, takes hold of both sliding parts on Turn MASTER[®], and rolls it down twice toward her/his own body.

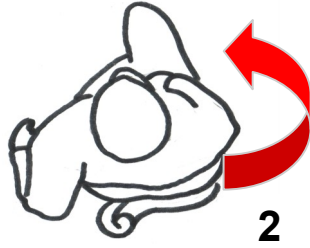


2
User then rolls onto the other side, over the rolled-up Turn MASTER[®].

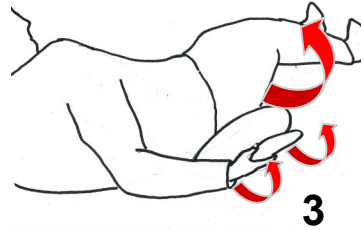


3
User takes hold of the lower sliding part of Turn MASTER[®], folds it out, and pulls it away.


E. Turn MASTER[®] (Get Into and Out of Car Seat)

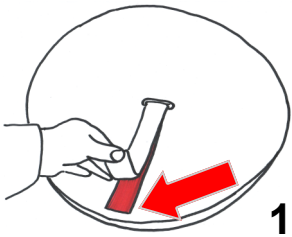


1
If you want Turn MASTER[®] to do nothing but rotate, tighten the Velcro band on the bottom of the nonslip sliding part all the way up to the edge of the sliding cushion.

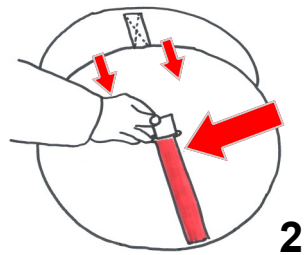


2
If you want Turn MASTER[®] to move the buttocks from the edge of the car seat and in toward the backrest, loosen the Velcro band, and adjust it as necessary.

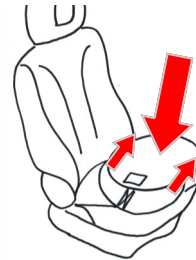

3
Place Turn MASTER[®] on the car seat with the nonslip sliding part below. Guide the upper sliding part with the netting out toward the edge of the seat.

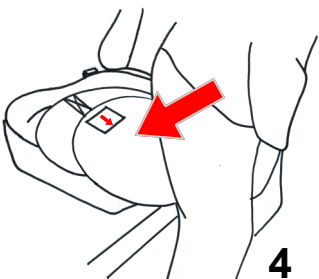


4
Position yourself in doorway, facing away from the seat. Sit down on Turn MASTER[®]. The "This Side up and against the back/headrest" label should be pointing toward your buttocks.

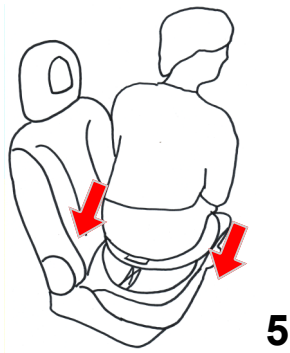

5
Now you can slide toward the centre of the seat.



6
Rotate and slide your buttocks toward the backrest, by the Turn MASTER[®] on the seat.

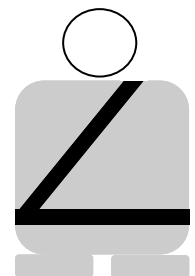

WARNING

ALWAYS wear a seatbelt when driving with Turn MASTER[®].

Turn MASTER[®]

The ORIGINAL Scandinavian Slide and Turn System



Descriptions of the Different Models

(1) Turn MASTER[®] Classic

Consists of an adjustable turning cushion with a specially constructed netted surface that provides the sliding disc with ventilation and prevents it from folding. The lower sliding part has a nonslip covering that lets it sit firmly on any surface. The two sliding parts are connected by a central band, which can be adjusted from the underside of the lower sliding part. This permits rotation and transfer in a way that replicates natural patterns of movement.


(2) Turn MASTER[®] Multi

Consists of an adjustable turning cushion with a specially constructed netted surface with 3 handles, that provides the sliding disc with ventilation and prevents it from folding. The upper sliding part has two or three handles, which allow helpers to assist in user transfer. The two sliding parts are connected by a central band, which can be adjusted from the underside of the lower sliding part. This permits rotation and transfer in a way that replicates natural patterns of movement.

Washing Instructions

Turn MASTER can be washed at 80° C



 Do not use washing detergent as this may lessen the product's sliding effect.

Max. User Weight


Bodyweight: max. 160 kg



WARNING

When driving with Turn MASTER[®], always remember to wear the seatbelt that is permanently fastened to the car.

In order to avoid accidents resulting in serious Personal injury, ensure that Turn MASTER[®] does not slide onto the edge of the bed.

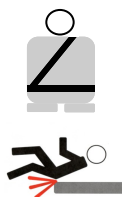

IMPORTANT

Make regular checks to ensure that this Master CARE assistive technology is intact. If this assistive technology has undergone serious wear or exhibits other signs of damage, do not use it.

If in doubt about the usability of this assistive technology, contact Master CARE immediately.

LIABILITY

In the event that this assistive technology is used incorrectly or inappropriately, Master CARE takes no responsibility for accidents resulting in serious personal injury.

Product Descriptions

Turn MASTER[®] consists of an adjustable turning and sliding cushion for use by individuals who desire easier rotation on chairs, in and out of bed, into or out of car seats, etc. To ensure that Turn MASTER[®] stays comfortably cool over long periods and that it does not fold up during use, the upper sliding part is fitted with specially constructed netting. Turn MASTER[®] is unique in that it not only rotates like a normal turning disc but can also be adjusted to assist in, for example, moving sideways. The "Multi" model also comes with two or three handles, which allow helpers to assist in the transfer.



Master CARE
Sofienlystvej 3
DK-8340 Malling
Denmark



Tel. +45 86 93 85 85
Fax. +45 86 93 85 05
E-mail: info@master-care.dk
Web: www.master-care.dk