Brugervejledning / User Guide

Light Turner & Light Cover

○ **MC 002-2700** [HMI: 114964]
Light Turner - **Large**

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
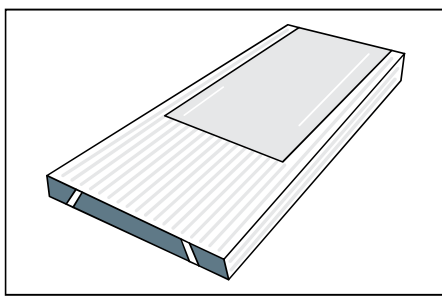
UK Light Cover - Large is fitted on the mattress.

Light Turner placeres ovenpå den glatte side nedad

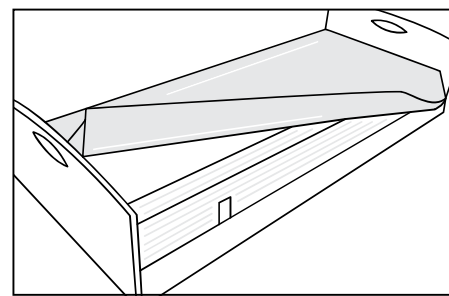
Light Cover - Medium is fitted on the mattress.

Light Turner placeres ovenpå coveret med den glatte side nedad – billedet illustrerer Light Turner og Light Cover - Large.


Place the Light Turner on top of the Light Cover with the smooth side down – the picture illustrates Light Turner and Light Cover - Large.

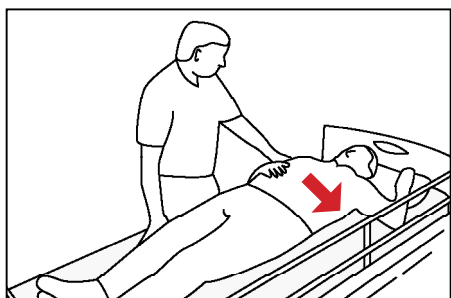2. Manuel vending i seng / Turning in bed manually

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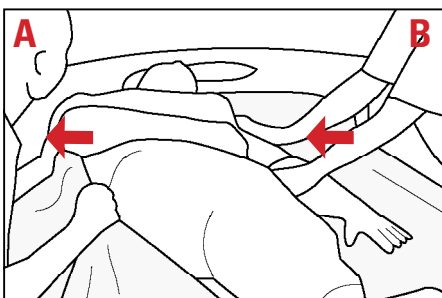
UK Cross users legs. Move the hand to the opposite shoulder. Move the pillow to the side which the user is being turned to.

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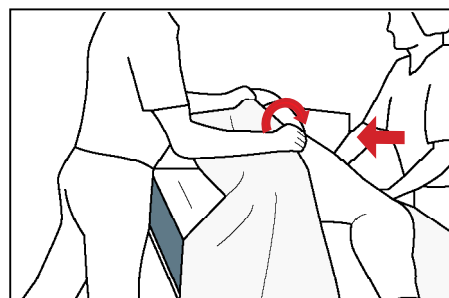A pulls and B push the user towards the edge of the bed.

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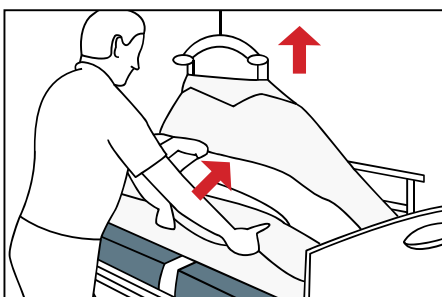
A pulls the turner and draws the user to the side.

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
UK Cross users legs and move pillow forward on the side to which the user is moved. Attach the straps of the Turner to the hoist.

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
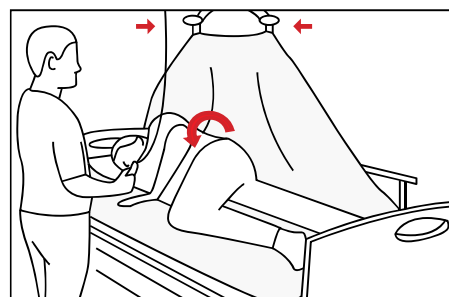
Elevate the user slightly from the mattress. Helper pushes the user to the side. Master Comfort Stomach Wedge can be used for greater side lying comfort.

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Elevate the hoist. Turn the user to the side.

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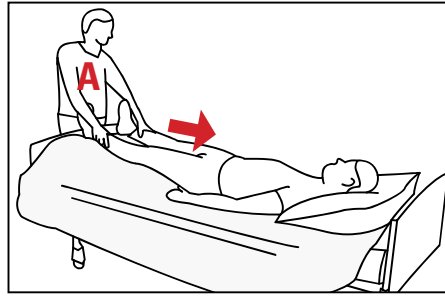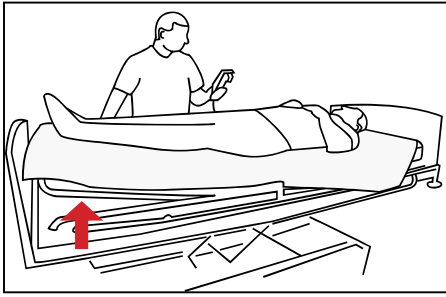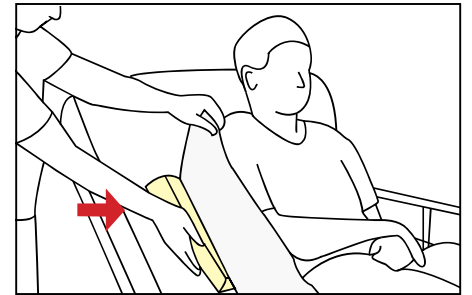
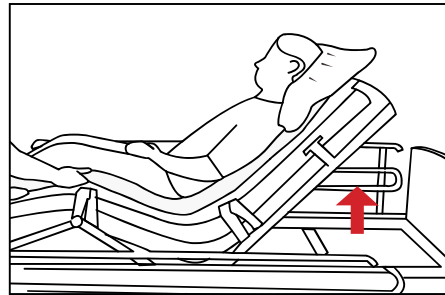
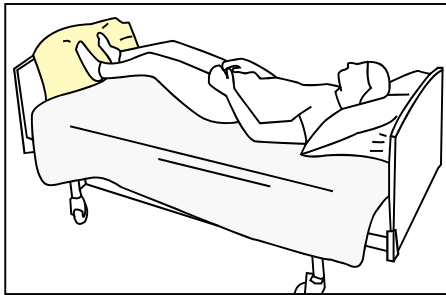
UK

Elevate the foot end of the bed.

A is placed at the end of the bed. With a light push the user slides upwards in the bed.

5. Op at sidde i seng / Sitting up in bed using the Light System

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Placér Master Comfort Støttekiler under vendelagnet i hver side af brugeren.

UK

Place Master Comfort Washing Wedge between the users feet and the foot of the bed and fully activate the adjustable bed under the users knees.

Elevate the head end of the bed.

Place Master Comfort Support Wedges between the Light Turner and the Light Cover under each side of the users torso.

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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