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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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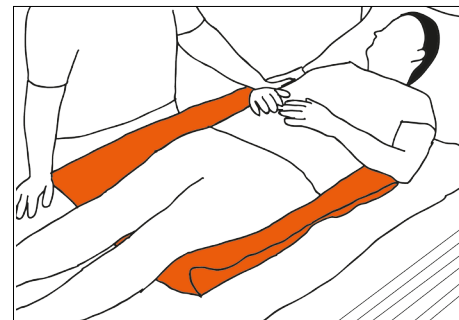
**UK** The patient bends his/her leg and grabs the bed rail. The Nylon Master/Slide Master is folded and inserted under the patient's pressure points

The patient grips the opposite bed rail and the helper takes hold of the Nylon Master/Slide Master and straightens it out

The patient returns to a supine position and the Nylon Master/Slide Master is now positioned correctly under the patient. Friction under the patient's pressure points has been reduced

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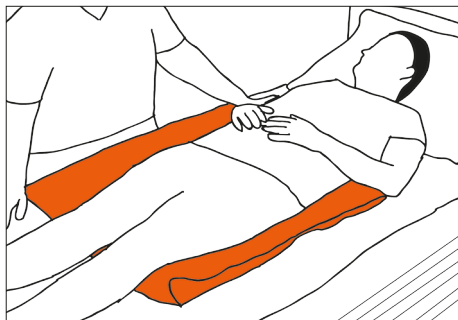
**UK** The helper takes hold of the bottom layer of the Nylon Master/Slide Master

The helper pulls the Nylon Master/Slide Master out using weight transfer, such that the double slide is utilized

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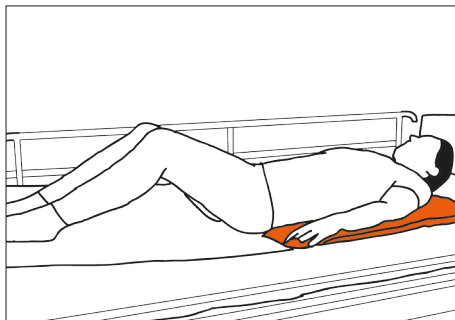
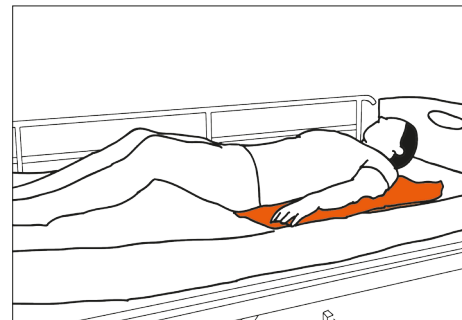
**UK** The Nylon Master/Slide Master is placed under the patient's shoulder and pelvis. See item 1

The patient bends his/her legs and, pushing off, lifts his/her pelvis. The foot may be elevated


Using the Nylon Master/Slide Master, the patient can now slide unaided higher up in the bed

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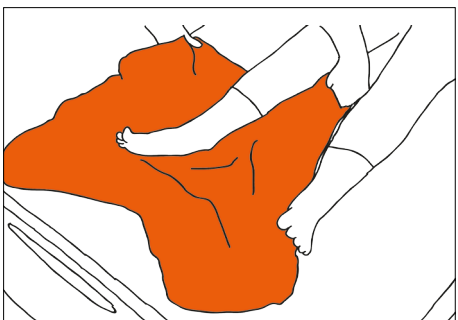**UK** The Nylon Master/Slide Master is placed under the patient's lower leg and pelvis.

The helper places his/her hands on the patient's foot, and in the hollow of the patient's knee.

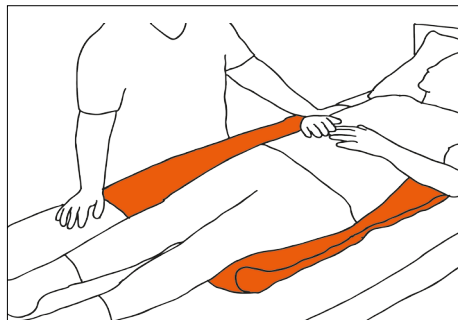
By means of weight transfer, the patient's leg is now pushed up into the desired bent position and the Nylon Master/Slide Master is removed

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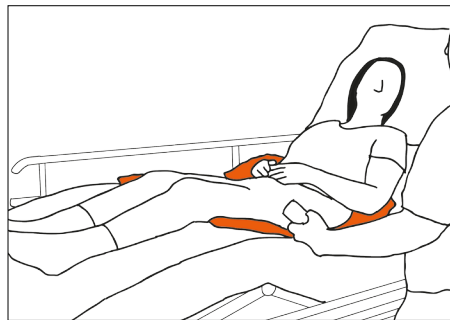
**UK** The Nylon Master/Slide Master is placed under the patient's pelvis (see item 1) at a distance from the bedside of approximately 15 cm

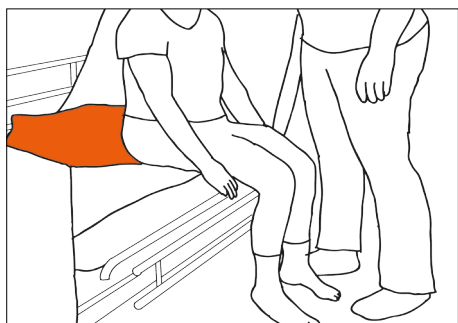
The fold by the patient's knee is activated and the head of the bed is elevated

The helper pulls the patient's legs out over the edge of the bed, while the patient either pushes off with his/her arms or uses the Master Bed Grip. The friction under the patient's pelvis is reduced and the patient can turn around

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
**UK** The bed is lowered so that the patient can reach the floor with his/her feet

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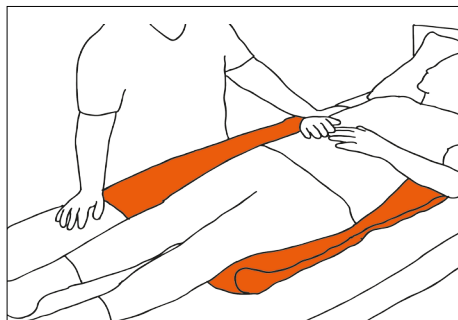**UK** The Nylon Master/Slide Master is placed under the patient's pressure points (see item 1)

Cross or bend the patient's legs and move the pillow to the side to which the patient is to be turned. The patient grips the bed rail while the helper places his/her hands on the patient's shoulder and hip. By using weight transfer, the patient is rolled into a lateral position

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UK

- Smooth on both sides
- Manufactured with extra strong nylon thread
- Easy to position and remove from under the user
- Enables mobilisation of the user
- Wide range of applications
- Soft and does not make noise
- Extremely durable

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**MC 001-8020** [ HMI: 45939 ]

Master Comfort Vaskekile - Medium /  
Washing Wedge / Waschkeil

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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