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○ **MC 002-1242** [HMI: 38144]
Elastic Cover / Cover / Bezug
- **X-Small**

Størrelse / Dimensions / Größe:
L 60 x B/W 65 cm

○ **MC 002-1277** [HMI: 39031]
Elastic Cover / Cover / Bezug
- **Small**

Størrelse / Dimensions / Größe:
L 65 x B/W 75 cm

○ **MC 002-1245** [HMI: 36386]
Elastic Cover / Cover / Bezug
- **Medium**

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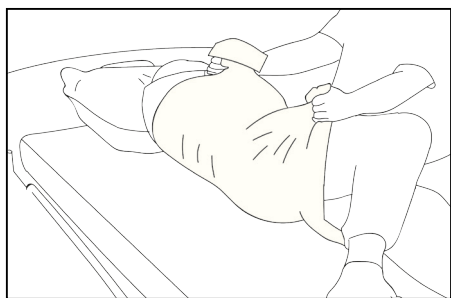
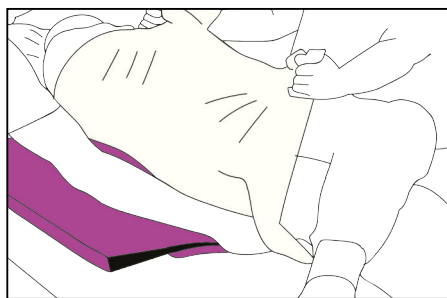
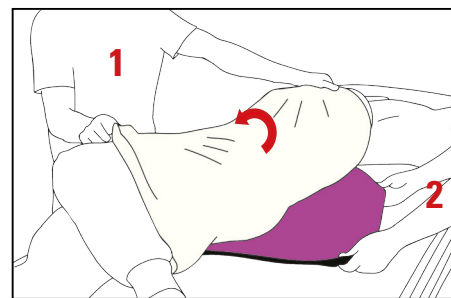
UK Cross or bend the user's legs and grasp the undersheet and use weight transfer to create space for the Slide Master

The Slide Master is folded double and placed under the user's pressure points

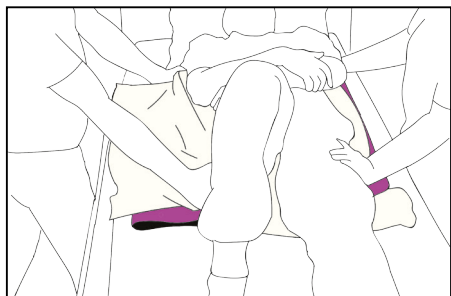
The user is rolled back into a supine position and 1 creates space using weight transfer, so that 2 can fold out the Slide Master under the user

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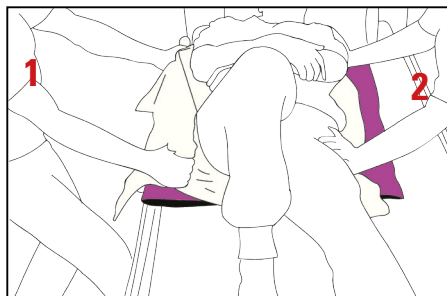
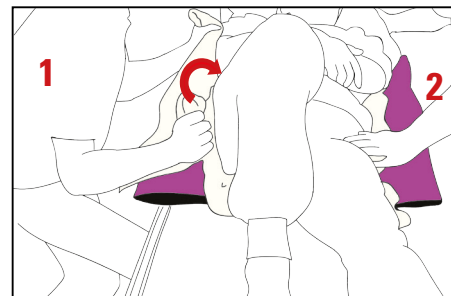
UK Slide Master is positioned as shown in 1. User lies in a supine position

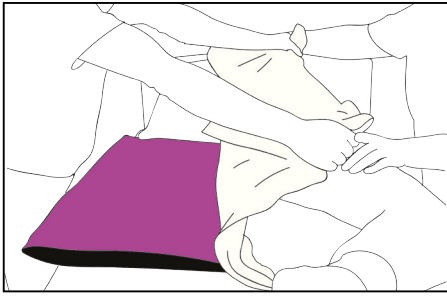
2 holds the user's hips and shoulders, while 1 uses weight transfer to pull the user towards him/herself

When the user has been slid out to the edge of the bed, 2 holds the user's hips and shoulders while 1 uses weight transfer and the drawsheet to roll the user on the side

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
UK Before the Master Turner is placed, a mattress cover is mounted on the mattress. Shown here as an Easy 3 Large mattress cover

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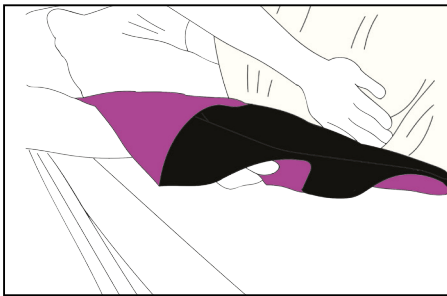
UK Slide Master is removed by grasping the lower part

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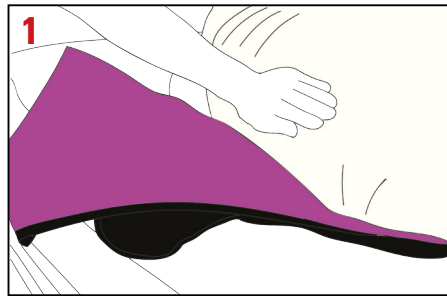
The Slide Master is then pulled away from the user, while 1 gently holds the user with the other hand

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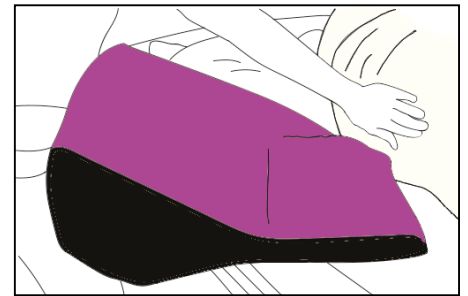
Slide Master slides away under the user

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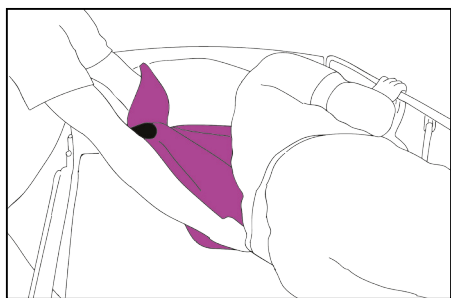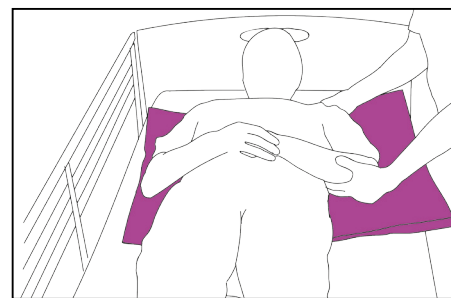
UK Roll the user into a lateral position. Place the Slide Master so that the rolling direction is towards the headboard and push the Slide Master in under the shoulders and hips

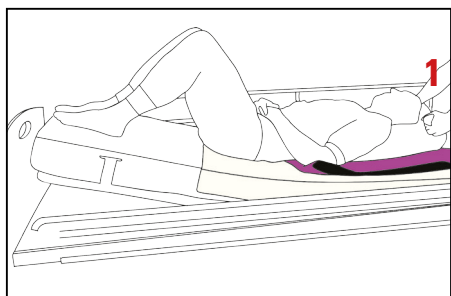
Roll the user in the opposite direction and spread the Slide Master out under the user

The user can now push with his/her legs, lift his/her head and slide higher up in the bed with Slide Master

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

UK If Slide Master is positioned under the hips, shoulders and pillow, the end of the bed can be elevated and the user can slide higher up in bed with 1 positioning him/herself beside the headboard and gently pulling the user's pillow

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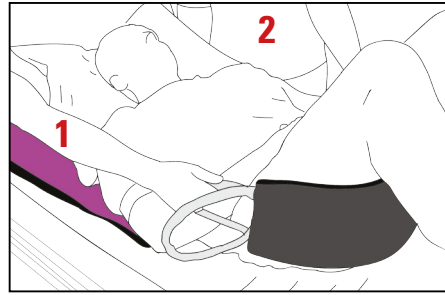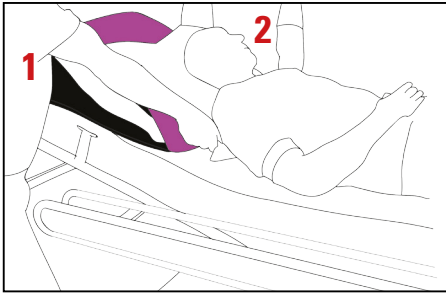UK 1 and 2 position themselves on each side of the bed's headboard and elevate it slightly. Place your palm into the Slide Master and push it under the user's shoulders

Lower the headboard. 1 and 2 now position themselves on either side of the bed, next to the user's hips. Place a Mini Master under the user's pelvis and use weight transfer to move the user higher up in the bed

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UK

- Reduces friction and therefore minimizes shear
- Easy to position and to remove from under the user
- Enables mobilisation of the user
- Covers for all sizes can be used as protection of Slide Master
- Reduces workload for staff
- Wide range of applications
- Extremely durable

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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