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I tvivl?: +45 8693 8585

## Washing- and tumble instructions

Washing temperature: **40 - 85°C**  
Tumble dry: **max 60°C**  
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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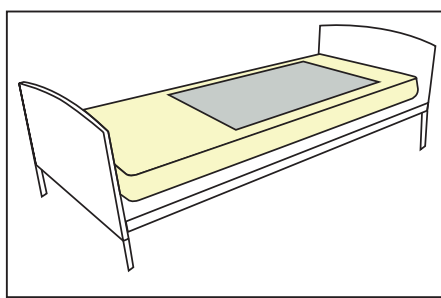
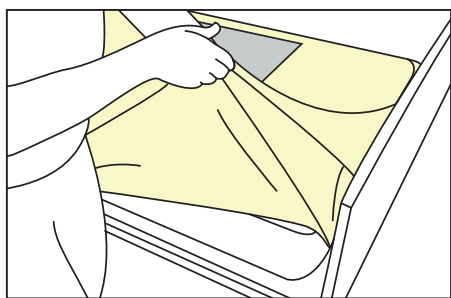
## 1. Montering af Quick ON cover / Fitting the Quick ON cover / Montage des Quick ON Bezug

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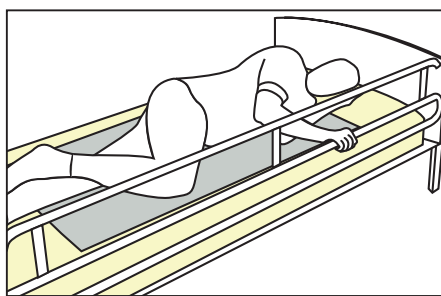
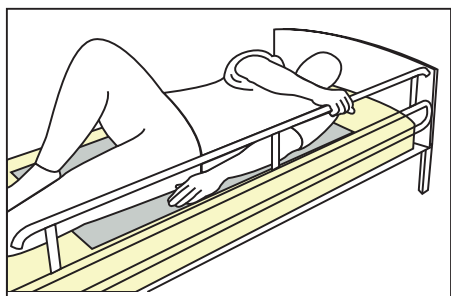
**UK** Place the Quick ON by the top of the mattress and fold the stretch sheet down around its corners

The Quick ON has now been tightly fitted around the mattress

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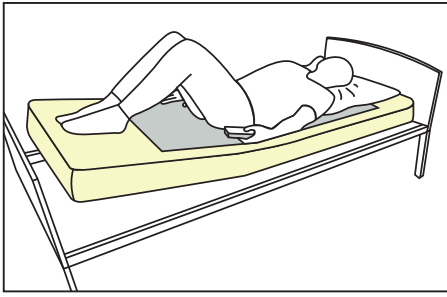
**UK** The user bends their right leg and grips the bed guard on the opposite side with their right hand

The user can now move their backside out and easily slide onto their side by pulling with their right arm and lowering their right leg against the bedding

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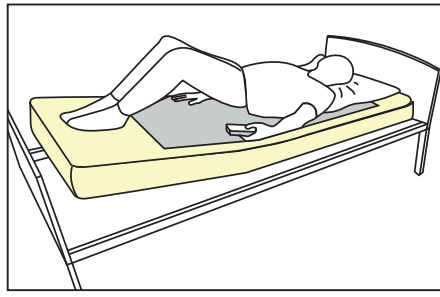
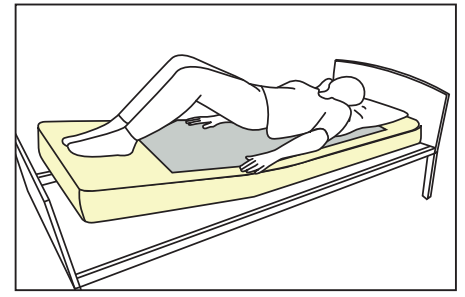
UK The user bends both legs and elevates the foot of the bed if necessary

The user then lifts their pelvis by using their legs

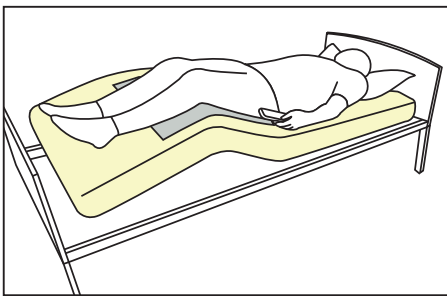
Assisted by the smooth silk layer, the user can now slide further up in the bed

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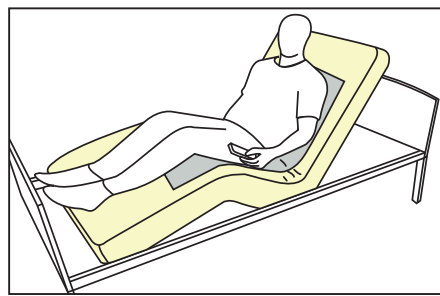
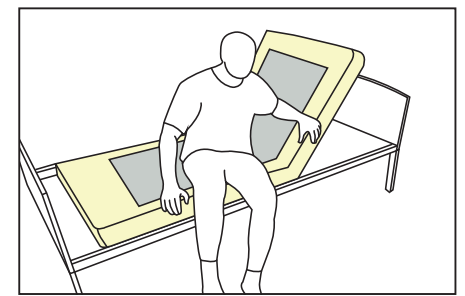
UK The user activates the bed's knee bending function

The user then elevates the bed's headboard

Due to the smooth silk layer, the user can now easily rotate on the side and slide their legs over the edge of the bed

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- UK • Reduces friction under the torso during bed transfers
- Skin-friendly and breathable
- Sits tightly without bunching up
- Can be fitted on all types of foam mattresses

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- UK • Reduces friction under the torso during bed transfers
- Skin-friendly and breathable
- Sits tightly without bunching up
- Can be fitted on all types of box spring mattresses

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**MC 001-2517** [ HMI: 53666 ]  
Master Turner Two In One Soft - **Small**

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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