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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately
- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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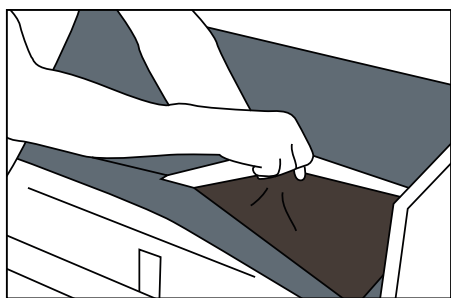
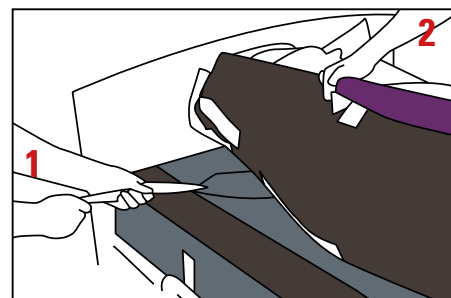
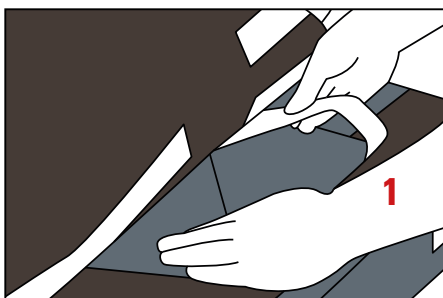
UK Place the mattress cover by the top of the mattress and fold the cover around its top corners. Then fold the cover around the corners of the bottom of the mattress

Fold the cover's non-slip wings under the middle of the mattress on both sides. If the cover has been fitted correctly, there should be 10 cm of non-slip on both sides and the non-slip wings can be locked against the bedframe

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UK Person 1 or a lift helps ensure that the Master Turner fits snugly around the user. Helper 2 loosens the sliding flap's two velcro straps and folds the bottom strap over the top one

Then fold the top strap over the bottom strap and slide the top strap under the user's neck

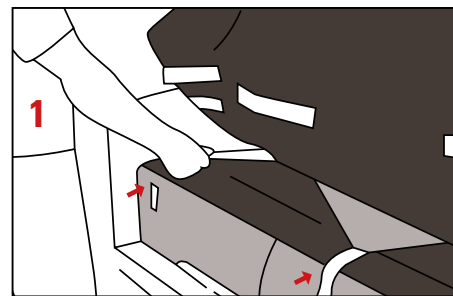
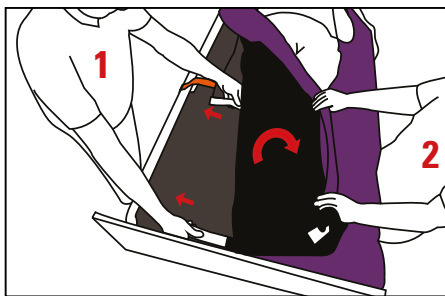
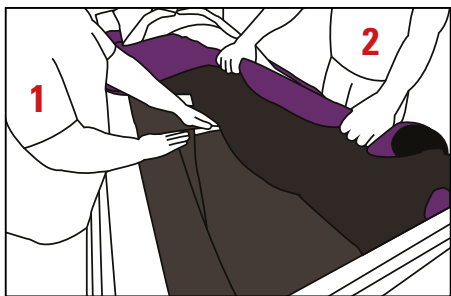
Person 2 or a lift ensures that the Master Turner fits snugly around the user and person 1 pulls the sliding flap out under the user

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UK Person 2 holds the turn sheet and helps position the user on their side. Double up the sliding flap and fold it under the user with straps at the front

Person 2 holds the turn sheet up on the opposite side. Person 1 takes hold of the straps and pulls the sliding flap into position under the user

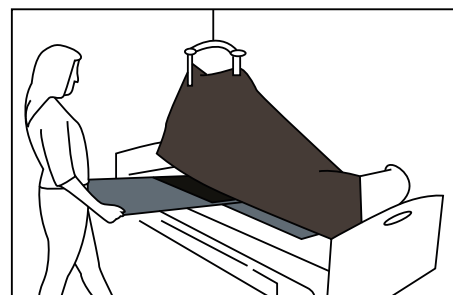
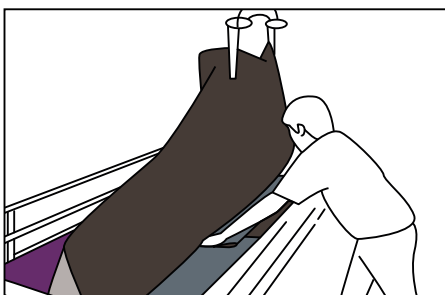
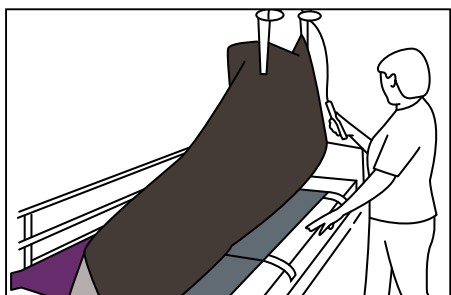
Fasten the straps onto the mattress cover

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UK Attach the lift to the turn sheet on the side where the sliding flap opening is located. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Insert the sliding flap under the user at the points of least friction (ankle, hollow of the knee, lower back or neck). Slide the straps in so they are pointing forward

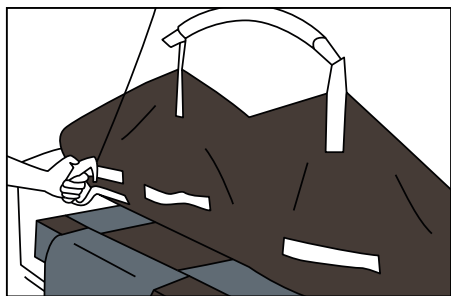
Mount the lift on the opposite side, and raise the side up. Pull out the sliding flap

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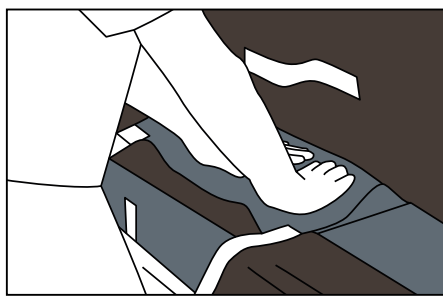
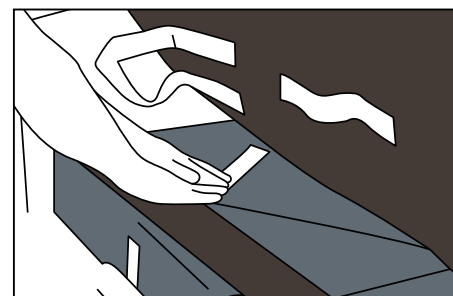
Se illustration

UK Attach the lift to the Master Turner on the opposite side of the sliding flaps opening. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Double up the sliding flap and slide it under the user using both hands. Smooth out the sliding flap so it is spread evenly across the cover's non-slip surface. Finish by maneuvering the straps under the user

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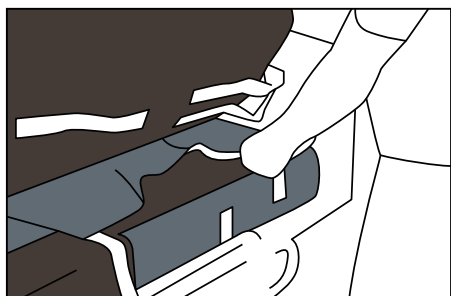
UK Then, attach the lift on the opposite side. Raise the lift until the user's shoulder and bottom are not touching the surface beneath. The sliding flap straps should now be visible and can be fastened

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UK

- Can be fitted while the user is in the bed
- The slide flaps can fix the turnsheet as needed - back and calf
- Works with the mattress
- Can be used with the slide flaps out to both right and left side
- 10 cm non-slip coating along the edge on both sides
- Unique sliding properties
- Prevents pressure ulcers

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UK

- Can be fitted while the user is in the bed
- The slide flaps can fix the turnsheet as needed - back, pelvis and calf
- Works with the mattress
- Can be used with the slide flaps out to both right and left side
- 10 cm non-slip coating along the edge on both sides
- Unique sliding properties
- Prevents pressure ulcers

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UK

- Can be fitted while the user is in the bed
- Works with the mattress
- 10 cm non-slip coating along the edge on both sides
- Unique sliding properties
- Prevents pressure ulcers

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MC 001-1732 [HMI: 49368]
Master Turner Soft - **Large**

Størrelse / Dimensions / Größe:
L 215 x B/W 250

MC 001-2540 [HMI: 49370]
Master Turner Two In One Soft - **Large**

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
Størrelse / Dimensions / Größe:
L 47 x B/W 4 cm

MC 001-2500 [HMI: 49873]
Master Turner DUO Two In One Soft Bund / Bottom
/ Unterteil - **Large**

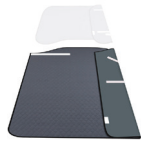Størrelse / Dimensions / Größe:
L 160 x B/W 145 cm

MC 001-2501 [HMI: 49872]
Master Turner DUO Two In One Soft Top / Top /
Oberteil - **Large**

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MC 001-1951 [HMI: 81134]
Mattress cover extender - **Large**

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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