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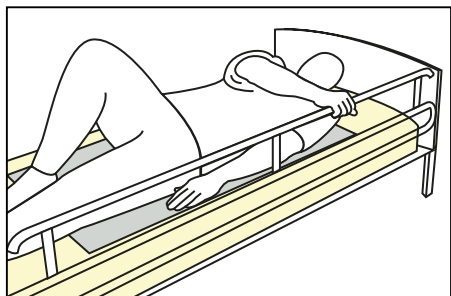
UK Place Silk Master so that the silk part is positioned in the middle of the mattress and both sides are hanging over the edges of the bed

Then, fold both sides under the mattress, making sure there are no folds on the silk part

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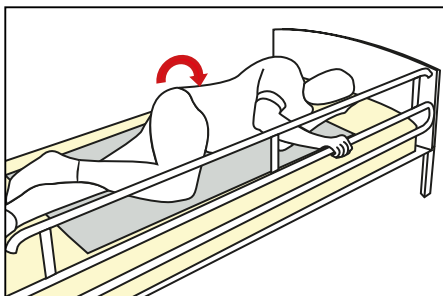
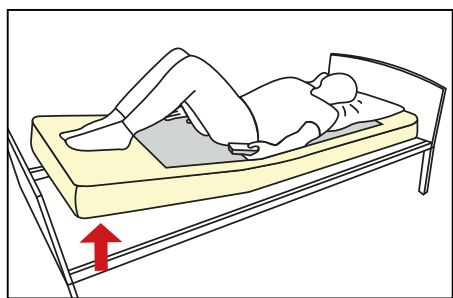
UK The user bends their right leg and grabs onto the safety rail on the other side with their right hand

The user should now be able to push out their bottom and easily slide onto their side by pulling with their right arm while resting their right leg on the surface

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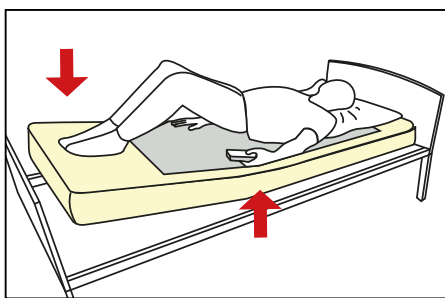
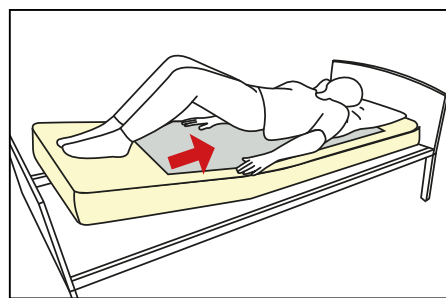
UK
The user bends both legs and elevates the foot end of the bed if necessary

The user then lifts their pelvis by pushing down with their legs

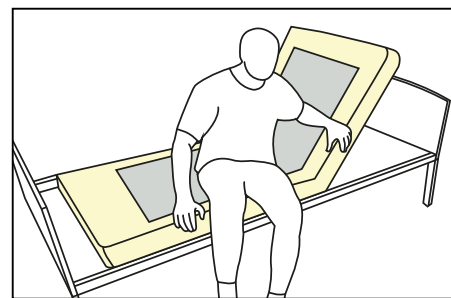
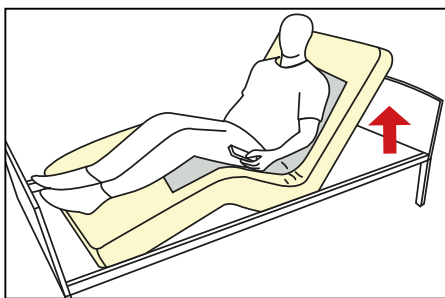
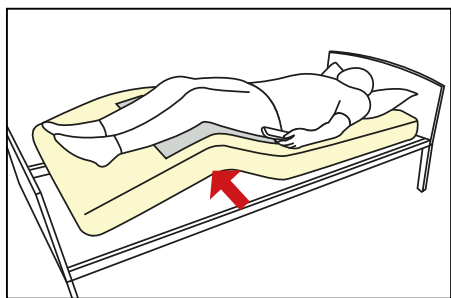
Thanks to the smooth silk part, the user can now slide further up the bed

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UK

The user activates the bed's retrograde back regulation

The user then elevates the head of the bed

The user can now easily rotate on the surface of the bed and easily slide their legs over the edge of the bed thanks to the smooth silk layer

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UK

- For incontinent users
- Reduces friction during bed transfers
- Skin-friendly and breathable
- Sits tightly under the mattress so it does not bunch up
- Can be fitted on all types of mattresses
- Extremely durable

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UK

- Reduces friction during bed transfers
- Skin-friendly and breathable
- Sits tightly under the mattress so it does not bunch up
- Can be fitted on all types of mattresses
- Extremely durable

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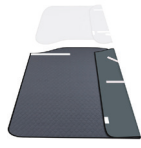
Størrelse / Dimensions / Größe:
L 47 x B/W 4 cm

MC 001-2500 [HMI: 49873]

Master Turner DUO Two In One Soft Bund / Bottom
/ Unterteil - **Large**

Størrelse / Dimensions / Größe:
L 160 x B/W 145 cm

MC 001-2501 [HMI: 49872]

Master Turner DUO Two In One Soft Top / Top /
Oberteil - **Large**

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Størrelse / Dimensions / Größe:
L 43 x B/W 25 x H 14 cm

MC 001-8020 [HMI: 45939]

Master Comfort Vaskekile - Medium / Washing
Wedge / Waschkeil

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

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IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the top piece

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