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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately
- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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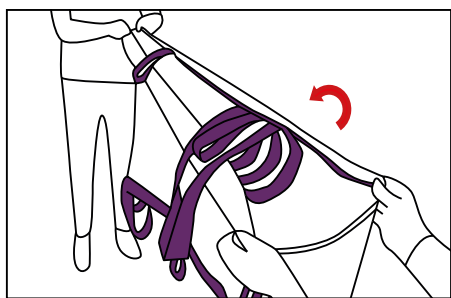
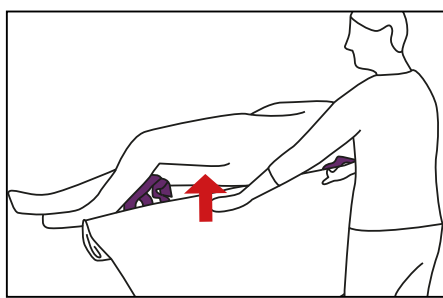
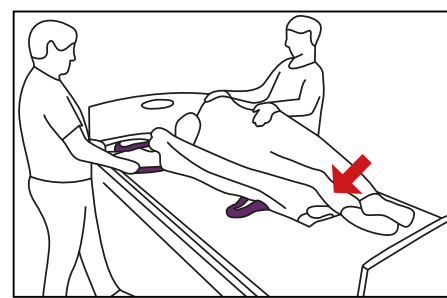
UK Fold the bathing sling down the middle. The label "This side up and against the headrest" should point upward. Take hold of the half that is hanging down and fold it up. Repeat this until there is as little fabric as possible on one side

Turn the user onto his or her side. Place the bathing sling up along the user's back from top to bottom. The label "This side up and toward the back/head area" should be at the back of the user's head. The user's spine should be aligned with the centre of the bathing sling. Slide the straps a little under the user

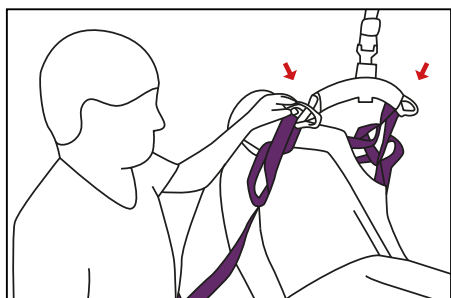
The user is laid back down on the bathing sling and turned to the other side. The assistant pulls on the bathing sling straps to unfold it. The user is laid on his or her back again

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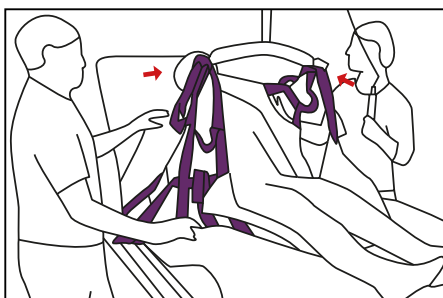
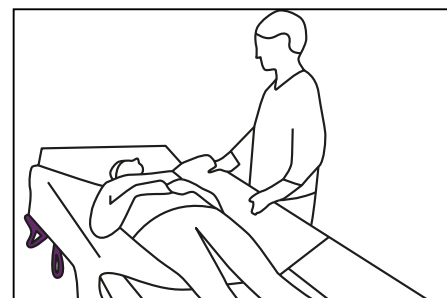
UK Raise the adjustable bed under the user's knees and raise the head of the bed. The straps on both sides by the head of the bathing sling are hooked onto the lifting beam. Use the inner straps so that the user sits up a bit

The straps by the knees are also hooked onto the lifting beam. Use the outer straps. Carefully lift the user up

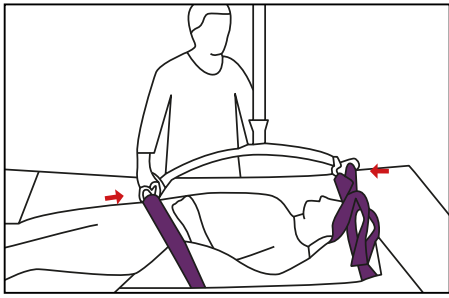
The user is lowered onto the bathing unit and is now ready to be washed

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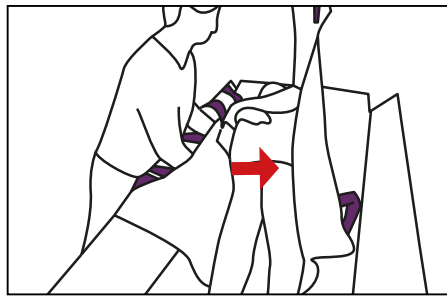
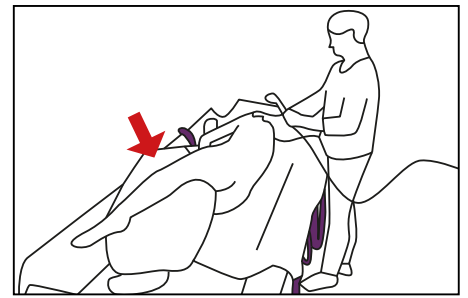
UK Hook the straps at the head and the hip area of the bathing sling onto the lifting beam on one side. Run the lift up a little bit

Slide the user into the middle of the unit to make room for the turn

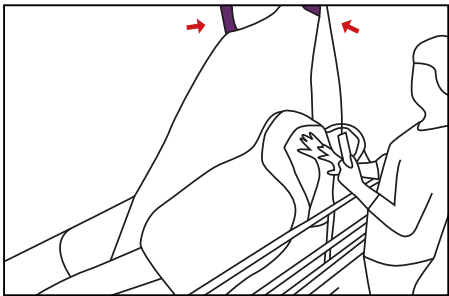
Remove the straps from the lifting beam. The Master Comfort washing wedge can be placed between the user's knees to help stabilise the hip

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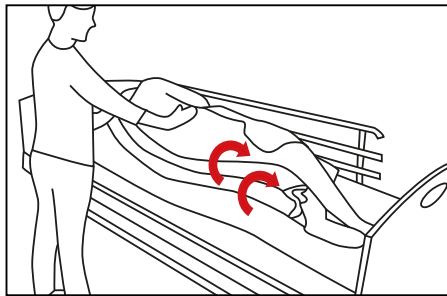
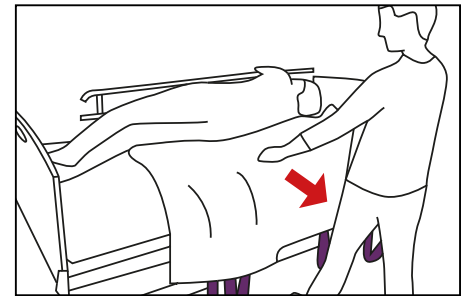
UK Lower the knee-bend and the head of the bed. The straps at the head and the hip area of the bathing sling are hooked onto the lifting beam on one side. Run the lift up so that the user is lying on his or her side

Remove the straps from the lifting beam. Roll the bathing sling and the Inko Master incontinence sheet up as far as possible under the user. Turn the user on his or her back

Turn the user over to the other side and pull the bathing sling and Inko Master out from under the user

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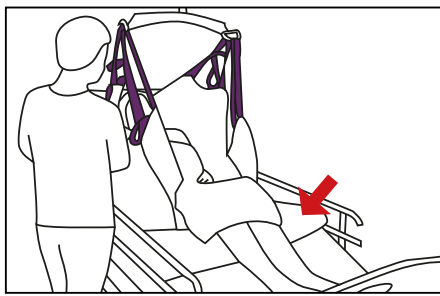
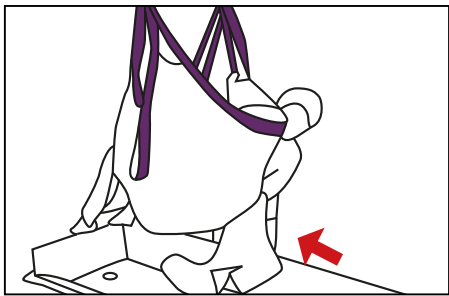
UK Hook the straps on each side of the bathing sling onto the lifting beam. Lift the user from the surface. Dry the back side of the bathing sling so that as little water as possible ends up in the bed

Place an Inko Master incontinence sheet on the bed to ensure that the surface does not get wet. Raise the knee-bend and head of the bed. Lower the user into bed and remove the straps from the lifting beam

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- UK • Sling and turn sheet in one unit
- Can be used with all types of floor or overhead lifts
- Made of high-strength polyester mesh

- Allows water to run through it
- Reduces the distance and bent back for helper

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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