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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the top piece

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
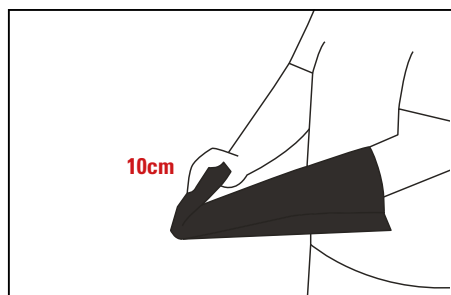
UK Insert hand in Sling Sock. Then fold the front part approx. 10 cm. See illustration

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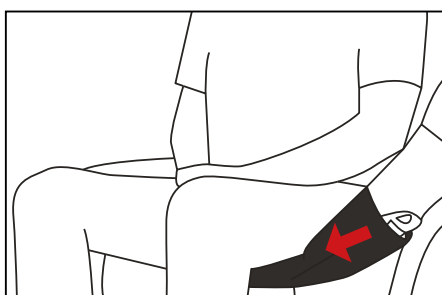
Insert one palm under the point by the user where the Sling Sock is to be inserted. Then slide the Sling Sock in under the hand

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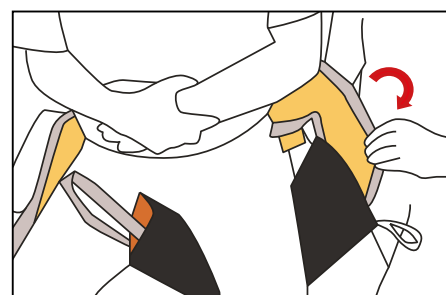
Insert the lifting sling's leg strap into the Sling Sock. This is done from the outside inward

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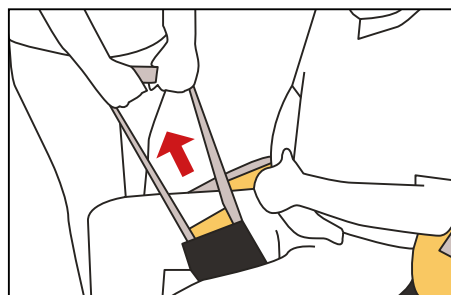
UK Then insert the hand into the Sling Sock from the opposite side and grasp the lifting sling's strap to pull it out. The sling is mounted without creating friction and Shear Force is avoided. Perform the same on the opposite side

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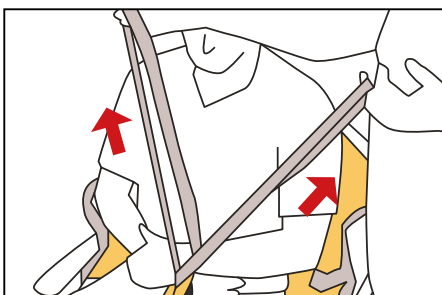
Finally, the helper inserts his/her palm into the Sling Sock and pushes the palm down towards the seat of the chair. The sling is simultaneously pulled to obtain the correct position, and it is now ready for lifting

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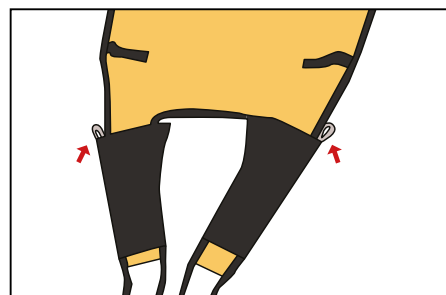
Keep in mind that the widest side of the Sling Sock must face the centre of the lifting sling so that the straps are located on the outside of the sling

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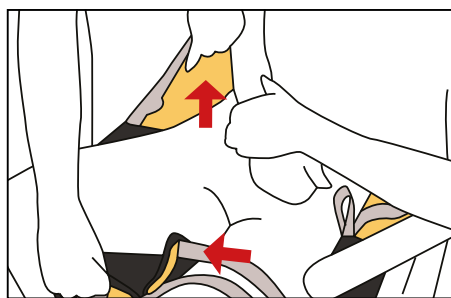
UK Finally, the helper inserts his/her palm into the Sling Sock and pushes the palm down towards the seat of the chair. The sling is simultaneously pulled to obtain the correct position, and it is now ready for lifting

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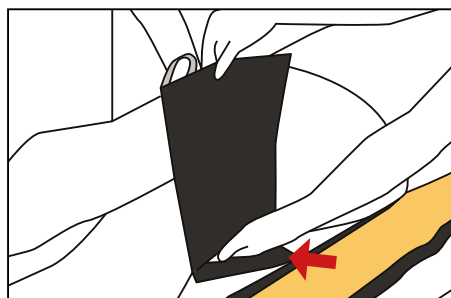
UK Follow the same procedure as in item 1.

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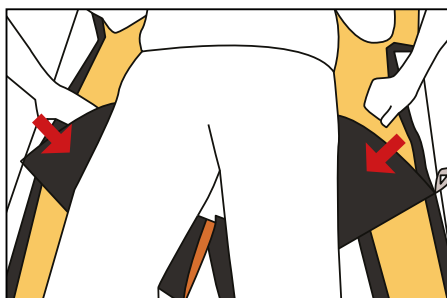
Slide the Sling Sock under the user's thighs on both sides

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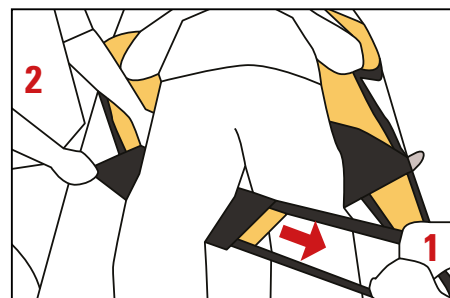
Insert one of the lifting sling's leg straps into the Sling Sock. Helper 1 pulls the leg strap while helper 2 holds the Sling Sock on the opposite side. This is done on both sides of the user. The sling is mounted without creating friction

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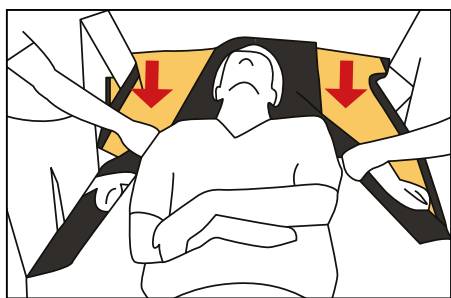UK Activate the break at the knee and remove the lower sliding flap on the mattress cover. Fold the lifting sling and insert it between the top and the mattress cover's smooth surfaces. Slip and slide the lifting sling from the top towards the user's lower back

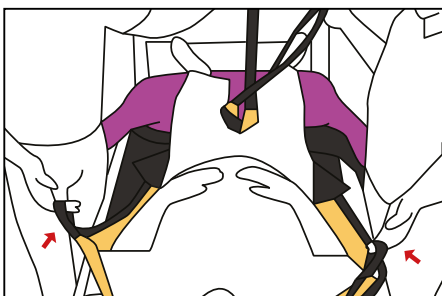
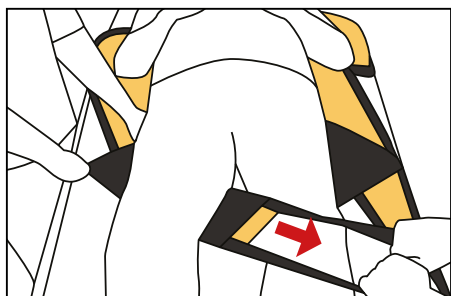
Disassemble the Master Turner Duo Top from the bottom by opening the Velcro fasteners on both sides

Fold and roll the bottom of the Master Turner Duo down towards the user's knees. The lifting sling's leg straps are brought forward

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UK Insert the Sling Sock under the user's leg as shown in item 3.1-3

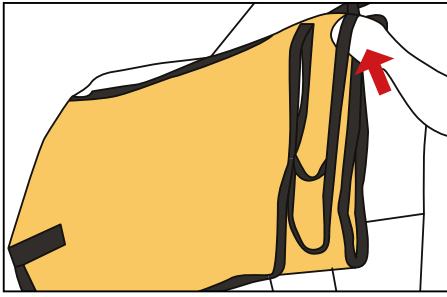
The lifting sling is now in place and the user can now be lifted from the base. The top piece may be included in the lift and finally removed in the chair. If not, simply fold it down and roll it up under the user's pillow

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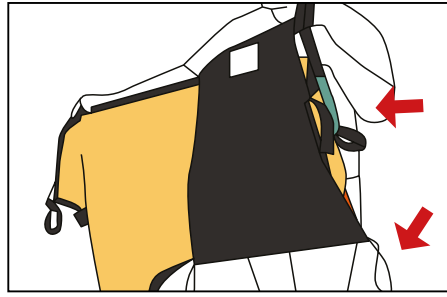
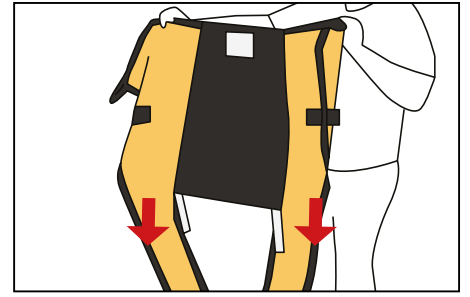
UK Lift one foot strap up on the lifting sling and hold it together with the sling

Insert the Sling Vest across the lifting sling. Make sure the wide part of the handle is facing downward

Then slip the foot strap, and the Sling Vest is now in place

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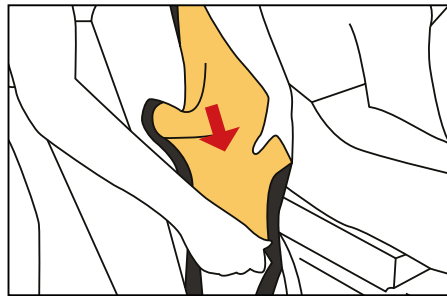
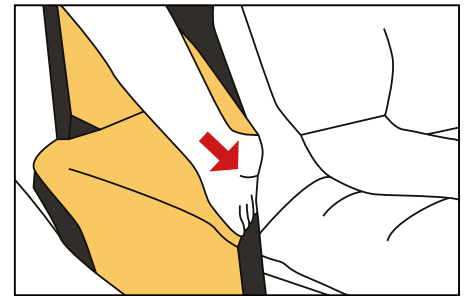
UK Stand behind the wheelchair. Hold the Sling Vest and lifting sling together, tighten and slide the sling down between the user's back and chair by first starting diagonally down towards one side

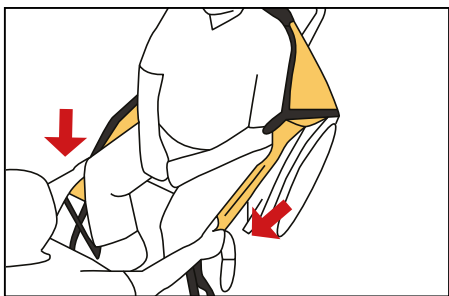
Keep the sling stretched and slide it down towards the other side, and alternate until the sling is positioned against the user's thighs

If possible, remove armrests for better access. Then insert your hand into the Sling Vest and slide it down and around the user's hips on each side

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
UK Finally, pull the lifting sling's leg straps in front of the
user. It then slides completely into place

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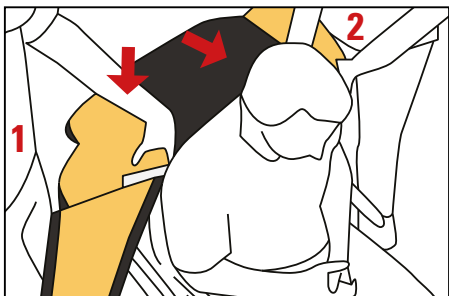UK Helpers 1 and 2 stand on either side of the
wheelchair. The lifting sling and Sling Vest are held
together tightly

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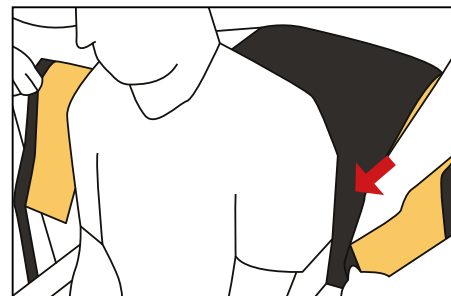
Hold the sling on the sides and insert hands into the
Sling Vest from the sides

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
The Sling Vest, with the lifting sling, is now pushed
down between the user and the wheelchair's back
from both sides

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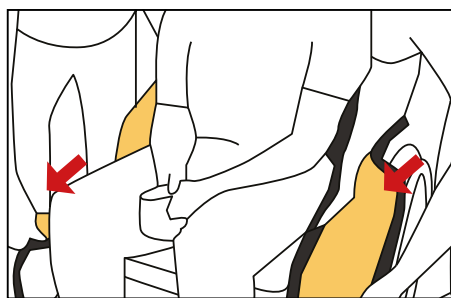
UK If possible, remove the armrests from the wheelchair and slide the Sling Vest with the lifting sling into place around the user's hips

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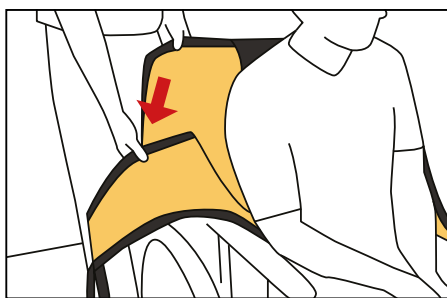
Insert the leg straps down and back at the sides

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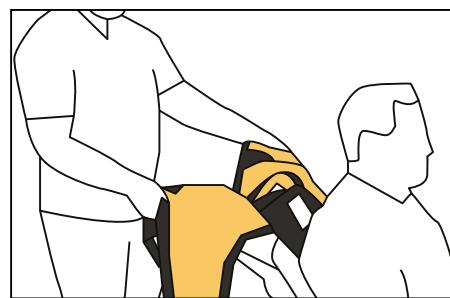Pull up the lifting sling. This slides into the Sling Vest and up against the back of the wheelchair. By using the Sling Vest, the user retains the optimal sitting position, as the pelvis is not tilted

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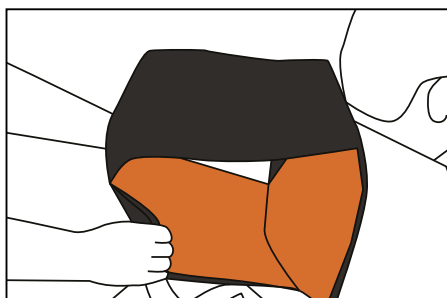
UK Keep the sling folded and pull it out to one side. Hold onto the Sling Vest's edge on the opposite side

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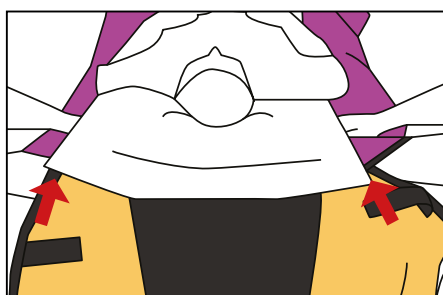
Grasp the sides at the bottom of the Sling Vest, facing the wheelchair's back and roll it up. The Sling Vest is removed without folding the user's clothing

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UK Place the Sling Vest on the lifting sling as shown in item 1. 1-3 Helpers 1 and 2 insert their palms into the lifting sling's hip section and fold it so that the leg straps fall down at the sides

Fold the lifting sling and insert it between the top and the mattress cover's smooth surfaces. Slip and slide the lifting sling from the top towards the user's lower back

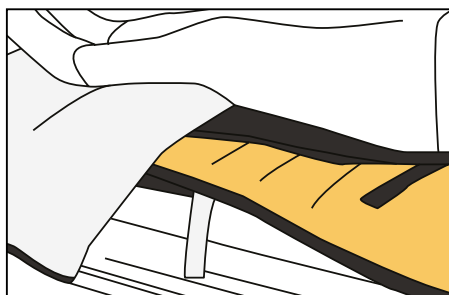
Disassemble the Master Turner Duo Top from the bottom by opening the Velcro fasteners on both sides. Roll the bottom downward and under the user's knees

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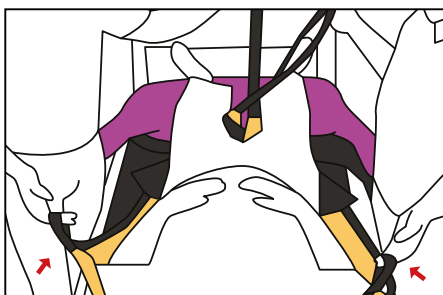
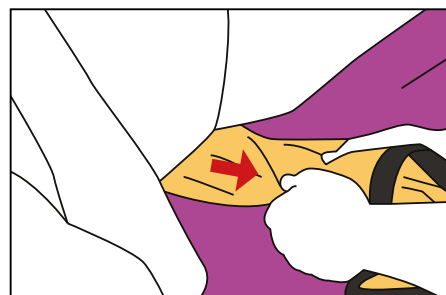
UK The lifting sling's leg straps are brought forward if necessary, use the Sling Sock to position the leg straps

The lifting sling is now in place and the user can be lifted from the ground. The top piece may be used in the lift. If not, fold it down and roll it up under the user's pillow

The lifting sling is removed from the Sling Vest by helper 1 pulling this out while helper 2 holds the edge of the Sling Vest on the opposite side. The Sling Vest is then pulled out from under the user

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UK

- Easy to attach the lifting sling's leg straps
- Polish the sling's leg straps
- Reduces friction and prevents Shear
- Optimises the workflow for helpers
- Increases user comfort
- Ensures seat position by disassembling the sling in a wheelchair sling

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UK

- Prevents the formation of pressure sores
- Optimises the workflow for the helpers •
- Increases user comfort when lifting and removing slings

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
Størrelse / Dimensions / Größe:
L 47 x B/W 4 cm

MC 001-2500 [HMI: 49873]

Master Turner DUO Two In One Soft Bund / Bottom
/ Unterteil - **Large**

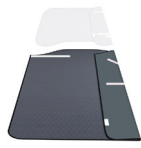Størrelse / Dimensions / Größe:
L 160 x B/W 145 cm

MC 001-2501 [HMI: 49872]

Master Turner DUO Two In One Soft Top / Top /
Oberteil - **Large**

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Størrelse / Dimensions / Größe:
L 43 x B/W 25 x H 14 cm

MC 001-8020 [HMI: 45939]

Master Comfort Vaskekile - Medium / Washing
Wedge / Waschkeil

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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