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○ **MC 001-1971** [HMI: 41940]
Master Turner Silk - **Medium**

Størrelse / Dimensions / Größe:
L 140 x B/W 145 cm

○ **MC 001-2525** [HMI: 125759]
Master Turner Two In One Soft
Combi - **Medium**

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○ **MC 001-2040** [HMI: 42390]
Master Turner Two In One Frotté
/ Terry Cloth / Frottee - **Large**

Størrelse / Dimensions / Größe:
L 215 x B/W 145 cm

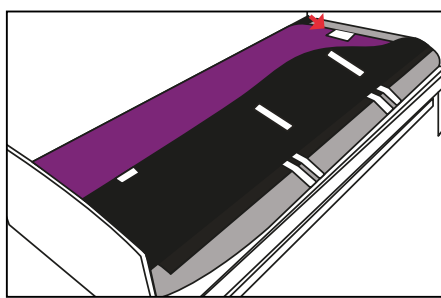
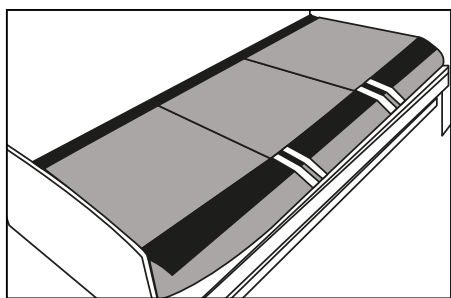
○ **MC 001-2540** [HMI: 49370]
Master Turner Two In One Soft
- **Large**

Størrelse / Dimensions / Größe:
L 215 x B/W 145 cm

1. Montering af Master Turner / Placing the Master Turner / Montage des Master Turner

DK Inden Master Turner placeres, monteres et valgfrit master **care** madrascover på madrassen. Her illustreret ved et Easy 3 Large cover

Placér Master Turner med den glatte del ned mod madrascoveret. Etiketten "Denne side opad og ind mod hovedenden" vender mod sengens hovedgavl

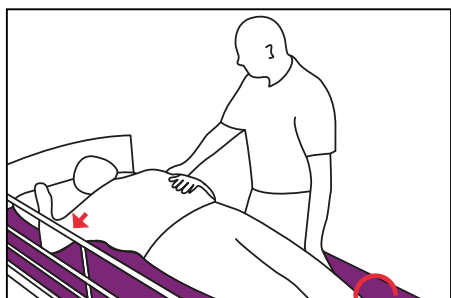
UK Before the Master Turner is placed, a mattress cover is mounted on the mattress. Shown here as an Easy 3 Large mattress cover

Place the Master Turner with the smooth side down on the mattress cover. The label "This side up and against the headrest" faces the headboard

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
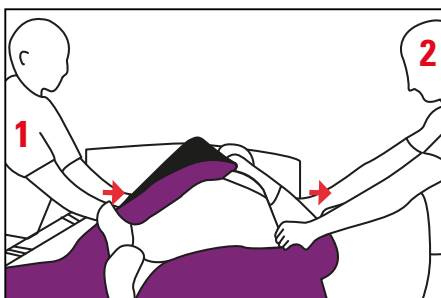
UK Cross the user's legs. Lead the user's hand to touch their opposite shoulder. Slide the pillow and shoulder toward the side that you need to turn the user

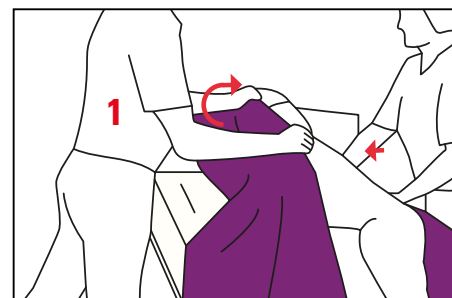Person 1 pulls and person 2 pushes the user toward the desired bedside

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
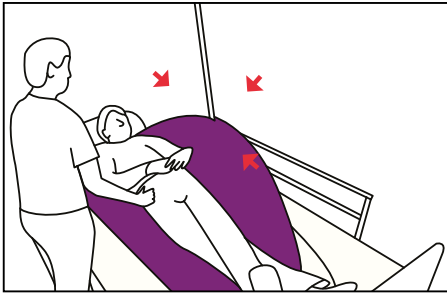
Person 1 pulls the turn sheet and helps position the user on their side

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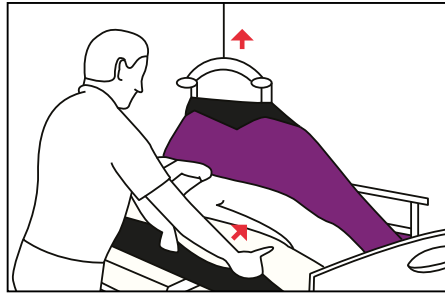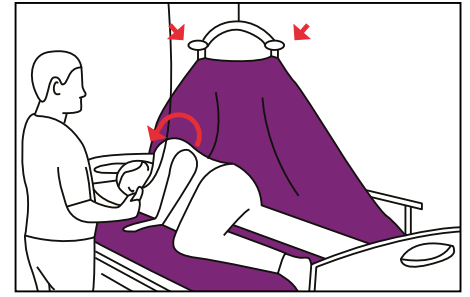
UK Cross the user's legs and pull their pillow and shoulder toward the side that you need to turn the user. Attach the turn sheet's straps to one side of the lift

Lift the user slightly above the surface beneath. The helper then positions the user toward the bed guard. Use the Master Comfort Stomach Wedge for additional relief and support during turn

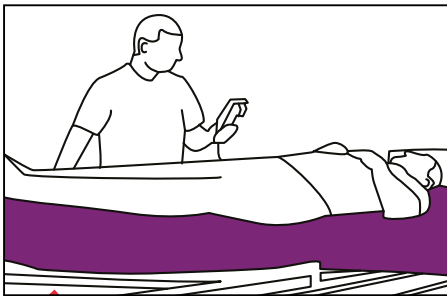
Raise the lift. Turn the user over onto a lateral position

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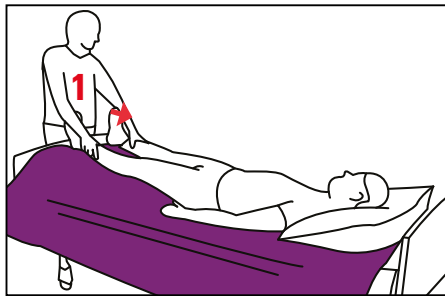
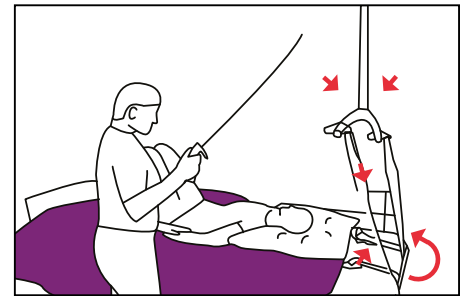
UK Elevate the foot of the bed

Person 1 stands at the foot of the bed and makes a weight transfer of the user's ankle/calf. The user then slides toward the head of the bed

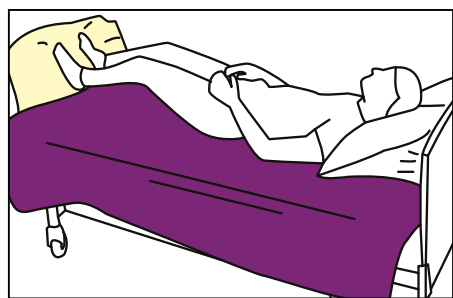
Alternatively, dismount the bed's headboard. Attach the Master Turner Pull Straps to the turn sheet's straps at the head of the bed, pulled under the bedframe and hooked onto the lift. Raise the lift to pull the user into an upright position in bed

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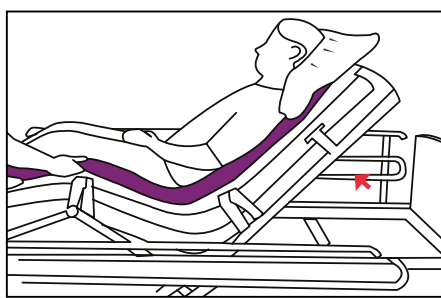
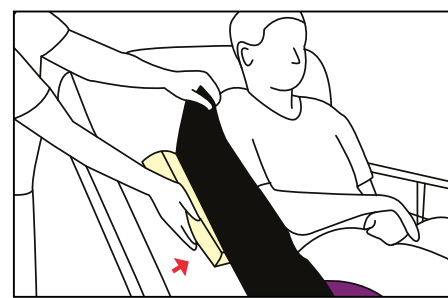
UK Activate the adjustable bed under the user's knees. Place the Master Comfort Washing Wedge at the foot of the bed

Elevate the head of the bed

Place the Master Comfort Support Wedges under the turn sheet on either side of the user

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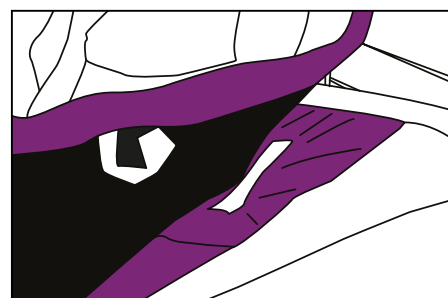
UK When this method of transfer is used, person 1 must ensure that the sliding flap's opening is facing the helper. Elevate the adjustable bed under the user's knees. Elevate the head of the bed. Attach the turn sheet's straps to the main section of the lift and raise until it fits snugly behind the user's back

Position the user's legs over the edge of the bed

The user can then slide into the desired seating position. In order to avoid sliding forward, the mattress covers sliding flap can be pulled back to make the anti-slip visible

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Fordele / Advantages / Vorteile

Master Turner Soft & Frotté / Terry Cloth / Frottee - Medium, Large & X-Large

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UK

- Specially designed for bariatric users and alternating pressure mattresses
- Thin padding prevents overheating
- Can be used with all types of floor or ceiling lifts
- Handles for easy handling
- Easy to transfer user further up in bed
- Reduces reaching distance and bent back for helpers

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Master Turner Two In One Soft & Frotté / Terry Cloth / Frottee - Medium, Large & X-Large

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UK

- Specially designed for incontinent users and alternating pressure mattresses
- Thin padding prevents overheating
- Can be used with all types of floor or ceiling lifts
- Handles for easy handling
- Reduces the risk of pressure ulcers/shear
- Easy to transfer user further up in bed
- Reduces reaching distance and bent back for helpers

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Størrelse / Dimensions / Größe:
L 200 x B/W 80 x H 13 cm

MC 004-3011 [HMI: 73522]
Madrascover / Mattress cover / Matratzenbezug
Exchange 3 - Large

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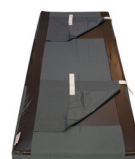
Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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