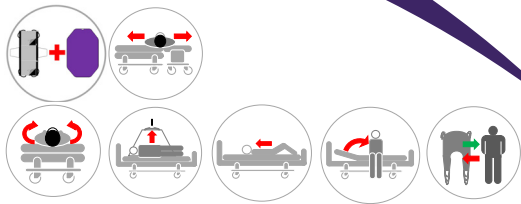
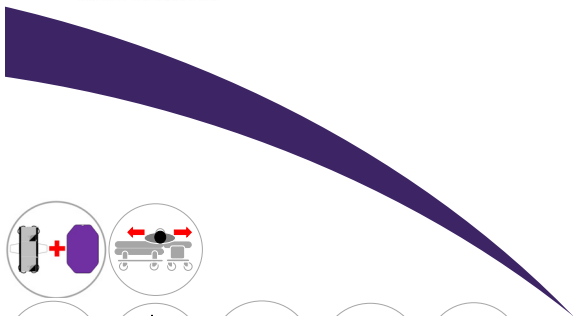
MADE IN EUROPE

This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

CE

ISO 12 31 03 21

(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1
(UK) Regulation MDR 2017/745 for medical devices class 1

The Original Nordic Human Handling Concept

Quick Guide

(DK) **Master Turner** - til madrascovers

(UK) **Master Turner** - for mattress covers

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Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torne fabric, seams and straps. Products with high friction or other deficiencies must be discarded.

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(UK) Never leave the user on the product when lying on the edge of the bed.

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(UK) Never leave the product on the floor.

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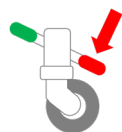
(UK) Never leave the user lying on the product whilst connected to a hoist.

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(UK) Always make sure that the user is not caught in the straps.

(DK) Sørg for, at alle relevante bremses er låste, inden forflytning foretages.

(UK) Make sure that all relevant brakes are activated before transferring the user.

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(UK) Do not use as a Sling – can be combined with a hoist for repositioning in bed.

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www.master-care.dk

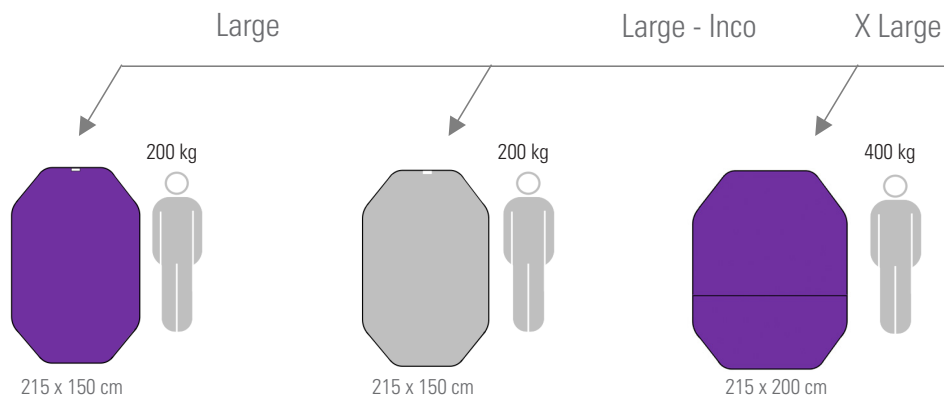
(UK) For more information, userguides, videos etc. visit

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

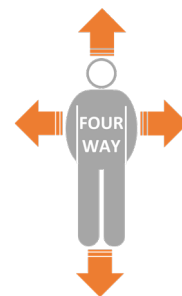
This side towards the user and the headrest.

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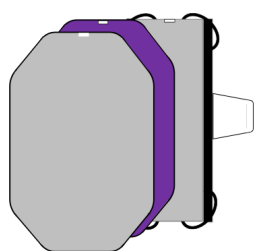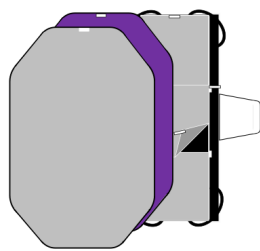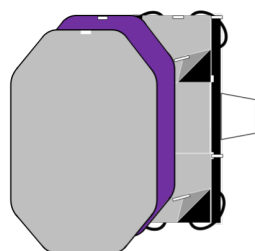
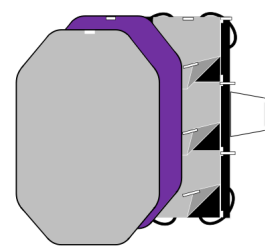
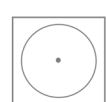
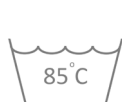
(UK) Gentle washing and dry tumbling significantly prolongs the durability.

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(UK) Never use fabric softener as this will reduce the sliding properties significantly.

master-care.dk
master care A/S

Sofienlystvej 3, DK 8340 Malling, Danmark
Tel. +45 86 93 85 85, E-mail: info@master-care.dk