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UK

BASIC Inco is rolled up

The user is turned in a lateral position and BASIC Inco is placed next to the area to be covered

The user is returned to a supine position and turned over onto the opposite side. BASIC Inco is now under the user and rolled out

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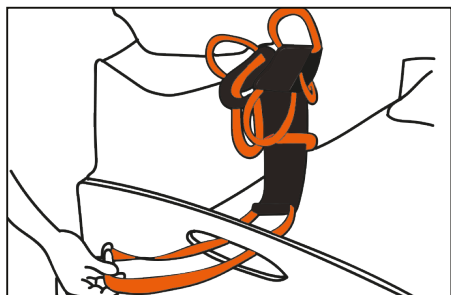
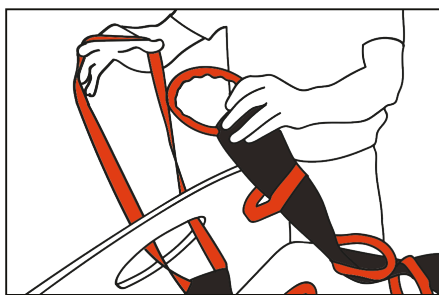
Se illustration

UK

Pass the lower part of the BASIC Grip through the headboard

While holding the lower eye, pass the BASIC Grip, with its handles, through this

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
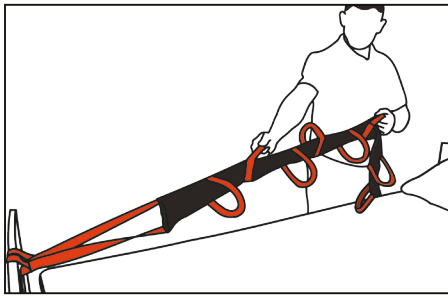
UK Pull until the BASIC Grip is fully tightened around the entire headboard. BASIC Grip is now ready for use

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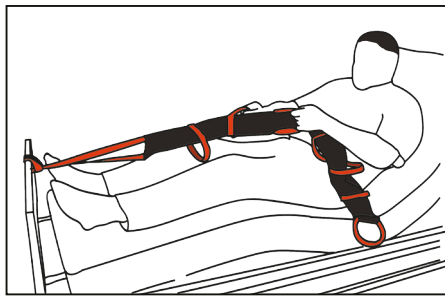
BASIC Grip is ideal when the user has to sit up in bed

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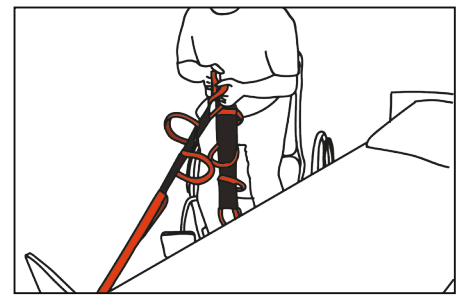
BASIC Grip is also used to transfer users between chair and bed. For this, we also recommend our Master Glide sliding board for a smooth transfer

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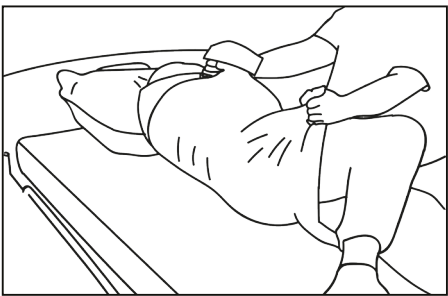
UK Cross or bend the user's legs and grasp the undersheet and use weight transfer to create space for the BASIC Slide

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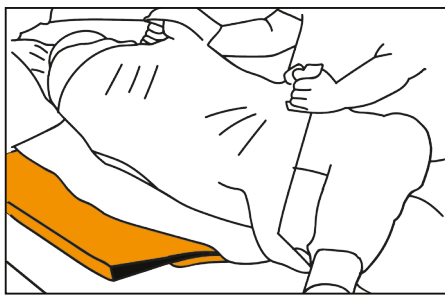
BASIC Slide foldes dobbelt og placeres under brugers trykpunkter

The BASIC Slide is folded double and placed under the user's pressure points

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
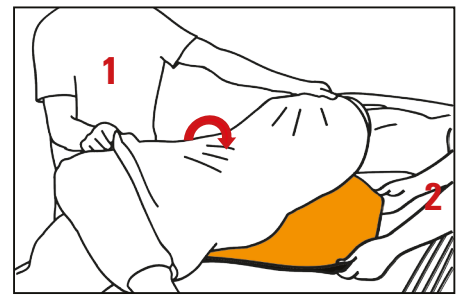
The user is rolled back into a supine position and 1 creates space using weight transfer, so that 2 can fold out the BASIC Slide under the user

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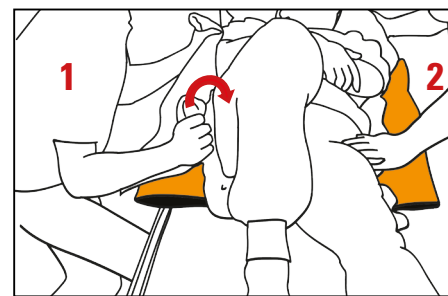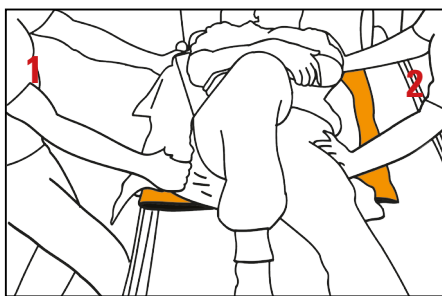UK BASIC Slide is positioned as shown in 1.1 on page 3. User lies in a supine position

2 holds the user's hips and shoulders, while 1 uses weight transfer to pull the user towards him/herself

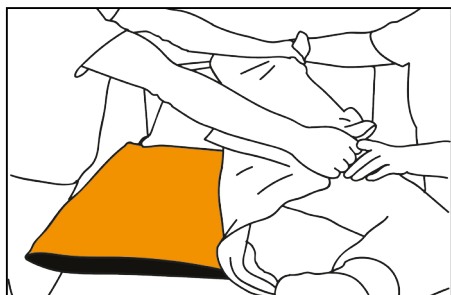
When the user has been slid out to the edge of the bed, 2 holds the user's hips and shoulders while 1 uses weight transfer and the drawsheet to roll the user on the side

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
UK The user is then turned in a lateral position and BASIC Slide is easily rolled away under the user

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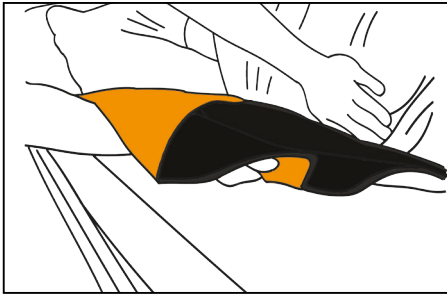
UK BASIC Slide is removed by grasping the lower part

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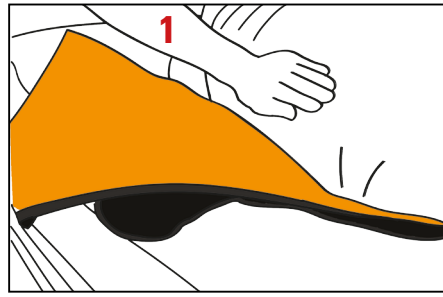
The BASIC Slide is then pulled away from the user, while 1 gently holds the user with the other hand

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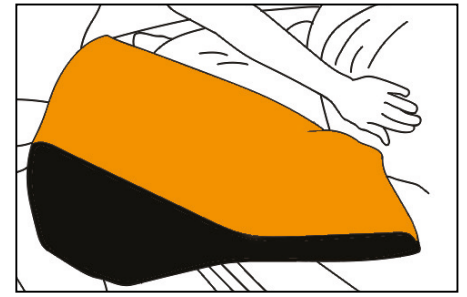
BASIC Slide slides away under the user

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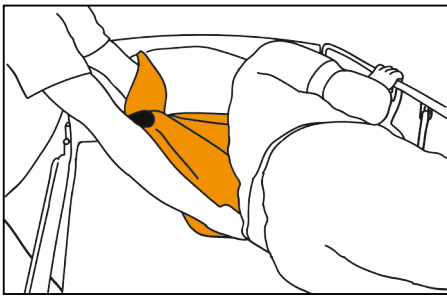
UK Roll the user into a lateral position. Place the BASIC Slide so that the rolling direction is towards the headboard and push the BASIC Slide in under the shoulders and hips

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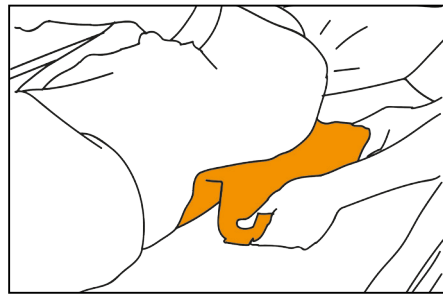
Rul bruger modsat og bred BASIC Slide ud under bruger

Roll the user in the opposite direction and spread the BASIC Slide out under the user

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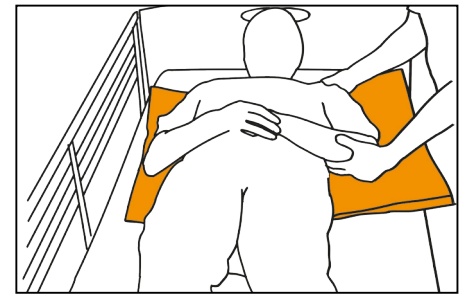
The user can now push with his/her legs, lift his/her head and, with BASIC Slide, slide higher up in the bed

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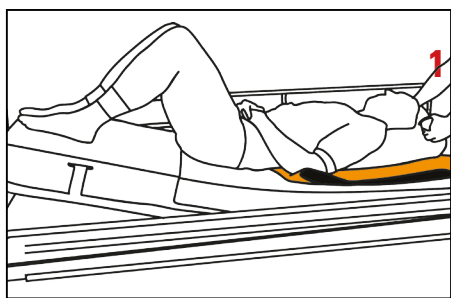
UK If BASIC Slide is positioned under the hips, shoulders and pillow, the end of the bed can be elevated and the user can slide higher up in bed with 1 positioning him/herself beside the headboard and gently pulling the user's pillow

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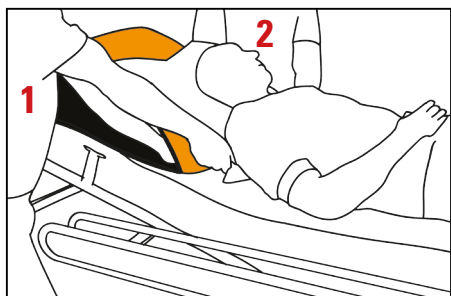
UK 1 and 2 position themselves on each side of the bed's headboard and elevate it slightly. Place your palm into the BASIC Slide and push it under the user's shoulders

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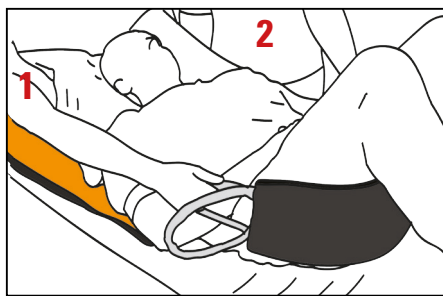Lower the headboard. 1 and 2 now position themselves on either side of the bed, next to the user's hips. Place a Mini Master under the user's pelvis and use weight transfer to move the user higher up in the bed

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- Fleksible anvendelsesmuligheder

- UK
- Can absorb up to 250 ml of liquid per m²
 - Can be used along with the master **care** turningsystem

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- UK
- Allows the user to independently rise up and sit on the edge of the bed
 - The diagonal move is gentle on the patient's back and supports normal movement patterns

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- UK
- Reduces friction between the patient and the bedding
 - Easy to position/remove
 - Makes it easier to mobilise the user, thus reducing staff workloads

- Makes the user more independent
- Great longevity

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantl

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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These products comply with the standards for Class 1 products in the Medical Device Regulation (EU) 2017/745.

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