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○ **MC 003-1125** [ HMI: 33238 ]  
Master Belt Multi Velour -  
**Medium**

Størrelse / Dimensions / Größe:  
L 80-120 cm  
Håndtag / Handles / Hangriff: 9

○ **MC 003-1126** [ HMI: 33239 ]  
Master Belt Multi Velour -  
**Large**

Størrelse / Dimensions / Größe:  
L 95-130 cm  
Håndtag / Handles / Hangriff: 11

○ **MC 003-1127** [ HMI: 42302 ]  
Master Belt Multi Velour -  
**X-Large**

Størrelse / Dimensions / Größe:  
L 130-150 cm  
Håndtag / Handles / Hangriff: 13

○ **MC 003-1220** [ HMI: 45645 ]  
Master Belt Multi Non-Slip -  
**Small**

Størrelse / Dimensions / Größe:  
L 65-100 cm  
Håndtag / Handles / Hangriff: 7

○ **MC 003-1227** [ HMI: 45648 ]  
Master Belt Multi Non-Slip -  
**X-Large**

Størrelse / Dimensions / Größe:  
L 130-150 cm  
Håndtag / Handles / Hangriff: 13

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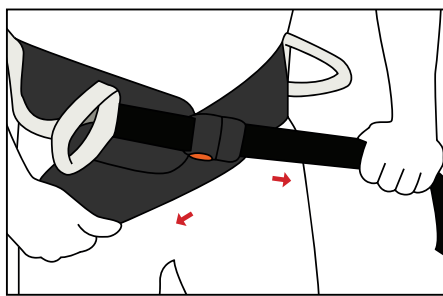
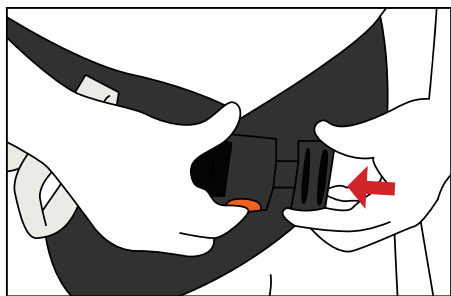
Belts only for use as a support function for users. NOT FOR PERSONAL LIFT

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 **master care**<sup>®</sup>  
HUMAN HANDLING SYSTEM

## 1. Montering af Master Belt / Fitting the Master Belt / Montage des Master Belt

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


**UK** The Master Belt is mounted around the waist of the user. Master Belt fastened using its adjustable lock buckle. See illustration

To customize the Master Belt to the user's waistline pull in the end of the belt itself in the opposite direction for tightening to

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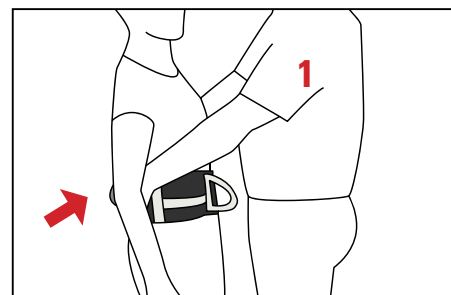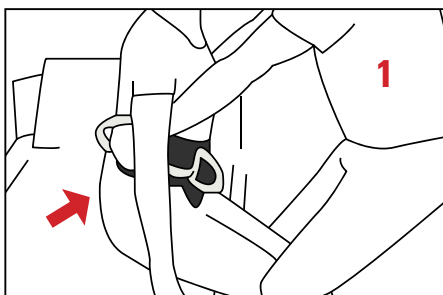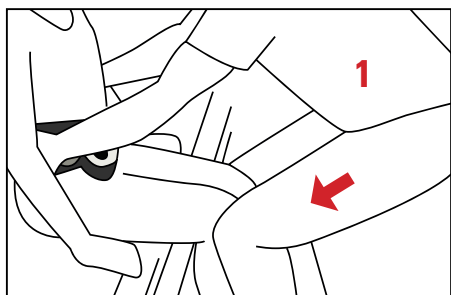**UK** Make sure that the user is capable of standing before transferring. 1 stands in front of the user and braces his/her legs against the user's legs

1 grasps the handles on both sides of the Master Belt and leans backwards using weight transfer, and stands up at the same time the user stands up


The user is now standing. 1 provides support by continuing to hold the handles on the Master Belt

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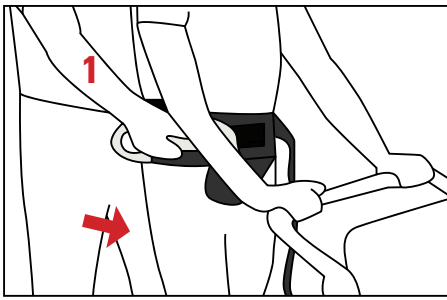
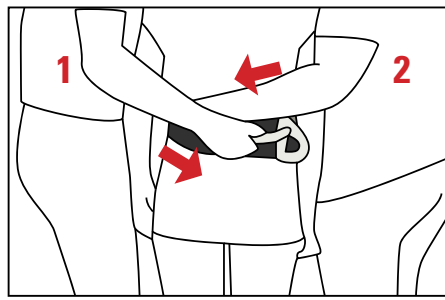
**UK** 1 stands behind the user and holds the handle on the Master Belt. See illustration

If there are two helpers, 1 and 2 grip the handles of the Master Belt, so that the arms of 1 and 2 cross each other and support the user's back

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
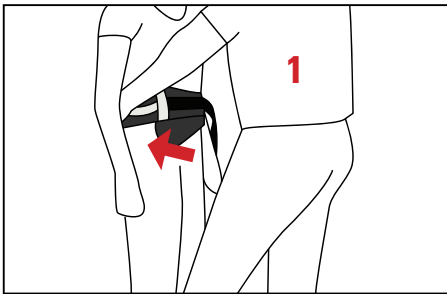
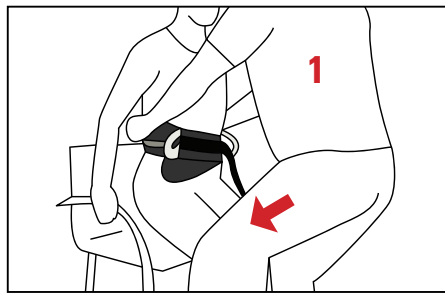
**UK** 1 stands in front of the user and grips the handles on both sides of the Master Belt

1 braces his/her knees against the user's and bends his/her knees with a straight back matching the user's movement as the user sits down in the chair. See illustration

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
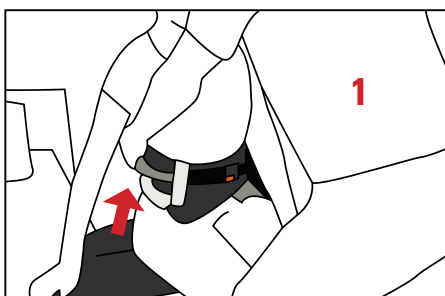
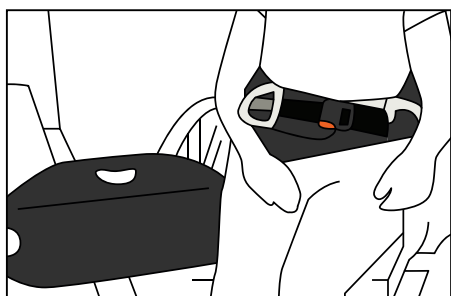
**UK** The user has a Master Belt fitted around the waist, and a Master Glide sliding board is placed between the wheelchair and bed

1 grasps the handles on both sides of the Master Belt and uses weight transfer to move the user in stages from wheelchair to bed

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## 6. Montering af Master Belt DUO / Fitting the Master Belt DUO / Montage des Master Belt DUO

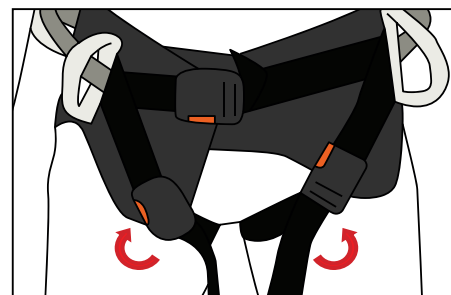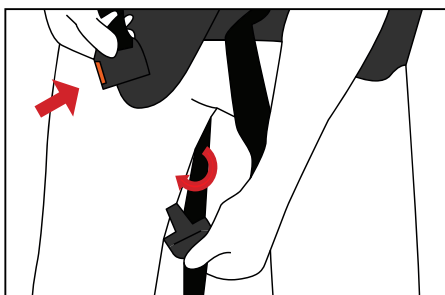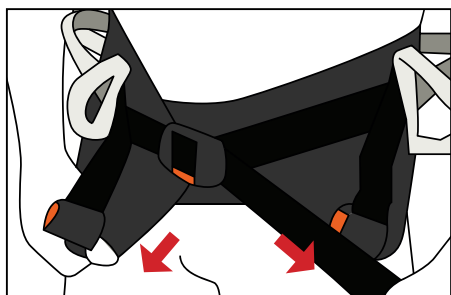**DK** Master Belt DUO monteres rundt om brugers talje. Master Belt fastgøres ved at benytte dets regulerbare låsespænde. Se illustration

Benstropperne på Master Belt DUO monteres rundt om benet og fastgøres ved at benytte det regulerbare låsespænde

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**UK** The Master Belt DUO is mounted around the waist of the user. Master Belt fastened using its adjustable lock buckle. See illustration

The legstraps on the Master Belt DUO are mounted around the leg and secured using the adjustable lock buckle

Subsequently, the second leg strap is mounted, and the Master Belt DUO is now mounted correctly

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# Fordele / Advantages / Vorteile

## Master Belt Junior, Junior DUO & Junior-X

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UK

- Can be adjusted to fit the individual child
- Many grip options
- Provide security for the child
- Provides the assistant many support and training options

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UK

- Can be adjusted to fit the individual user
- Many grip options
- Provides security for the user
- Provides the assistant many support and training options

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UK

- Can be adjusted to fit the individual user
- Leg straps prevent sliding
- Many grip options
- Provides security for the user
- Provides the assistant many support and training options

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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These products comply with the standards for Class 1 products in the Medical Device Regulation (EU) 2017/745.

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