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UK When delivered, the Auto Slide Turner is set to rotate. To make it more usable, just open the Velcro strap on the underside of the Auto Slider and fasten it as needed

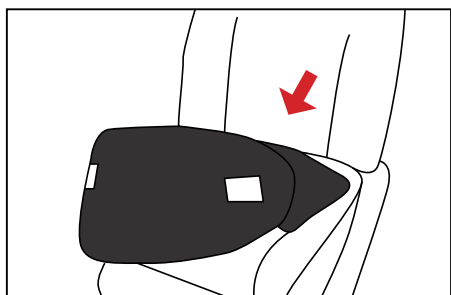
If Auto Slide is used on the driver's seat, the Velcro strap must be attached on the "White" Velcro strap. If Auto Slide is used on the passenger seat, it must be attached to the "Red" Velcro strap. This is to prevent you being bothered by the seat belt folding under you

Auto Slide is positioned on the car seat. The lowest sliding part is placed on the car seat. The uppermost sliding part is guided out to the edge of the seat

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

UK The Auto Slide Turner is now installed and ready for use on the driver's side

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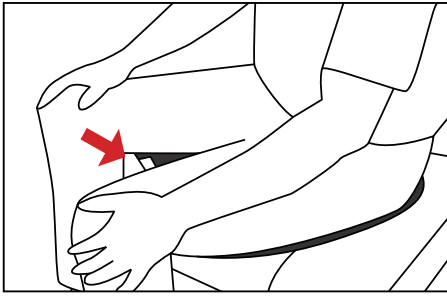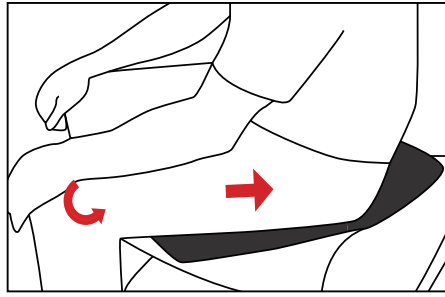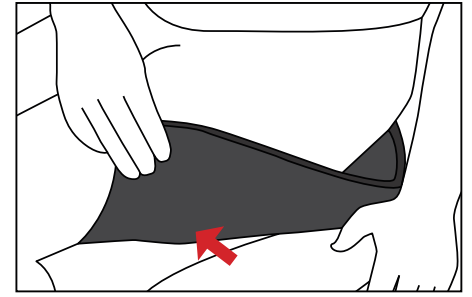
UK When using Auto Slide Turner, you can just sit down on the seat. When the "Fasten Seat Belt" label is in between your legs, you are then sitting correctly.

Slide backwards to the middle of the seat, rotate and slide in towards the backrest

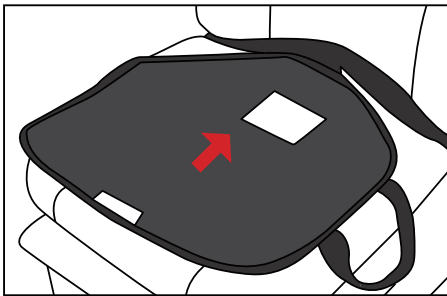
The corners of the bottom glide cushion are cut. This means that Auto Slide locks when you place the weight of your thighs on the front edge of the seat. In doing so, you will avoid sliding around on the seat during transport

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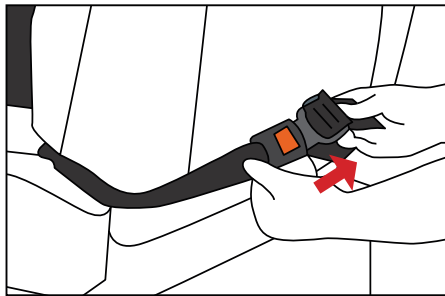
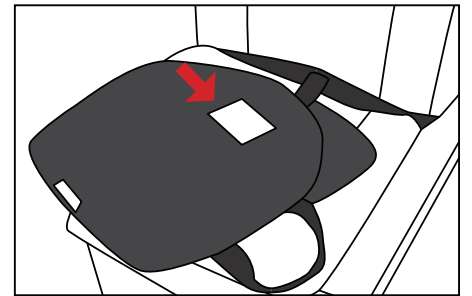
UK 1) Place Auto Slide Back on the car seat with the smallest sliding part at the bottom. The uppermost sliding part, which you sit on, is placed so that the seat belt turns in towards the backrest

Open the locking buckle on the seat belt and guide both ends around the back of the backrest. Then fasten the locking buckle and tighten as required

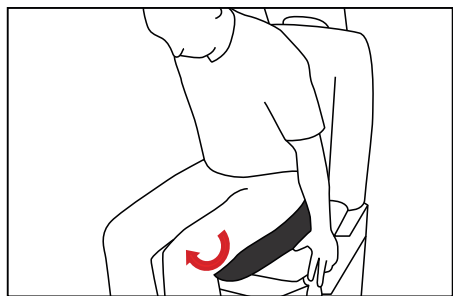
Auto Slide Back is now installed and ready to be used

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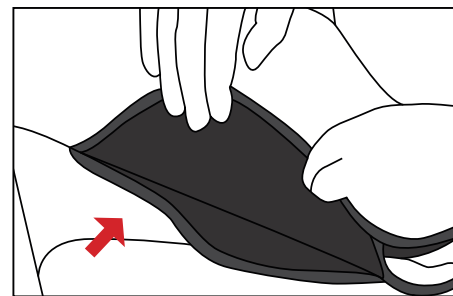
UK Turn the uppermost sliding part diagonally out towards the edge of the car seat and sit down on this. When you can see the "Fasten Seat Belt" label in between your legs, then you are sitting correctly

Slide backwards into the middle of the seat and slide in towards the backrest

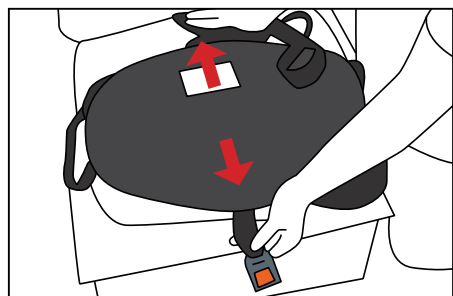
The corners of the bottom glide cushion are cut. This means that Auto Slider locks when you place the weight of your thighs on the front edge of the seat. In doing so, you avoid sliding around on the seat during transport

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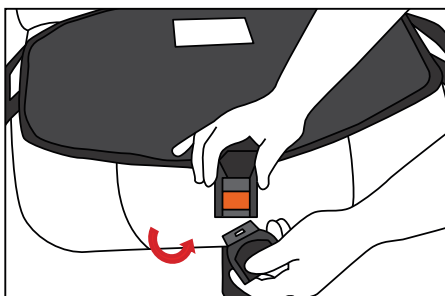
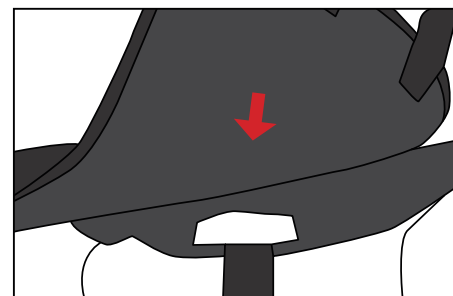
UK Place Auto Slide Seat on the car seat with the smallest sliding part at the bottom. Open the locking buckle on the bottom part of the sliding part and guide the male buckle in between the seat and backrest. Let the female buckle hang on the front edge of the seat

Guide the male buckle down under the car seat and out towards the front edge to the female buckle. Then fasten the locking buckle at the front edge of the seat and tighten as required

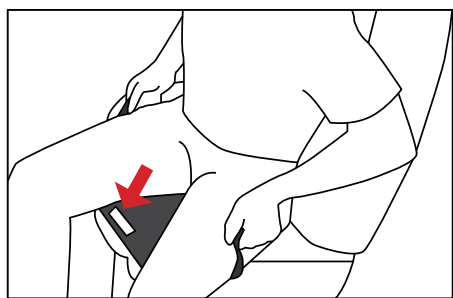
Auto Slide Seat is now installed and ready

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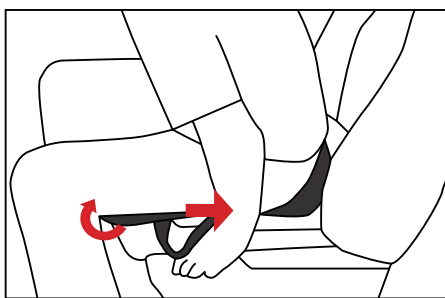
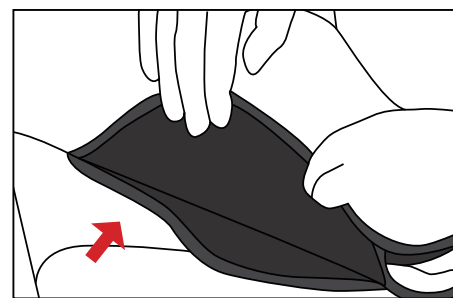
UK Turn the upper sliding part diagonally out towards the edge of the car seat and sit down on it. When you can see the "Fasten Seat Belt" label in between your legs, then you are sitting correctly

Slide backwards into the middle of the seat, rotate and slide in towards the backrest

The corners on the bottom glide cushion cut. This means that Auto Slide locks when you place the weight of your thighs on the front edge of the seat. In doing so, you will avoid sliding around on the seat during transport

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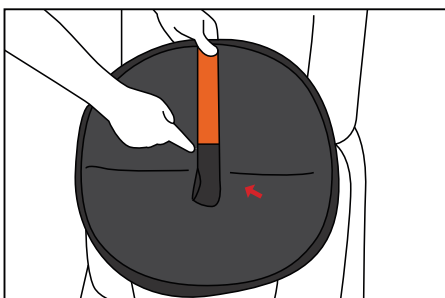
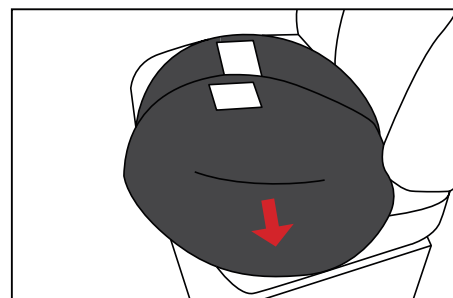
UK If you only need Turn Master to rotate, the Velcro straps on the underside of the non-slip sliding part must be locked all the way to the edge of the turning cushion

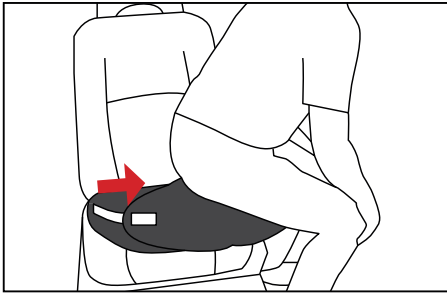
If you need Turn Master to move the seat section from the edge of the car seat into the middle of the seat and then move the seat section all the way back towards the back of the car seat, you must loosen the Velcro straps and adjust this as needed

Now place Turn Master on the car seat with the non-slip part at the bottom. The uppermost sliding part with the black netting is guided out towards the edge of the seat

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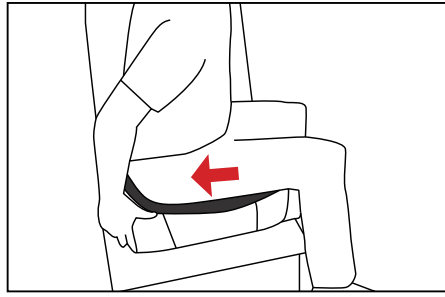
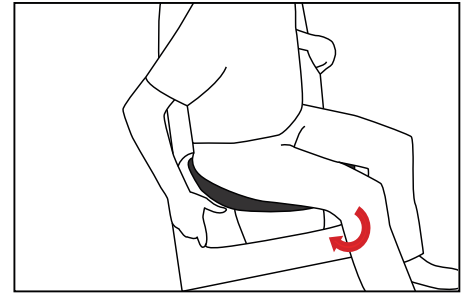
UK Place yourself in the door opening with your back towards the car seat. Sit on Turn Master. The white "this side up and in towards back/head section" label with the arrow must point towards your buttocks

You can now slide in and back into the middle of the seat

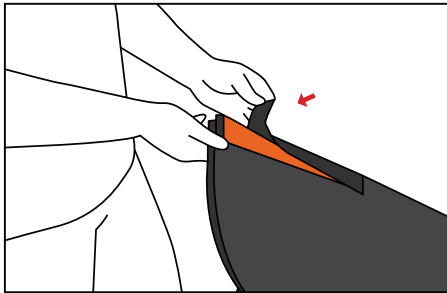
Rotate and now slide in towards the backrest of the seat. This means that you will now slide into place on the seat with Turn Master

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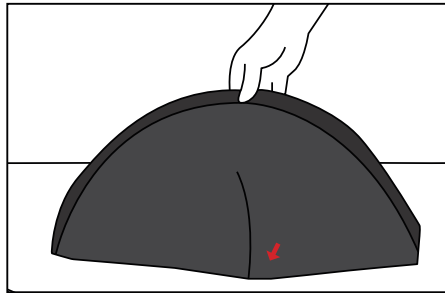
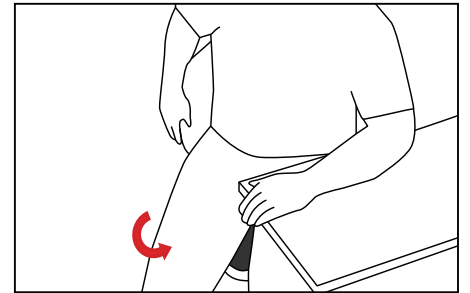
UK Tighten the belt for the back so that the Turn Master can only rotate

Place Turn Master on the chair with the non-slip sliding part at the bottom

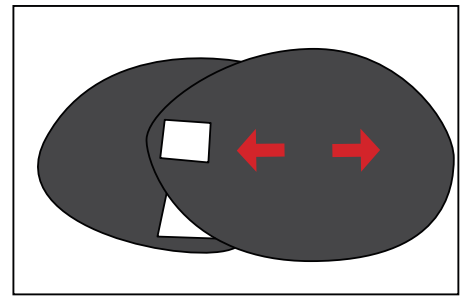
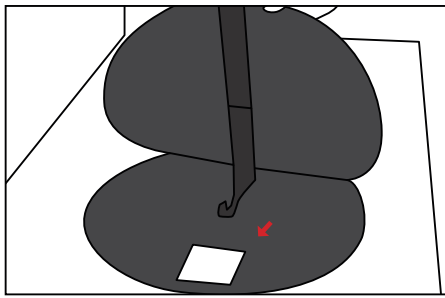
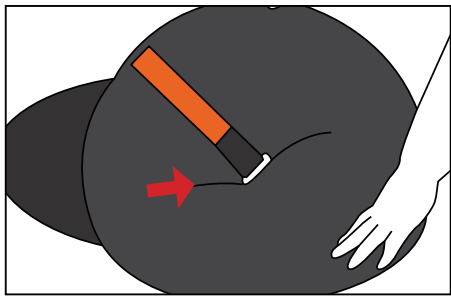
The user sits down on the Turn Master and can now rotate around on the seat

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UK If flexible moving is required so that the user can both slide further into the bed and then rotate in the bed, the Velcro strap is adjusted on the back and the height of the bed is adjusted to the user

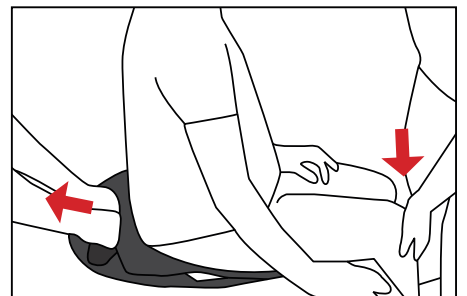
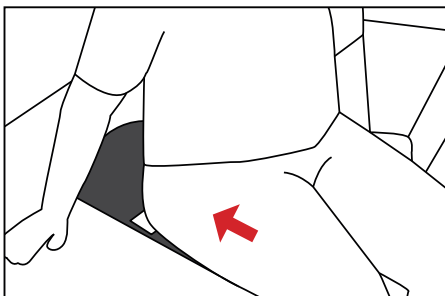
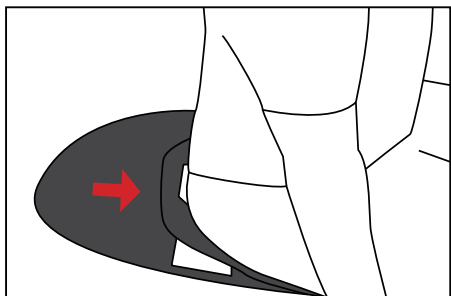
Place the bottom sliding part in the centre of the bed

Move the upper sliding part out towards the edge of the bed. The "This side up and in towards the back/head section" now points in towards the centre of the bed

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UK The user places himself on Turn Master with the buttocks facing the "This side up and in towards the back/head section" label

The user now slides back towards the centre of the bed unassisted, rotates and swings his legs into the bed and then slides into the desired position

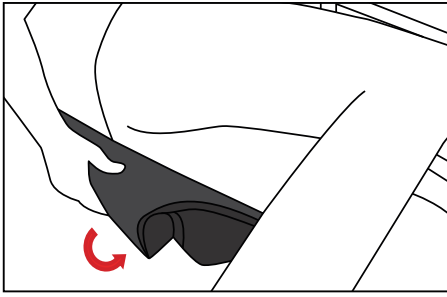
When using Turn Master Multi, the assistants can use the handles and thus help the user further into the bed while the other help supports the user's legs

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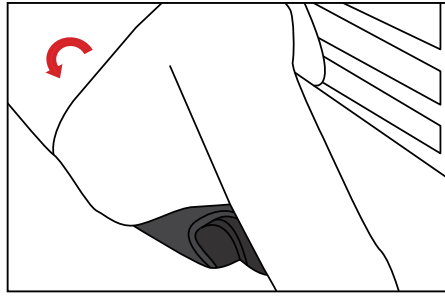
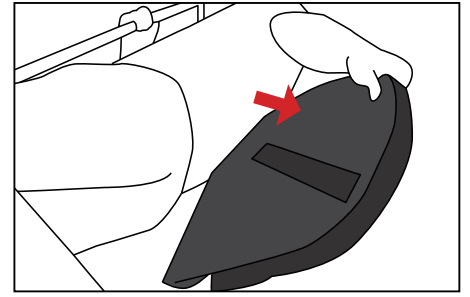
UK The user rolls to one side. Takes hold of both sliding parts of Turn Master and roll this in towards the body

The user then rolls his body to the opposite side, i.e. over the rolled up Turn Master

The user takes hold of Turn Master behind the back, folds it and pulls it away from the body

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UK

- Reduces friction when getting into or out of a vehicle
- Prevents injuries to the back, hip, lower back and/or knees.
- Easy to carry and move around with
- Can be used on all seats in a vehicle
- Locks while driving
- Is skin-friendly and breathable for sitting
- Can be used with all types of vehicle seats

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UK

- Reduces friction when getting into or out of a vehicle
- Prevents injuries to the back, hip, lower back and/or knees.
- Locks while driving
- Is skin-friendly and breathable for sitting
- Can be used with most types of vehicle seats
- Easy to install around the back of a vehicle seat

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UK

- Reduces the friction when moving where the user needs to rotate in the pelvis area
- Reduces the workload for the staff
- Is easy to place and remove after use
- The Multi model has handles so the assistant can take hold of these

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MC 001-1732 [HMI: 49368]
Master Turner Soft - **Large**

Størrelse / Dimensions / Größe:
L 215 x B/W 150

MC 001-2540 [HMI: 49370]
Master Turner Two In One Soft - **Large**

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
Størrelse / Dimensions / Größe:
L 47 x B/W 4 cm

MC 001-2500 [HMI: 49873]
Master Turner DUO Two In One Soft Bund / Bottom
/ Unterteil - **Large**

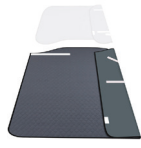Størrelse / Dimensions / Größe:
L 160 x B/W 145 cm

MC 001-2501 [HMI: 49872]
Master Turner DUO Two In One Soft Top / Top /
Oberteil - **Large**

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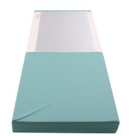
Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantl

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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These products comply with the standards for Class 1 products in the Medical Device Regulation (EU) 2017/745.

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