

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

00_20260427_UK

Master Turner OP


The Original Master Turner System



MC 002-1390



Regulation MDR 2017/745 for medical devices class 1

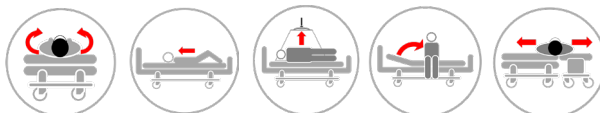

REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
MC 002-1390	54099	Master Turner OP	05744001972038

ISO 12 31 03 21

The Original Nordic Human Handling Concept



Scan Me

Developed & Manufactured by:



Thank you for choosing a product from master care A/S.

Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product can be used to turn a patient to the side or to prone position. The slippery surface on the backside can be used directly on the top of the treatment/operating table or combined with a sliding material to reduce friction. Should be used in combination with a hoist.

Correct use of the product:

The Master Turner OP can be used e.g. on operating tables or in other situations where the user with disabilities are turned to the side or to prone position.

Indication:

For use on the operating table where patients need assistance being turned due to cognitive or physical impairment. It can also be used if the patient is sedated and support is needed during turning, for example due to BMI or limited turning space.

Contraindication:

If the patient is awake and unable to cooperate during the turning process—for example, if the patient makes unexpected movements that may create a risk of falling to the floor—an alternative solution should be found, or additional caregivers should assist with the turning to ensure safety for both the patient and the caregivers.

Warning:

If the surface isn't slippery a piece of sliding material is recommended between the table and the Master Turner OP to uphold a sound and healthy work environment.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Date of manufacture and Life span:

The product's production date can be found on the product label under 'LOT' and is displayed as YYMMDD - e.g. 251020, which refers to the product being produced on the 20th of September 2025.

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Inspection:

Make visual inspections frequently and preferably before every use. Products that no longer provide sufficient friction reduction to ensure a health-safe transfer for both caregiver and user should be discarded. Products should be inspected regularly, and if defective seams or damaged surfaces are found, the product should be discarded.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Turner OP

How to use Master Turner OP:




1. Place the user on one side of the Master Turner OP and along the edge of the bed.



2. Place some folded Nylon Master under the Master Turner OP.



3. Extend the arm the user is to roll over and place the palm up/towards the thigh.

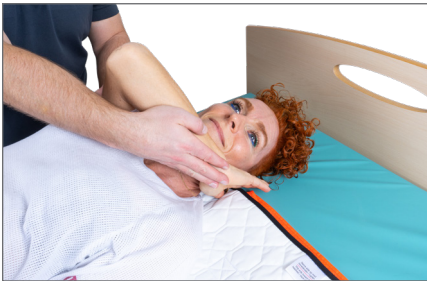

4. Place the opposite arm with the palm facing towards the mattress on the opposite side of the head.



5. Bend the user's legs.



6. Connect the straps to the ceiling hoist and start the turning. The Master Turner OP slides easily on top of the Nylon Master.



7. Place the user's arms comfortably during the prone lying. Leave the user on top of the Master Turner OP but remove the Nylon Master.



8. Use Master Turner OP when returning the user to supine lying. Remember to place the folded Nylon Master under the Master Turner OP prior to the turning.

User Manual - Master Turner OP

Technical Specifications

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

REF - Product Name:	Size:	Handles:	SWL:	Content:	Product weight:
MC 002-1390 Master Turner OP	L 85 x W 120 cm	4	200 kg	Polyester, Cotton, Polyurethane, Polyamide, Silicone	0,72 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Never leave the user lying on the product whilst connected to a hoist.



Do not use as a sling sheet. Can be combined with a hoist for e.g. repositioning in bed.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

These products are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 200 kg



Washing:

This product can be washed in a washing machine at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

