

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

02_20250114_UK

Slide Master & Unique Slide

The Original Slide Master



Unique Slide - L
Slide Master Terry Cloth - L



Slide Master
Elastic Padded - XS



Slide Master
Terry Cloth - XS



Regulation MDR 2017/745 for medical devices class 1



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 002-1477	125007	Unique Slide - L	05744001972083
<input type="radio"/> MC 002-1240	33258	Slide Master Terry Cloth - XS	05744001971857
<input type="radio"/> MC 002-1243	34166	Slide Master Terry Cloth - M	05744001973455
<input type="radio"/> MC 002-1246	34167	Slide Master Terry Cloth - L	05744001971864
<input type="radio"/> MC 002-1340	38143	Slide Master Elastic Padded - XS	05744001972007
<input type="radio"/> MC 002-1345	39032	Slide Master Elastic Padded - M	05744001973493
<input type="radio"/> MC 002-1346	39033	Slide Master Elastic Padded - L	05744001973509

ISO 12 31 03 20
ISO 12 31 03 21

The Original Nordic
Human Handling Concept



Developed &
Manufactured by:



User Manual - Slide Master & Unique Slide

Table of Contents

Product information: page 2

Placing the Slide Master: page 3

Turning to side lying position independently using a Slide Master or with one helper: page 3

Turning to side lying position with two helpers using a Slide Master: page 3

Removing the slide Master from under the user:..... page 4

Moving up in bed using a Slide Master - S: page 4

Technical Specifikations page 5



Thank you for choosing a product from master care A/S.

Product information:

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

Patted Sliding Tube which reduces the friction under a user with reduced function during turning and transfers in bed.

Correct use of the product:

Place the product either directly underneath the user or with a drawsheet between the product and the user. The sliding properties of the nylon on the inner side of the product makes repositioning and transfers in bed easier.

Indication:

The product can be used by users who have difficulty turning in bed due to reduced physical function, pain, and where the use of a turning system (satin or nylon) would overcompensate in relation to the user's needs.

Contraindications:

The product does not have a 'stop function' and there may therefore be a risk of the user slipping over the edge of the mattress. The user must therefore have the cognitive ability to understand and react accordingly to avoid falls - see "Warning"

Warning:

The product does not have an anti-skid /stop function. Therefore, it is important that the user understands e.g., not to reposition too close to the edge of the bed, as this can lead to the user sliding of the bed. It is also important that the user has sufficient trunk stability and support underneath the feet whilst sitting on the edge of the bed to prevent falling/sliding to the floor.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Slide Master & Unique Slide

Placing the Slide Master:




1. Place the Slide Master under the pressure points and where reduced friction is desired during transfer.

Place the Unique Slide with the nonslip facing downwards towards the middle of the mattress.



2. **Unless otherwise specified, the following applies to both Slide Master and Unique Slide.**

The user turns to side lying position. Fold the Slide Master and place it along the spine under the relevant pressurepoints.

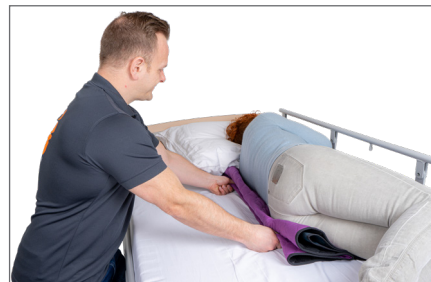

3. Turn the user to the opposite side and straighten out the Slide Master.

Turning to side lying position independently with one helper:



4. Lying on top of the Slide Master the user turns to side lying position with the feet on the sheet using the sliding properties of the Slide Master.



5. Turning with one helper: Ask the user to bend the knees. Helper places one hand on the user's lower hip and the other on the user's knee.



6. Rotate to side lying position.

Turning to side lying position with two helpers:



7. Helper A pulls the draw sheet and helper B rotates the pelvis.



8. Turn the user to side lying position.



9. If the user is not turned all the way to side lying position helper A can make the final adjustments with the draw sheet.

User Manual - Slide Master & Unique Slide

Removing the slide Master from under the user:



10. Turn the user all the way to side lying position.



11. Push the Slide Master under the user.



12. Turn the user to the back lying in the middle of the bed. If necessary use the slide under the user to correct the position. Slide the Slide Master out from underneath the user.

Moving up in bed using a Slide Master - S / M (not possible with size L or Unique Slide):



13. Turn the user to side lying position. Place the Slide Master with the sliding properties towards the head end of the bed. Slide Master must be placed under the shoulders and pelvis.



14. Turn the user to the opposite side and straighten out the Slide master. The Slide Master is now placed with the opening along the edge of the mattress.



15. The user bends the knees, lifts up the head, pushes with the feet and uses the Slide Master to slide upwards in the bed. Remove the slide Master afterwards. For more help elevate the end of the bed.

User Manual - Slide Master & Unique Slide

Technical Specifications

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

REF (Article no.) - Product Name	Size:	Max. User Weight:	Product content:	Product weight:
MC 002-1477 Unique Slide - L	L 180 x W 95 cm	180 kg	69% Polyester, 21% Polyamide, 7% Cotton, 1% Polyurethane, 2% Silicone	1,26 kg
MC 002-1240 Slide Master Terry Cloth - XS	L 50 x W 60 cm	180 kg	74% Polyester, 16% Polyamide, 8% Cotton, 2% Silicone	0,36 kg
MC 002-1243 Slide Master Terry Cloth - M	L 100 x W 60 cm	180 kg	71% Polyester, 19% Polyamide, 8% Cotton, 2% Silicone	0,66 kg
MC 002-1246 Slide Master Terry Cloth - L	L 190 x W 60 cm	180 kg	75% Polyester, 16% Polyamide, 8% Cotton, 2% Silicone	1,14 kg
MC 002-1340 Slide Master Elastic Padded- XS	L 50 x W 60 cm	180 kg	59% Polyester, 20% Polyamide, 10% Cotton, 9% Polyurethane, 2% Silicone	0,42 kg
MC 002-1345 Slide Master Elastic Padded - M	L 100 x W 60 cm	180 kg	57% Polyester, 22% Polyamide, 10% Polyurethane, 9% Cotton, 2% Silicone	0,74 kg
MC 002-1346 Slide Master Elastic Padded - L	L 190 x W 60 cm	180 kg	60% Polyester, 18% Polyamide, 10% Polyurethane, 9% Cotton, 2% Silicone	1,30 kg

Warnings



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 180 kg



Washing:

This product can be washed in a washing machine at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling: This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

