

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.


# (UK) User Manual

03\_20221221\_UK

## Hand Master



MC 001-1002

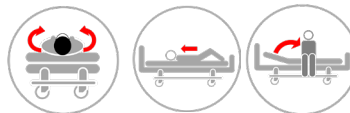

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-1002	86521	Hand Master - Grey	574400197003

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 20

The Original Nordic  
Human Handling Concept



Developed &  
Manufactured by:



# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## **MDR (EU) 2017/745:**

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

The product is to be used bilayed to reduce friction for users with decreased function during e.g. transfer, dressing or training. The product can be used to reduce the risk of shear injuries.

## **Correct use of the product:**

The product can be used to slide a user backwards on the seat, to reduce friction during turning in bed or to adjust a flap on a mattress cover.

## **Warning:**

Do not leave the product underneath the user for a longer time period since this may cause changes in the microclimate of the skin.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the

user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



# User Manual - Hand Master

## Table of Contents

Transferring a less active user backwards in a chair (1 helper)	.....	page 2
Turning a less active user to side lying position in bed (1 helper)	.....	page 2
Place/replace the Mattress Covers flaps using the Hand Master – no hoist (1 helper)	.....	page 3
Technical Specifications	.....	page 4



# User Manual - Hand Master

## Transferring a less active user backwards in a chair (1 helper):



1. The helper places the hand in the Hand Master.



2. The user shifts the weight onto the one buttock. Possibly ask the user to lean to the side and use the armrest for support.



3. The helper slides the hand with the Hand Master under the slightly elevated buttock.



4. Ask the user to push the buttock towards the Hand Master and slide backwards. The helper pushes the user's knee with the opposite hand.



5. Simultaneously the helper slides the hand with the Hand Master backwards. This way the user slides backwards into the chair. Repeat on the opposite so the user sits straight.

## Turning a less active user to side lying position in bed (1 helper):



6. The helper slides the hand into the Hand Master. The user bends the legs and helps with the transfer if possible. The helper places the hand with the Hand Master under the user's hip.



7. The helper places the other hand on the opposite hip/thigh. Slide the user to side lying position by sliding the pelvis backwards. Use the sliding properties of the Hand Master. The user's legs can be used as levers.



8. The helper pulls out the hand from the Hand Master, grips the loose Hand Master and pulls it out.



# User Manual - Hand Master

Place/replace the Mattress Covers flaps using the Hand Master – no hoist (1 helper):



9. Lift the flap, place it folded between the Mattress Cover and the Master Turner. Slide the Hand Master on one or both hands.



10. Place the palm of the hands in the fold on the middle of the flap and slide them forwards and towards the sides to replace the flap. Repeat several times. Place the white straps on top of the flap and slide them underneath the user.



11. Slide the user a bit sideways towards the helper. Walk to the opposite side and take the Hand Master off. Pull out the flap using the white straps. Reattach the Hook-and-Loop tape.

# User Manual - Hand Master

## Technical Specifications

### Materials:

Nylon

### ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

### Article no. - Product Name

### Size:

MC 001-1002 Hand Master - Grey	L 50 x W 18 cm		
--------------------------------	----------------	--	--

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

### User Weight:

Max user weight depends on the users functionality and the transfer being done. All transfers must be sound and healthy for both user and helper.



**Ironing:**  
Do not iron.



### Washing:

This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



**Dry cleaning:**  
Do not dry clean.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



**Bleach:**  
Do not bleach.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric and seams. Products with high friction or other deficiencies must be discarded.



### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.

Find us on:



For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)