

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

11\_20230223\_UK

## ILF Cover

The Original Master Turner System



	Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/>	MC 001-1911	33254	Elastic Cover 1 70x80	5744001970782
<input type="radio"/>	MC 001-1914	35537	Mattress Cover ILF - S	5744001970799
<input type="radio"/>	MC 001-1924	35538	Mattress Cover ILF - M	5744001970805
<input type="radio"/>	MC 001-1925	35529	Mattress Cover ST - M	5744001970812
<input type="radio"/>	MC 001-1931	33256	Mattress Cover ILF Elastic - L	5744001970843
<input type="radio"/>	MC 001-1937	34332	Mattress Cover ILF - XL	5744001970850
<input type="radio"/>	MC 001-2031	40023	Mattress Cover ILF Elastic - XL	5744001971000
<input type="radio"/>	MC 001-2038	40022	Mattress Cover ST - XL	5744001971017
<input type="radio"/>	MC 001-2231	80992	Mattress Cover ILF Elastic - L	5744001973332
<input type="radio"/>	MC 001-1951	81134	Mattress Cover Extenders	5744001970935

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic Human Handling Concept



Developed & Manufactured by:



# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## **MDR (EU) 2017/745:**

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

The intended use for the product is to reduce friction between the user and the mattress to make it easier for the helper and gentler for the user whilst turning, moving up/ downwards, and moving to sitting position on the edge of the bed. This product should always be used in combination with a Master Turner from master care a/s.

## **Correct use of the product:**

The combination of the Mattress Cover and Master Turner from master care A/S constitutes a turning system, which can be used for users with a decrease in function where transfers in bed and between beds can be difficult.

## **Warning:**

Always make sure not to position the user too close to the edge of the mattress if sitting on top of the Mattress Cover since this poses a risk of sliding off/down. If the user is transferred to a sitting position on the edge of the bed, always make sure the knees have a 90-degree

angle, the feet can reach the floor and to have enough non-skid under the user's thighs.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear and washing averages 1-3 years. The product life span depends on the use, user's load, washing and drying temperatures and how often it is washed and dried.

Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



# User Manual - ILF Cover

## Table of Contents

Fitting the mattress cover ILF and ILF Elastic	.....	page 2
Fitting of mattress cover on high mattresses	.....	page 2
Removing the sliding flap from the ILF and ILF Elastic mattress cover manually	.....	page 2
Replacing the sliding flaps	.....	page 3
Technical Specifications	.....	page 4




master-care.de

# User Manual - ILF Cover

## Fitting the mattress cover ILF and ILF Elastic:



1. Place cover at the head of the mattress with the black flaps facing upwards. Hold the cover firmly against the mattress and fold the mattress in half.

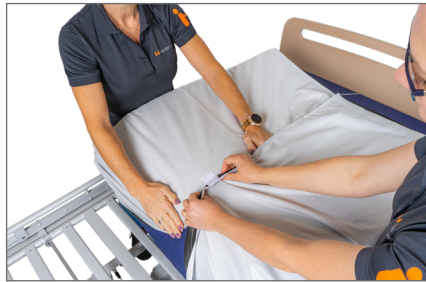

2. Close the ends of the mattress cover on the underside of the mattress with the Hook and Loop tape. Straighten out the mattress and fold the other end. Close the other end of the cover in the same way.



3. The mattress cover is now fitted. Check at least once a day. Place the Master Turner on top of the Mattress Cover. The turn system is ready for use.

## Fitting of mattress cover on high mattresses:



4. If you are fitting the mattress cover on a pressure relieving mattress, you can purchase 4 mattress cover extensions.



5. If there is a mattress topper on top of a regular mattress, fit the mattress cover around both mattresses.


## Removing the sliding flap from the ILF and ILF Elastic mattress cover manually:



6. Helper A holds on to the Master Turner, while Helper B loosens the two Hook and Loop strips on the sliding flap.



7. Slide the two straps underneath the user's neck, lower back, or knees.

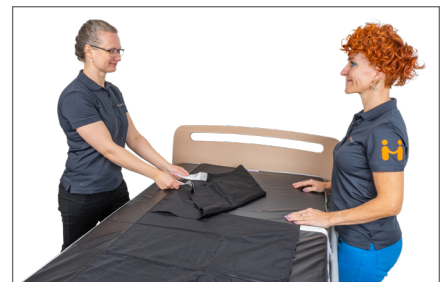

8. Helper 2 pulls the straps out on the other side. Let the flap hanging down visibly.

# User Manual - ILF Cover

## Replacing the sliding flaps:



9. Helper A takes hold of the Master Turner and tightens it around the user's body.



10. Helper B folds the flap and places their hands on the fold.



11. Helper B slides the middle of the flap underneath the user. Smooths it out towards the sides.



12. The straps are tucked on top of the gliding flap and under the user's cavities - neck, lower back, the knees, and over the heels.



13. Helper B grips the Master Turner. Helper A pulls out the straps and fix them to the cover.

# User Manual - ILF Cover

## Technical Specifications

### Materials:

Nylon, polyester, polyurethane

Article no. - Product Name	Size:	Sliding piece	Number of flaps	Max. User Weight
MC 001-1911 Elastic Cover 1 70x80	L 70 x W 80 cm	L 70 x W 60 cm	1	150 kg
MC 001-1914 Mattress Cover ILF - S	L 70 x W 80 cm	L 70 x W 60 cm	1	150 kg
MC 001-1924 Mattress Cover ILF - M	L 135 x W 80 cm	L 135 x W 60 cm	2	180 kg
MC 001-1925 Mattress Cover ST - M	L 135 x W 80 cm	L 135 x W 60 cm	0	180 kg
MC 001-1931 Mattress Cover ILF Elastic - L	L 200 x W 80 cm	L 200 x W 60 cm	3	200 kg
MC 001-1937 Mattress Cover ILF - XL	L 200 x W 100 cm	L 200 x W 80 cm	3	200 kg
MC 001-2031 Mattress Cover ILF Elastic - XL	L 200 x W 100 cm	L 200 x W 80 cm	3	200 kg
MC 001-2038 Mattress Cover ST - XL	L 200 x W 100 cm	L 200 x W 80 cm	3	200 kg
MC 001-2231 Mattress Cover ILF Elastic - L	L 200 x W 80 cm	L 200 x W 60 cm	3	200 kg
MC 001-1951 Mattress Cover Extenders	L 70 x W 4 cm	-	-	- kg

## ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



### User Weight:

Recommended max. user weight: 150 kg or 200 kg. See table for more info.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



### Washing:

Products *with* Hook-and-Loop tape can be washed in a washing machine at max. 60 °C. Products *without* Hook-and-Loop tape can be washed in a washing machine at max. 85 °C. Gentle washing significantly prolongs the durability.



### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)

