

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

Master Belt

15_20260908_UK



MC 003-1020



MC 003-1026



MC 003-1120



MC 003-1125



MC 003-1126



MC 003-1127



MC 003-1220




MC 003-1227



MC 003-1325



MC 003-1326

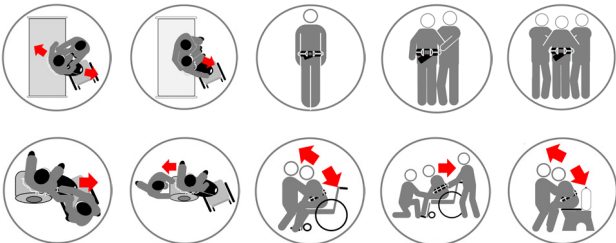

REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
MC 003-1020	36484	Master Belt Junior	05744001972304
MC 003-1026	39018	Master Belt Junior - DUO	05744001973578
MC 003-1120	33237	Master Belt Multi Velour - S	05744001972311
MC 003-1125	33238	Master Belt Multi Velour - M	05744001972328
MC 003-1126	33239	Master Belt Multi Velour - L	05744001972335
MC 003-1127	42302	Master Belt Multi Velour - XL	05744001972342
MC 003-1220	45645	Master Belt Multi Non-Slip - S	05744001972373
MC 003-1227	45648	Master Belt Multi Non-Slip - XL	05744001972380
MC 003-1320	39019	Master Belt Duo - S	05744001972397
MC 003-1325	39020	Master Belt Duo - M	05744001972403
MC 003-1326	39021	Master Belt Duo - L	05744001974193

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 15 04

The Original Nordic
Human Handling Concept



SCAN ME

Developed &
Manufactured by:



master-care.dk

User Manual - Master Belt

Table of Contents

Important information before using the product(s):	page 2
Fitting the Master Belt:	page 3
Fitting the Master Belt DUO:	page 3
Sitting to standing position:	page 3
Walking training with Master Belt:	page 4
Sitting the user down from a standing position:	page 4
Transfer between wheelchair and bed:	page 4
Technical Specifications:	page 5



Thank you for choosing a product from master care A/S.

Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the costumer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is used while facilitating walk or during sitting transfers with users who are physically and/ or mentally disabled. The product is a facilitating aid and should not be used as a lifting aid as this can cause injuries to user and/or helper.

Correct use of the product:

Basic Belt is mounted around the user's waist. With the handles the helper can facilitate the user in different transfers without directly touching the user. The handles on the Basic Belt gives the helper the opportunity to get a more upright standing-position and therefore a better working posture.

Indication:

Used for persons who, due to reduced physical or cognitive function, need light support during walking, in the repositioning from a sitting to a standing position, or in sitting transfers.

Contraindications:

If the user has no control over their upper body, another method must be used to change their position.

Warning:

The Basic Belt is not intended as lifting-belt of any kind. Nor is the Basic Belt intended for use in combination with a hoist. The Master Belt must not be used to restrain the user.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product. master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Date of manufacture and Life span:

The product's production date can be found on the product label under 'LOT' and is displayed as YYMMDD - e.g. 251020, which refers to the product being produced on the 20th of September 2025. The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 2-5 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Inspection:

Make visual inspections frequently and preferably before every use. Products with defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Belt

Fitting the Master Belt:



1. The Master Belt is mounted around the waist of the user. Master Belt is fastened using its adjustable buckle.



2. To customize the Master Belt to the user's waistline, pull the end of the belt itself and in the opposite direction of the strap for tightening.

Fitting the Master Belt DUO:



3. The Master Belt DUO is mounted around the waist of the user. Master Belt is fastened using its adjustable buckle.



4. The leg straps on the Master Belt DUO are mounted around the legs and secured using the adjustable buckle.



5. The Master Belt is mounted and ready for use.

Sitting to standing position:



6. The user and the helper sit on the bed frame. The helper supports the user on the hips.



7. The helper supports the user whilst standing up.



8. A Rollator can be placed in front of the user.

User Manual - Master Belt

Walking training with Master Belt:



9. Helper 1 stands beside the user whilst guiding the user by using the handles on the Master Belt.



10. If there are 2 helpers, the 2. helper can support the user's trunk or guide the legs.

Sitting the user down from a standing position:



11. The helper stands in front of the user and grips the handles on both sides of the Master Belt.



12. The helper supports his/her knees against the user's knee whilst bending their knees with a straight back matching the user's movement as the user sits down in the bed/chair.

Transfer between wheelchair and bed:



13. The user has a Master Belt fitted around the waist, and a Master Glide sliding board is placed between the wheelchair and bed.



14. The helper grasps the handles on both sides of the Master Belt and supports the transfer to the side.

User Manual - Master Belt

Technical Specifications

REF Article no. - Product Name	Size:	Number of handles:	Max. user weight:	Product weight:
MC 003-1020 Master Belt Junior	Waist size: 45 - 65 cm	5	150 kg	0,30 kg
MC 003-1026 Master Belt - Junior DUO	Waist size: 45 - 65 cm	5	150 kg	0,32 kg
MC 003-1120 Master Belt Multi Velour - S	Waist size: 65-100 cm	7	150 kg	0,32 kg
MC 003-1125 Master Belt Multi Velour - M	Waist size: 90-120 cm	9	150 kg	0,36 kg
MC 003-1126 Master Belt Multi Velour - L	Waist size: 95-130 cm	11	150 kg	0,42 kg
MC 003-1127 Master Belt Multi Velour - XL	Waist size: 130-150 cm	13	150 kg	0,44 kg
MC 003-1227 Master Belt Multi Non-Slip - XL	Waist size: 130-150 cm	13	150 kg	0,50 kg
MC 003-1325 Master Belt Duo - M	Waist size: 70-120 cm	9	150 kg	0,6 kg
MC 003-1326 Master Belt Duo - L	Waist size: 95-130 cm	11	150 kg	0,66 kg

Materials:

Mohair, Alvelen, Nylon, Polyester

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 150 kg



Washing:

This product can be washed in a washing machine at max 40 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. When damaged the product must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos
Quick Guide etc., visit
www.master-care.dk