

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

20_20250115_UK

Sling Sock

Pat. BA 2011 00176



MC 003-9030



MC 003-9031
MC 003-9034
MC 003-9035



MC 003-9032



MC 003-9033



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 003-9030	106546	Sling Sock Regular - S	05744001972540
<input type="radio"/> MC 003-9031	48792	Sling Sock Basic - M	05744001972557
<input type="radio"/> MC 003-9032	48793	Sling Sock Toilet - M	05744001972564
<input type="radio"/> MC 003-9033	48794	Sling Sock Amputation - M	05744001972571
<input type="radio"/> MC 003-9034	55707	Sling Sock Basic - L	05744001972588
<input type="radio"/> MC 003-9035	82003	Sling Sock Basic - XL	05744001972595

Regulation MDR 2017/745 for medical devices class 1



ISO 12 36 21 37

The Original Nordic
Human Handling Concept



SCAN ME

Developed &
Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is used to make it easy and more comfortable to place and remove a sling. It reduces the risk of shear injuries and skin tearing on the thighs and the back while lifting.

Correct use of the product:

Mount the product(-s) around the back and/or the leg straps of the sling before the user is lifted. After mounting the product and before lifting the user the sling can be adjusted into correct position using the slippery surface of the product(-s).

Indication:

If the user is at risk of getting shear injuries on the thighs when lifting or applying slings that are ergonomically heavy for the carer, Sling Socks can be used to advantage.

Contraindications:

Do not use the Sling Sock directly on open wounds. Dress the wound before use of the Sling Sock.

Warning:

It is recommended to remove the product(-s) when the user isn't being lifted. This reduces the risk of pressure wounds and overheating in the skin.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 2-5 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Sling Sock

Table of Contents

Placing the sling sheet using Sling Socks. Seated user – 1 helper:	page 2
Placing the sling sheet under a lying person using Sling Socks - 1 helper:	page 3
Technical Specifications	page 4



User Manual - Sling Sock

Placing the sling sheet using Sling Socks. Seated user – 1 helper:



1. The sling sheet is placed behind the user's back. If it is difficult to place, a Sling Vest (MC 003-9040), Master Glide Thor (MC 001-7061) or Nylon Master can be used.



2. Slide the hand into the Sling Sock from the wide end and bend up the front edge. Place the Sling Sock with the curved side towards the user's pelvis.



3. Slide the Sling Sock under the user's leg – preferably under the user's knee and slide it into place. Make sure the feet are supported.



4. Slide the leg strap of the sling sheet through the Sling Sock from the outside and pull the strap into place from the opposite side. Be aware that the strap does not twist.



5. When lifting, the leg strap slides into place inside the Sling Sock under the user's leg. Thus, the user avoids shear and scraping of the skin due to the comfortable padding.



6. To remove the sling sheet first pull out the leg strap from the Sling Sock.



7. Pull or roll the Sling Sock out from under the thigh, while supporting the leg. Thus, retaining the user's perfect seated position.

User Manual - Sling Sock

Placing the sling sheet under a lying person using Sling Socks - 1 helper:



8. The sling sheet is placed behind the user's back. If it is difficult to place, a Sling Vest (MC 003-9040), Master Glide Thor (MC 001-7061) or Nylon Master can be used.



9. Slide the hand into the Sling Sock from the wide end and bend up the front edge.



10. Place the Sling Sock with the curved side towards the user's pelvis. Slide the Sling Sock under the user's knee with the fold first.



11. Pull out the hand from the Sling Sock while supporting the other end. Grab both ends of the Sling Sock and slide it into place towards the user's pelvis.



12. Slide the leg strap of the sling sheet through the Sling Sock from the outside.



13. Walk to the opposite side of the user for a good work position. Pull the leg strap through the Sling Sock and into place. Be aware that the strap does not twist.



14. Elevate the bedrests for at seated position. Begin hoisting the user. The sling sheet slides into place under the user without shear or scraping of the skin.



15. Transfer the user into the wheelchair. When removing the sling sheet pull out the leg straps from the Sling Socks.



16. Pull or roll out the Sling Sock from under the thigh, while supporting the leg. Thus, retaining the user's perfect seated position.

User Manual - Sling Sock

Technical Specifications

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Materials:

55% Polyester, 41% polyamide, 5% Silicone.

Article no. - Product Name:	Size:	Max. user weight (SWL):	Product weight:
MC 003-9030 Sling Sock Regular - S	L 52 x W 13-15 cm	250 kg	0,14 kg
MC 003-9031 Sling Sock Basic - M	L 51 x W 17-28 cm	250 kg	0,18 kg
MC 003-9032 Sling Sock Toilet - M	L 45-62 x W 16-28 cm	250 kg	0,20 kg
MC 003-9033 Sling Sock Amputation - M	L 56-73 x W 18-25 cm	250 kg	0,20 kg
MC 003-9034 Sling Sock Basic - L	L 60-70 x W 22-29 cm	250 kg	0,30 kg
MC 003-9035 Sling Sock Basic - XL	L 56 x W 22-33 cm	250 kg	0,30 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Never leave the user lying on the product whilst connected to a hoist.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 250 kg



Washing:

This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability. The product can be reused by others when cleaned and intact - see "Maintenance"



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos
Quick Guide etc., visit
www.master-care.dk

