

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

28_20250825_UK

Master Stop



Master Stop



Master Stop Måtte

	Article no.	HMI no.	Name	UDI-DI/GTIN	Size
<input type="radio"/>	MC 001-1929	102406	Master Stop 30x0,6m	5744001970829	L 30 x W 0,6 m
<input type="radio"/>	MC 001-1956	88227	Master Stop Måtte - S	5744001970959	L 40 x W 45 cm
<input type="radio"/>	MC 001-1957	88228	Master Stop Måtte - L	5744001970966	L 80 x W 60 cm
<input type="radio"/>	MC 001-1959	81192	Master Stop 30x1,2m	5744001970973	L 30 x W 1,2 m



1. Place the StayUp Strap on top of the pants around the thigh.



2. Pass the strap through the plastic ring.



3. Tighten the strap and close the hook band against the elastic band.

The Original Nordic
Human Handling Concept



SCAN ME

Developed &
Manufactured by:



User Manual - Master Stop

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

Intended purpose:

The intended use is to increase the friction between two surfaces e.g. table and chopping board or between a user's feet and the mattress.

Indication:

Can be used if the user has difficulty keeping their feet on the floor when pushing themselves up in bed or when standing up. It can also be used if the user has difficulty keeping the cutting board steady, for example when preparing food.

Kontraindication:

The product should be used with caution if the user has a tendency to trip over objects. No other contraindications are known.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product the activity will be ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of the product with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, load and how often it is washed/dried.

Check that the friction on the surface of the product is sufficient for the intend.

If not or if the product is broken it must be discarded.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

ADVARSLER



Make sure the user isn't at risk of falling over the Master Stop.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

For more information, videos Quick Guide etc., visit www.master-care.dk



User Weight:

No max. user weight.



Washing:

This product can be washed at 60 °C - hand-wash or in a machine. When clean and if intact the product can be reused by others.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

Tumbling drying is NOT recommended since the product can melt/shrink.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

The expected life span of products with daily wear, tear and washing averagely are 1-3 years. The products life span depends on the use, load and how often it is washed/dried. Check that the product. If the products is defect /broken it must be discarded. Make visual inspections frequently and preferably before every use.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.