

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

29_20221209_UK

Auto Slide

The Original Auto Slide



MC 001-9100



MC 001-9110



MC 001-9120



MC 001-8210



MC 001-8200



MC 001-8230



Regulation MDR 2017/745 for medical devices class 1



ISO 12 12 81 01

The Original Nordic
Human Handling Concept



Developed &
Manufactured by:



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 001-8110	35526	Auto Slide Seat - M	5744001971710
<input type="radio"/> MC 001-8200	124600	Auto Slide Back - Velour	5744001971727
<input type="radio"/> MC 001-8210	124601	Auto Slide Turner - Velour	5744001971734
<input type="radio"/> MC 001-8230	124984	Auto Slide Seat - Velour	5744001971741
<input type="radio"/> MC 001-9100	35601	Auto Slide Turner - M	5744001971802
<input type="radio"/> MC 001-9110	38950	Auto Slide Back - Air	5744001971819
<input type="radio"/> MC 001-9120	38951	Auto Slide Seat - Air	5744001971826
<input type="radio"/> MC 001-9140	128782	Auto Slide Turner - L Auto Slide Turner - Air	5744001971840

Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The intended use is to reduce friction between the user and the car seat when moving in and out of the car. Especially useful for users with back / hip pain or to prevent pain during movement.

Correct use of the product:

Mount the product on the car seat. Rotate it slightly towards the doorframe and sit down on top of product. Slide/rotate to the correct sitting position on the car seat. Small adjustments to the sitting position can be made during driving without having to lift the pelvis from the seat.

Warning:

Always use the safety belt while using this product.

Risk assessment:

Always make a risk assessment and a functional assessment of the

user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material, buckles, or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.


Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

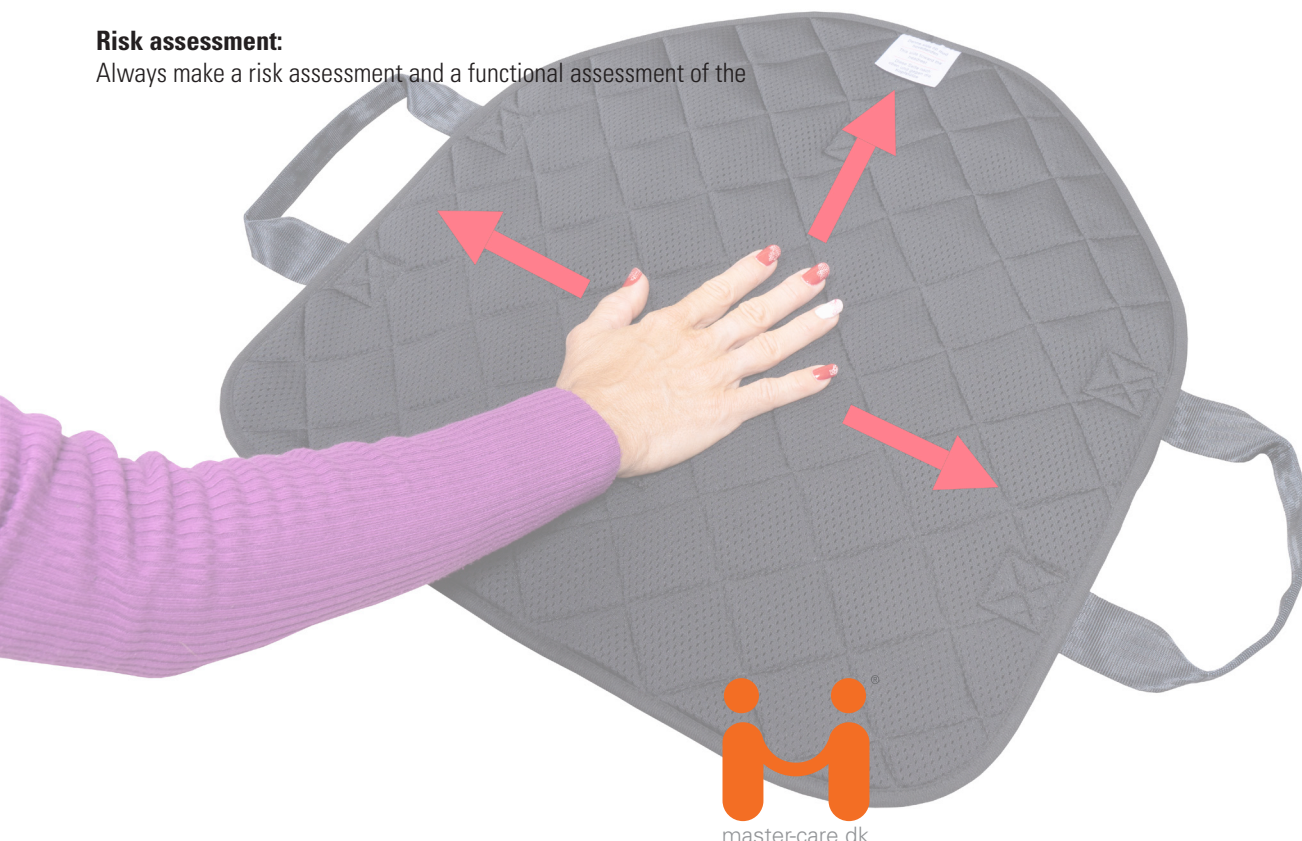

User Manual - Auto Slide

Table of Contents

Mounting the Auto Slide Seat	page 2
Mounting the Auto Slide Back	page 2
Mounting the Auto Slide Turner	page 2
How to use the Auto Slide	page 3
Safety and functionality	page 3
Technical Specifications	page 4



User Manual - Auto Slide

Mounting the Auto Slide Seat:



Place the Auto Slide on the car seat with the smallest part downwards and the two nylon surfaces towards each other.



The pin part of the buckle goes through the space between the seat and the back of the car seat. Mount the pin part around the car seat and click the parts together.



Fasten the strap and ensure that the bottom piece of the Auto Slide is securely mounted on the car seat. Turn the upper part of the Auto Slide Seat towards the door.

Mounting the Auto Slide Back:



Place the Auto Slide on the car seat with the smallest part downwards and the two nylon surfaces towards each other.



Mount the Auto Slide Back around the back of the seat using the buckle. Turn the upper part of the Auto Slide Back towards the door.

Mounting the Auto Slide Turner:



Place the Auto Slide on the car seat with the smallest part downwards and the two nylon surfaces towards each other.



The Auto Slide Turner lies loose on the car seat and is easy to reposition. Turn the upper part of the Auto Slide Turner towards the door.

User Manual - Auto Slide

How to use the Auto Slide:




Position the pelvis on the Auto Slide Back so that the rounding and the label is facing towards the back of the seat.



Turn and slide the pelvis towards the middle of the car seat. A small slide backwards can easily be made if necessary



With an Auto Slide in the car, it is super easy to adjust your position in all directions while driving. The Seat and Back models can be placed further out on the edge of the seat which eases the access into tall cars/seats.

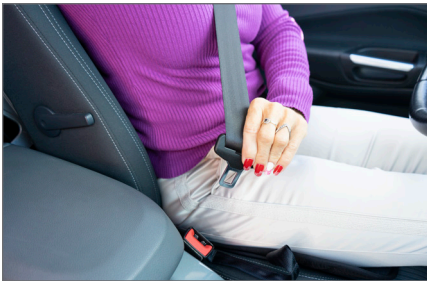

Fasten the seat belt and extend the legs. The weight of the legs automatically fixes the Auto Slide Back while driving. This way it is super easy to make small adjustments sideways and backwards while driving.

Safety and functionality:



All the Auto Slides automatically locks when one straightens out the legs. Because the lower sliding surface is big at the back and very narrow at the front the weight of the legs pushes the front edge of the upper sliding surface towards the seat and creates friction.



What makes the Auto Slide Back so unique is the ability to easily slide in all directions AND it reduces any unnecessary friction. The forwards sliding ability is controlled by the thighs pushing against the car seat



The handles can be used by both user and helper to reposition while driving or when getting into the car.

User Manual - Auto Slide

Technical Specifications

Materials:

Polyester, polyethylene, nylon

Article no. - Product Name	Size:	Handles:	Max. User Weight
MC 001-8110 Auto Slide Seat - M	L 48 x W 48 cm	2	180 kg
MC 001-8200 Auto Slide Back - Velour	L 50 x W 50 cm	2	180 kg
MC 001-8210 Auto Slide Turner - Velour	L 50 x W 50 cm	2	180 kg
MC 001-8230 Auto Slide Seat - Velour	L 50 x W 50 cm	2	180 kg
MC 001-9100 Auto Slide Turner - M	L 48 x W 48 cm	0	180 kg
MC 001-9110 Auto Slide Back - Air	L 50 x W 50 cm	2	180 kg
MC 001-9120 Auto Slide Seat - Air	L 50 x W 50 cm	2	180 kg
MC 001-9140 Auto Slide Turner - Air	L 50 x W 50 cm	2	180 kg

WARNINGS



Never leave the product on the floor.

Always fasten your seat belt during driving.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 180 kg



Ironing:

Do not iron.



Washing:

This product can be washed in a washing machine at max 40 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Dry cleaning:

Do not dry clean.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Bleach:

Do not bleach.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams, buckles and straps. Products with high friction or other deficiencies must be discarded.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



User Manual:

Always read the User Manual thoroughly before using the product.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.

For more information, videos Quick Guide etc., visit www.master-care.dk

Find us on:

