

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

31\_20221209\_UK

## Mini Master



MC 003-2020



MC 003-2021



MC-003-2022



MC 003-3020



MC 003-3021



MC 003-3022



Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 15 02

The Original Nordic  
Human Handling Concept



Developed &  
Manufactured by:



master-care.dk



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 003-2020	36416	Mini Master Non-slip – S	5744001973585
MC 003-2021	37424	Mini Master Non-slip – M	5744001971529
MC 003-2022	37425	Mini Master Non-slip – L	5744001972458
MC 003-3020	36417	Mini Master Velour – S	5744001973592
MC 003-3021	37426	Mini Master Velour – M	5744001972465
MC 003-3022	37427	Mini Master Velour – L	5744001973608

# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## **MDR (EU) 2017/745:**

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

The product is used as a guiding strap during transfers sitting, transfer from sitting to standing position, guiding legs into bed e.g. The product provides a better ergonomic working posture for the helper.

## **Correct use of the product:**

The Mini Master is placed with the anti-slide side against the user. Can be used in transfer situations to obtain a good working posture for the helper.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear 2-5 years. The products life span depends on the use, user's load, washing and drying temperatures and how often it is washed and dried.

Products without nylon sliding material have a much longer life span.

Products with insufficient friction or reduction of friction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



# User Manual - Mini Master

## Table of Contents

Helping user to a standing position	.....	page 2
Move the user upwards in bed	.....	page 2
Sitting transfer from chair to bedside or opposite	.....	page 2
Transferring legs into the bed	.....	page 3
Transferring leg out of bed	.....	page 3
Technical Specifications	.....	page 4





# User Manual - Mini Master

## Helping user to a standing position:



1. While the user leans forward the helper places the Mini Master behind the users pelvic with the non-slip/ velour side towards the user.



2. The helper supports the user's legs with his/ her own knees.



3. The helper grabs the Mini Master handles and supports the user to a standing position using weight transferring.

## Move the user upwards in bed:



4. Place a piece of sliding material under the user's head, shoulders and if needed bottom. Bend the user's legs. Place the Mini Master under the user's pelvis with the non-slip/velour side towards the user.



5. A helper on each side grabs the handle of the Mini Master. Ask the user to lift the pelvis if possible. Using weight transfer the helpers slide the user upwards in bed.



6. To remove the Mini Master bend the user's leg. Remove the Mini Master by pulling slightly towards the users knees. Rotate the user's pelvis if resistance is detected.

## Sitting transfer from chair to bedside or opposite:



7. Place the wheelchair as close to the bed as possible. If needed place a Master Glide transfer board under the users bottom.



8. Place Mini Master under the user's pelvis with the non-slip/velour side towards the user.



9. While holding on to the handles the helper uses weight transferring to support the user coming forward and upwards while doing the transfer to the bed.

# User Manual - Mini Master

## Transferring legs into the bed:



10. Place the Mini Master with the non-slip/velour side towards the user. With extended arms the helper extends his/her knees. The user's legs should be in the same level as the mattress.



11. While the user rotates into bed, the helper supports the user's legs into the bed. A piece of sliding material can be placed on the mattress to minimize friction.



12. If the user has heavy legs it is recommended moving one leg at a time. If the user is able to move one leg into the bed then place the user with the active leg nearest to the bed. Place the Mini Master under the other leg.

# User Manual - Mini Master

## Technical Specifications

### Materials:

Polyester, Polyurethan, Nylon and Polyethylene.

### ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name	Size:	Number of handles	Max. User Weight
MC 003-2020 Mini Master Non-slip - S	L 50 x W 22 cm	4	200 kg
MC 003-2021 Mini Master Non-slip - M	L 60 x W 22 cm	4	200 kg
MC 003-2022 Mini Master Non-slip - L	L 70 x W 22 cm	4	200 kg
MC 003-3020 Mini Master Velour - S	L 50 x W 22 cm	4	200 kg
MC 003-3021 Mini Master Velour - M	L 60 x W 22 cm	4	200 kg
MC 003-3022 Mini Master Velour - L	L 70 x W 22 cm	4	200 kg

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Make sure that all relevant brakes are activated before transferring the user.



#### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



#### User Weight:

Recommended max. user weight: 200 kg



#### Washing:

This product can be washed in a washing machine at max 40 °C. Gentle washing and dry tumbling significantly prolongs the durability.



#### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



#### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



#### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



#### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



#### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



#### Ironing:

Do not iron.



#### Dry cleaning:

Do not dry clean.



#### Bleach:

Do not bleach.



#### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with low friction or other deficiencies must be discarded.



#### User Manual:

Always read the User Manual thoroughly before using the product.



#### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos  
Quick Guide etc., visit  
[www.master-care.dk](http://www.master-care.dk)



Find us on:

