

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

34_20221209_UK

Master Bed Grip



MC 003-3510




MC 003-3520



MC 003-3530



MC 001-1313

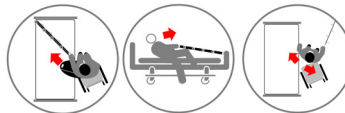

-
-
-
-

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 003-3510	36414	Master Bed Grip - Soft	5744001972472
MC 003-3520	36415	Master Bed Grip - Hard	5744001972489
MC 003-3530	63305	Master Bed Grip - Single	5744001972496
MC 001-1313	89661	BASIC Grip	5744001970416

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 12 02

The Original Nordic
Human Handling Concept



Developed &
Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is mounted on the bedframe and can be used when the user pulls him or herself to sitting position in the bed.

Correct use of the product:

The user's ability to grab the handles on the product is vital for the choice of product.

Warning:

The product can be mounted on most bed frames and footboards. If there is any doubt that the bed can sustain the pull force/weight from the user – contact the manufacture of the bed for confirmation.

Risk assessment:

Always make a risk assessment and a functional assessment of the

user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 2-5 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Bed Grip

Mounting the Master Bed Grip:



1. Mount the Master Bed Grip to e.g., the bed frame or the footboard to get the right angle when pulling.



2. Master Bed Grip - Soft and Hard: The users can pull them self to a sitting position by pulling the handles one after the other.



3. Master Bed Grip - Single: Mount the Master Bed Grip to e.g., the bed frame or the footboard. The length of the Master Bed Grip - Single can be regulated via the black buckle.

Getting from supine lying to sitting position:



4. Place the Master Bed Grip close to the user e.g., along the user's side for easy accessibility.



5. Use the handles one by one to get to the sitting position. Elevation of the head rest can be an advantage.



6. Move the legs from the bed towards the floor to get to the sitting position.

Assisted sitting transfer from chair to bed:



7. Place the chair in a relevant position next to the bed. A Sliding board can be useful.



8. The user leans forward using the Master Bed Grip and can use the bed for support during the transfer.

User Manual - Master Bed Grip

Technical Specifications

Materials:

Polyester,

Handles: Hard - Polycarbonat, Soft - Ethylene Vinyl Acetate Copolymer.

Article no. - Product Name	Size:	Number of handles	Max. User Weight
MC 003-3510 Master Bed Grip - Soft	L 180 x W 9 cm	8	160 kg
MC 003-3520 Master Bed Grip - Hard	L 180 x W 15 cm	8	160 kg
MC 003-3530 Master Bed Grip - Single	L 150 x W 5 cm	1	160 kg
MC 001-1313 BASIC Grip	L 140 x W 8 cm	5	180 kg

WARNINGS



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

For more information, videos Quick Guide etc., visit www.master-care.dk



User Weight:

Recommended max. user weight: Master Bed Grip 160 kg and BASIC Grip 180 kg.



Washing:

Master Bed Grip Single, Soft and Hard can be washed at 60 °C. BASIC Grip can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with flaws or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

Find us on:

