

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

35\_20230109\_UK

## One Way



MC 001-1266



MC 001-1272



MC 001-1276



MC 001-1277



MC 001-1291



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
○ MC 001-1266	134575	One Way Open - S	5744001974278
○ MC 001-1272	134576	One Way Open - M	5744001974285
○ MC 001-1276	134577	One Way Open - L	5744001974292
○ MC 001-1277	81135	One Way Open - Pull Strap	5744001970317
○ MC 001-1291	134574	One Way Tube – M	5744001974247

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 19

The Original Nordic  
Human Handling Concept



Developed &  
Manufactured by:



# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## **MDR (EU) 2017/745:**

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

The product is used by users with a tendency to slide forward in the wheelchair. The product can be used to slide the user backwards to a better position. The design of the product prevents the user from sliding forward.

The product should be replaced after repositioning two- three times.

## **Correct use of the product:**

The product can be placed on the wheelchair seat or the car seat. For users who need support moving backwards on the seat.

## **Warning:**

Use with caution for users with low trunk stability.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used



# User Manual - One Way

## Table of Contents

Placing One Way Open	.....	page 2
Placing and using One Way Tube	.....	page 2
Transferring backwards active user – open back – One way Open	.....	page 2
Transferring backwards with a less active user – open back – One Way Open	.....	page 3
Transferring backwards – close back – One Way Open	.....	page 3
Repositioning the One Way Open	.....	page 3
Technical Specifications	.....	page 4



# User Manual - One Way

## Placing One Way Open:



1. Place the One Way Open with the velour downwards and the three handles facing upwards.



2. Fold the One Way and have the label: 'This side up ...' towards the back rest. Fold and leave a 10 cm gap between the top part and the back rest.



3. The handles are facing backwards and away from the seat. The user sits on top of the One Way Open.

## Placing and using One Way Tube:



4. One Way Tube is placed with the label: 'This side up ...' and is placed 10 cm from the back rest.



5. Place the user on top of the One Way Tube.



6. With the feet well supported and leaning forward, the user can slide backwards on top of the seat using the slippery inner side of the Tube.

## Transferring backwards active user – open back – One way Open:



7. Make sure the user's knees are/thighs are not heavily supported by the seat using e.g., the footrests or a MC PMA-8090 Foot Master under the user's feet.



8. The user leans forward and pushed with the feet and simultaneously pushes with the hands on the arm rests.



9. The helper supports the transfer by pulling the corner or the middle handle e.g., with a Pull Strap or supporting with the side handles.

# User Manual - One Way

## Transferring backwards with a less active user – open back – One Way Open:



10. One Way - Pull Strap is mounted onto the middle strap/handle of the three on the One Way Open.



11. With the user's feet fully supported and the upper body leaning forward, the helper pulls the One Way – Pull Strap. The user pushed with the feet to support the transfer. If it is a big user two One Way – Pull Straps can be mounted into the two side straps/handles of the three.

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


12. Alternatively: The helper can step down into the One Way – Pull Strap to support the backwards transferring.

## Transferring backwards – close back – One Way Open:



13. If it is impossible to pull the straps through a gap between the seat and the back rest, the user can be transferred backwards using the handles on the side of the One Way Open.



14. Alternatively: The helper supports/pushes the user's knees, while the user leans forward and pushed with the feet.

## Repositioning the One Way Open:



15. After repositioning the user 2-3 times it can be necessary to reposition the One Way Open, to achieve the optimal function of the product.

# User Manual - One Way

## Technical Specifications

### Materials:

Polyester, Mohair, Nylon

### ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name	Size:	Antal håndtag:	Max. User Weight
MC 001-1266 One Way Open - S	L 105 x W 40 cm	5	200 kg
MC 001-1272 One Way Open - M	L 115 x W 50 cm	5	200 kg
MC 001-1276 One Way Open - L	L 125 x W 60 cm	5	200 kg
MC 001-1277 One Way Open - Pull Strap	L 49 x W 2,5 cm	1	200 kg
MC 001-1291 One Way Tube - M	L 50 x W 50 cm	0	200 kg

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



### User Weight:

Recommended max. user weight: 160 - 200 kg. See table for more information.



### Washing:

This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)

Find us on:

