

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.


Choose quality, save money, and protect the environment.

(UK) User Manual

38_20241003_UK

Master Comfort
Positioning Cushions

Top Class Positioning Cushion

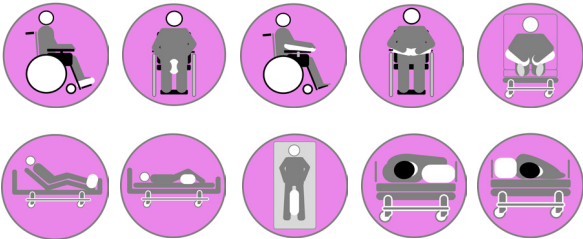

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-8015	71193	Master Comfort Positioning Cushion – XS	5744001971437
MC 001-8019	50150	Master Comfort Positioning Cushion – S	5744001971468
MC 001-8020	45939	Master Comfort Positioning Cushion – M	5744001971475
MC 001-8021	50151	Master Comfort Positioning Cushion – L	5744001971482
MC 001-8014	137066	Master Comfort Positioning Cushion Cover - XS	5744001974520
MC 001-8022	137076	Master Comfort Positioning Cushion Cover – S	5744001974537
MC 001-8023	137080	Master Comfort Positioning Cushion Cover – M	5744001974544
MC 001-8024	137081	Master Comfort Positioning Cushion Cover – L	5744001974551

Regulation MDR 2017/745 for medical devices class 1



ISO 09 07 06 06

The Original Nordic
Human Handling Concept



Developed &
Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The intended use is to support and stabilize a user in a correct position whilst minimizing the pressure and shear forces on the tissue. Thereby minimizing the risk of tissue injuries and treat injuries in a coordinated medical and care intervention for users with reduced physical or cognitive function.

Correct use of the product:

The Master Comfort products can be placed between the user and the area of support and thereby reduce the pressure on the skin. Alternatively, the products can be used to support the sitting position or when lying on the side to prevent falling forward or backwards. The product is suitable for use in beds with adjustable lying surfaces.

Contraindication/Warning:

If sufficient pressure relief cannot be achieved for the user in need of care despite the correct use of the Master Comfort Positioning Cushion, additional or other options for appropriate relief can be considered. The use of wedges and pillows and other repositioning aids can

only supplement the need for repositioning the person and daily skin assessment.

The top edge of the side rails must be at least 22 cm above the unloaded top edge of the mattress.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of the products is 1 - 5 years depending on the use, user's load and how often it is washed and dried.

Products which no longer provide sufficient support, with defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Comfort Positioning Cushions

Table of Contents

Supporting upper leg when side lying (Passive user, one helper):	page 2
Avoid sliding downwards in bed in sitting position (Passive user, one helper):	page 2
Supporting the arm/-s in sitting position:	page 3
Minimizing tissue injuries:	page 3
Technical Specifications	page 4



User Manual - Master Comfort Positioning Cushions

Supporting upper leg when side lying (Passive user, one helper):



1. The user is lying on the back on a Master Turner or a Light Turner. Place the chosen Cushion next to the legs.



2. Fold a Nylon Master Open and place it with the fold between the user's legs. A Positioning Wedge - S MC 001-8035 can be used to support the stomach.



3. Turn the user to side lying position using a ceiling hoist and the Master or Light Turner. The Turn Sheet turns the legs automatically.



4. Use the folded Nylon Master Open to slide the upper leg into position on top of the Positioning Cushion. It can be an advantage to use a Leg Strap or a Master Grip to "extend" the arms reach if the upper leg is positioned far away.



5. Remove the Nylon Master Open by pulling the lower part of the sliding material. If it gets stuck pull from the opposite lower corner. It is a good idea to stabilize the top leg while pulling the Nylon Master to keep it in position.



6. Make sure the leg is supported and stabilized comfortably. A sheet can be placed on top to avoid direct contact between Cushion and bare skin.

Avoid sliding downwards in bed in sitting position (Passive user, one helper):



7. The user is lying on a Master Turner. Make sure the user's pelvis is placed in the correct position in the bed. Preferably use the two Positioning Wedges - XS to maintain this position. Fully elevate the knee-bend of the bed.



8. Place the chosen Positioning Cushion under the user's feet. Elevate the head-end of the bed. Because of the unique sliding abilities of the Master Turner the user can move/be moved around but feels no shear or tear of the skin. The Positioning Cushion helps to prevent the user from sliding down. The feet should be supported comfortably and not restrained.

User Manual - Master Comfort Positioning Cushions

Supporting the arm/-s in sitting position:



9. Choose the appropriate Positioning Cushion. Lift the user's arm and place the Cushion. Position the arm comfortably on top of the Positioning Cushion.



10. Often it is more comfortable to have both arms supported at the same time. This way the body is stimulated equally on both sides while being relaxed and stable.



11. All Master Comfort products can be used on anti-decubitus mattresses (foam, hybrid, alternating).

As an accessory a cotton cover is available.

Minimizing tissue injuries:



12. To avoid tissue injuries our Positioning Cushion contains small Original Krøyer Balls. They are easy to mold out and shape to support the body in the perfect way. Hence the weight on the tissue is distributed on a bigger area and thereby reducing the risk of tissue injuries. On top of this the balls are flame retardant and washable.

User Manual - Master Comfort Positioning Cushions

Technical Specifications

ISO certifications:

INCO COVER:

DS/EN ISO 12952-1:2010

Textiles – Assessment of the ignitability of bedding items – Part 1: Ignition source: smouldering cigarette.

DS/EN ISO 12952-2:2010

Textiles – Assessment of the ignitability of bedding items – Part 2: Ignition source: match-flame equivalent.

Article no. - Product Name	Size:	Max/min height:	Max. user weight (SWL):	Contents:	Package weight:
MC 001-8015 Master Comfort Positioning Cushion – XS	L 50 x W 20 x H 10 cm	Not relevant	Not relevant	72% Polystyrene, 14% Polyester, 10% Polyurethane, 4% Cotton	0,55 kg
MC 001-8019 Master Comfort Positioning Cushion – S	L 70 x W 40 x H 20 cm	Not relevant	Not relevant	84% Polystyrene, 9% Polyester, 6% Polyurethane, 2% Cotton	1,62 kg
MC 001-8020 Master Comfort Positioning Cushion – M	L 70 x W 40 x H 30 cm	Not relevant	Not relevant	87% Polystyrene, 7% Polyester, 5% Polyurethane, 1% Cotton	2,34 kg
MC 001-8021 Master Comfort Positioning Cushion – L	L 70 x W 40 x H 40 cm	Not relevant	Not relevant	88% Polystyrene, 7% Polyester, 5% Polyurethane, 1% Cotton	3,40 kg
MC 001-8014 Master Comfort Positioning Cushion Cover – XS	L 50 x W 20 x H 10 cm	Not relevant	Not relevant	61% Cotton, 32% Polyester, 5% Spandex, 2% Polyamide	0,11 kg
MC 001-8022 Master Comfort Positioning Cushion Cover – S	L 70 x W 40 x H 20 cm	Not relevant	Not relevant	61% Cotton, 32% Polyester, 5% Spandex, 2% Polyamide	0,22 kg
MC 001-8023 Master Comfort Positioning Cushion Cover – M	L 70 x W 40 x H 30 cm	Not relevant	Not relevant	61% Cotton, 32% Polyester, 5% Spandex, 2% Polyamide	0,26 kg
MC 001-8024 Master Comfort Positioning Cushion Cover – L	L 70 x W 40 x H 40 cm	Not relevant	Not relevant	61% Cotton, 32% Polyester, 5% Spandex, 2% Polyamide	0,32 kg

WARNINGS



Never leave the product on the floor.



The use of pillows and wedges can't replace the need of frequent repositioning and daily skin assessment.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



Washing:

This product can be washed at max 85 °C but disinfection with alcohol is preferable to extend the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may be tumble dried gently at max 40 °C but air drying is preferable to extend the durability.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make frequent visual inspections to check that the incontinence surface and all seams are intact before use.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos
Quick Guide etc., visit
www.master-care.dk

