

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

40_20250114_UK

Super Glide **ECO** Open



REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 002-1269	117518	Super Glide ECO - Open 1x100 pcs	05744001971925
<input type="radio"/> MC 002-1272	125008	Super Glide ECO - Open 50 pcs	05744001971956

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 20

The Original Nordic
Human Handling Concept



SCAN ME

Developed & Manufactured by:



User Manual - Super Glide ECO Open

Table of Contents

Important information before using the product(s):	page 2
Turning to sidelying using Super Glide ECO:	page 3
Higher up in bed – Dependant user, one helper:	page 3
Sideways transfer with Super Glide ECO – Two helpers:	page 3
Transfer between bed and chair:	page 4
Turning with Super Glide ECO, Multi Sheet, and a hoist:	page 4
Technical Specifications	page 5
Materials:	page 5



Thank you for choosing a product from master care A/S.

Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product can be used as a Single Patient Use sliding material for users who need friction reduced during transfers. Recyclable plastic.

Correct use of the product:

Super Glide ECO Open can be used during transfers from e.g. wheelchair to bed, on operating tables, in bed and during other transfers where the user needs help due to high friction from the surface.

Indication:

The product is used as a sliding aid for users who require reduced friction, for example during seated transfers due to reduced physical or cognitive function. It can advantageously be used instead of nylon sliding sheets in situations where hygiene is challenging.

Contraindications:

There are no known contraindications for using this product.

Warning:

As the material is made of plastic and is therefore not breathable, The product should not be left under the user for a long time, as this can cause damage to the tissue. Always use the product in double layers to achieve optimal friction reduction.

Never leave the user on the product when lying on the edge of the bed since this poses a risk of the user falling to the floor. Do not leave the product on the floor due to risk of falling.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Date of manufacture and Life span:

The product's production date can be found on the product label under 'LOT' and is displayed as YYMMDD - e.g. 251020, which refers to the product being produced on the 20th of September 2025.

The product is for Single Patient Use and can be used several times for the same patient if the hygienic requirements are fulfilled.

Cleaning with alcohol is allowed. Storage in original packaging at medium temperature is recommended.

Inspection:

Products with insufficient friction reduction or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Super Glide ECO Open

Turning to sidelying using Super Glide ECO:



1. Bend the user's legs (Super Glide ECO can be used here) and turn user slightly to one side. Place the Super Glide ECO with the fold close to the user's spine or under the draw sheet at the side you wish to turn the user towards.



2. Turn the user to supine. Place the user's arms on the stomach or the pillow. Turn the user to the opposite side and slide the user to the perfect position using the Super Glide ECO.



3. Remove the Super Glide ECO by pulling the lower part from one corner. If you feel resistance pull the opposite lower corner. You can also pull the fold in between the sides and let the Super Glide ECO slide on itself.

Higher up in bed – Dependant user, one helper:



4. Place the hand in the Super Glide ECO fold. From the top slide the Super Glide under the user's shoulders - begin on one side and continue the same way on the other side.



5. Elevate the bed end to use the gravity. Bend the user's legs, fixate the feet and ask the user to push the upper body upwards. Alternatively place the Super Glide ECO under the sheet and use this as a draw sheet.



6. Super Glide ECO glides under the user's shoulders and the user can easily slide upwards in bed with reduced friction. Remove the Super Glide ECO by pulling the lower part from one corner. If you feel resistance pull the opposite lower corner.

Sideways transfer with Super Glide ECO – Two helpers:



7. Turn the user to side lying. Helper A places the folded Super Glide ECO with the fold close to the spine. To achieve comfort and perfect working conditions a draw sheet could also be placed over the Super Glide ECO.



8. Return the user to supine lying. Helper B pushes the user to the side and Helper A pulls the draw sheet. The Super Glide ECO reduces friction enormously during the transfer.



9. Remove the Super Glide ECO by pulling the lower part of the Super Glide ECO. You can also pull the fold in between the sides and let the ECO slide on itself.

User Manual - Super Glide ECO Open

Transfer between bed and chair:



10. Ask the user to lean to the opposite side. Place the Super Glide ECO – Open under the user's lifted buttocks.



11. Place the Master Glide under the folded Super Glide ECO, to build the bridge and make the transfer easier. Remember to make use of the user's resources during the transfer.



12. Make the transfer. Remove the sliding board. Pull the lower part of the Super Glide ECO to get it out.

Turning with Super Glide ECO, Multi Sheet, and a hoist:



13. Mount the Multi Sheet on the hoist. Turn the user slightly to one side. Place the folded Super Glide ECO Open close to the spine. Make sure to place it under the pressure points of the head, shoulder, and buttocks.



14. Mount the Multi Sheet on the opposite side to the hoist. Lift up, slide, and place the user in a perfect position using the Super Glide ECO. Thus reducing friction between the sheet and the Multi Sheet.



15. Grab a corner of the lower part of the Super Glide ECO and pull it out. If resistance is felt grab the opposite lower corner instead and pull here.

User Manual - Super Glide ECO Open

Technical Specifications

Article no. - Product Name:	Size:	SWL:	Product weight:
MC 002-1269 Super Glide ECO - Open 1x100 pcs	L 100 x W 62 (124) cm	Not relevant	5,40 kg
MC 002-1272 Super Glide ECO - Open 1x50 pcs	L 100 x W 62 (124) cm	Not relevant	2,72 kg

Materials:

MDPE - recyclable as LDPE.

Warnings



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product. Can be recycled as LDPE or MDPE.



Maintenance:

The product is for Single Patient Use and can be used several times for the same patient if the hygienic requirements are fulfilled. Products with insufficient friction reduction or other deficiencies must be discarded.

Storage:

Storage in original packaging at medium temperature is recommended.

For more information, videos Quick Guide etc., visit www.master-care.dk

Find us on:



master-care.dk