

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

79\_20260326\_UK

## Inco Sheet



MC 001-1307



MC 001-1801



MC 001-1811



MC 001-1820



Regulation MDR 2017/745 for medical devices class 1



ISO 09 30 45 01

The Original Nordic  
Human Handling Concept



SCAN ME

| REF (Article no.) | HMI no. | Name - Size                 | UDI-DI/GTIN    |
|-------------------|---------|-----------------------------|----------------|
| ○ MC 001-1307     | 89657   | BASIC inco - S              | 05744001973158 |
| ○ MC 001-1801     | 50803   | Inco Sheet - S              | 05744001970690 |
| ○ MC 001-1811     | 50804   | Inco Sheet - M              | 05744001970713 |
| ○ MC 001-1820     | 128704  | Inco Sheet Terry Cloth - XS | 05744001970720 |

Developed & Manufactured by:



master-care.dk

# Thank you for choosing a product from master care A/S.

## Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## The intended use of the product/-s:

The product has an incontinence membrane which prevents liquid from penetrating the mattress. The filling is breathable and absorbing which minimizes the risk of overheating and tissue damage.

## Correct use of the product:

Place the product on top of the mattress in the area where an incontinence function is needed. Place the user directly on top of the product or with a sheet between if needed.

## Indication:

The product is used for users who are incontinent and where protection of, for example, a mattress or repositioning system is necessary.

## Contraindications:

No contraindications regarding use of the product is known.

## Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

## Date of manufacture and Life span:

The product's production date can be found on the product label under 'LOT' and is displayed as YYMMDD - e.g. 251020, which refers to the product being produced on the 20th of September 2025.

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

## Inspection:

Products that no longer provide sufficient liquid absorption or that have a slippery bottom surface must be discarded. The products should be inspected regularly, and if broken seams or damaged surfaces are detected, the product must be discarded.

## Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

# User Manual - Inco Sheet

## Variants:



1. Inco Sheet - S  
Jersey top layer and soft and absorbent filling on top of a high quality incontinence membrane.



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
2. Inco Sheet - M  
Jersey top layer and soft and absorbent filling on top of a high quality incontinence membrane.




3. Inco Sheet Terry Cloth - XS  
Terry cloth top layer and soft and absorbent filling on top of a high quality incontinence membrane.




4. BASIC Inco - S  
Nice polyester/cotton surface and filling on top of a high quality incontinence membrane.



## How to mount the Inco Sheet:

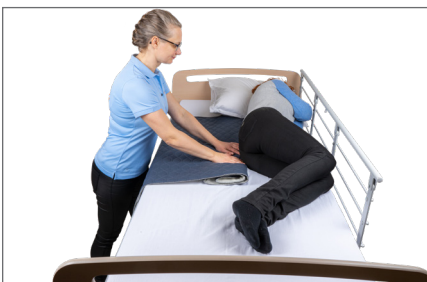

5. Turn the user to the side. Roll/ fold the Inco Sheet half way and place it underneath the user in the needed area.

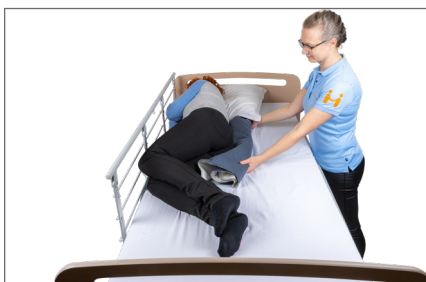

6. Turn the user to the opposite side and roll out the Inco Sheet.



7. Turn the user to supine. The Inco Sheet is now placed correctly underneath the user.

# User Manual - Inco Sheet

## Technical Specifications

### ISO-certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

| REF - Product Name:                     | Size:            | Max. user weight: | Product contents:                           | Product weight: |
|---|------------------|-------------------|---|-----------------|
| MC 001-1307 Basic Inco - S              | L 90 x W 140 cm  | 200 kg            | 9% Cotton, 78% Polyester, 13% Polyurethane. | 0,82 kg         |
| MC 001-1308 Basic Inco - M              | L 120 x W 140 cm | 200 kg            | 9% Cotton, 78% Polyester, 13% Polyurethane. | 1,04 kg         |
| MC 001-1801 Inco Sheet - S              | L 90 x W 140 cm  | 200 kg            | 5% Cotton, 81% Polyester, 14% Polyurethane. | 0,96 kg         |
| MC 001-1811 Inco Sheet - M              | L 120 x W 140 cm | 200 kg            | 4% Cotton, 82% Polyester, 14% Polyurethane. | 1,14 kg         |
| MC 001-1820 Inco Sheet Terry Cloth - XS | L 90 x W 69 cm   | 200 kg            | 7% Cotton, 81% Polyester, 12% Polyurethan.  | 0,38 kg         |

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



### User Weight - SWL:

Recommended max. user weight: 200 kg



### Washing:

This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric and seams. Products with deficiencies must be discarded.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)

Find us on:

