

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual
Master Glide Sif



01_20241001_UK



MC 001-7060



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-7060	128744	Master Glide Sif (2 pcs)	5744001974148

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 17

The Original Nordic
Human Handling Concept



SCAN ME

Developed &
Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

- 1) The product is placed in pairs underneath the user's thighs to reduce friction when the user moves or is moved backwards on the seat.
- 2) Place the pair under the user's thighs. Slide the leg strap of the sling between the plates to minimize the risk of shear injuries.

Correct use of the product:

The product is intended for users who have problems with moving backward on the seat or where mounting of the sling may cause risk of shear injuries.

If the user isn't able to move backwards in the seat, place 1-2 plates under the thigh(-s). If possible, have the user lean away from the side where the plate is mounted. Users who are active in one side of the body, placement of a Sif plate in the opposite side may be enough. Otherwise place a plate underneath both thighs.

Contraindication/Warning:

If the user does not have sitting balance (truncus stability), another

transfer method should be used. Never move over distances that are longer than the maximum distances stated on the product. If the product is used for users with high risk of shear injuries a sliding sheet is recommended between the user and the product.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with normal wear and tear is 2-5 years. The products life span depends on the use and the user's load.

Products with insufficient friction reduction, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user. Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Glide Sif

Move backwards in the wheelchair (1-2 helpers):



1. Master Glide Sif is the perfect aid to help a user backwards in a wheelchair. To erase the pressure under the thighs, place the feet on the chair's footplates or e.g., on a Master Step On - PMA-8091.



2. If possible, ask the user to lean to the opposite side. The helper places one Master Glide Sif under each thigh by pushing the tip of the plate into the cushion and then twisting it under the thigh.



3. The helper places them self in front of the user and helps the user lean forward to free the back from the backseat of the wheelchair.



4. The helper places the hands or the knee next to the user's knee and the user can easily be pushed backwards into the correct position. The more active user can push them self backwards.



5. Remove the Master Glide Sif on both sides by rotating it and pulling it out. The user can lean backwards again.

Placing a sling sheet using Master Glide Sif:



6. Place the sling sheet behind the user's back - Master Glide Thor can be used. Place the two Master Glide Sif together under the user's thigh by pushing the tip of the boards into the surface.



7. Insert the leg strap between the two Master Glide Sif and pull it into place under the user's thigh.



8. Remove the Master Glide Sif one at a time. Repeat on the opposite side.

User Manual - Master Glide

Technical Specifications

Certifications:

REACH
DODD
BSE/TSE

Materials: Polyethylene

Article no. - Product Name:	Size:	Max. user weight (SWL):	Max gap between support points:	Package weight:
MC 001-7060 Master Glide Sif	L 45 x W 20 x H 0,3 cm	200 kg	0 cm	0,42 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to scratches and tears to the surface. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos
Quick Guide etc., visit
www.master-care.dk

