

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

02_20251215_UK

Sliding Material Quiet



-
-
-
-

REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-1025	129860	Sliding Material Quiet - Roll 100 m	05744001973875
MC 001-1026	129862	Sliding Material Quiet - Roll 10 m	05744001973882
MC 001-1030	130215	Sliding Material Quiet - Roll 20 m	05744001971567
MC 001-1031	129861	Sliding Material Quiet - Roll 50 m	05744001973905

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 22

The Original Nordic
Human Handling Concept



SCAN ME

Developed & Manufactured by:



User Manual - Sliding material Quiet

Table of Contents

Important information before using the product(s):	page 2
Help when putting on shoes (Active/ passive user):	page 3
How to put on pants with Sliding Material Quiet when lying (Less active user):	page 3
Putting on a shirt lying down (Less active user):	page 4
Putting on compression socks (Less activer user):	page 4
Technical Specifikations	page 5



Thank you for choosing a product from master care A/S.

Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is to be used bilayered to reduce friction for users with decreased function during e.g. transfer, dressing or training. The product can be used to reduce the risk of shear injuries.

Correct use of the product:

Place the Nylon underneath the points/support areas where friction is to be reduced. Always use the product double layered to achieve maximum functionality and reduction of friction. Always remove the product under the user after use to avoid fall injuries and risk of skin overheating.

Indication:

The product can be used during transfers for users who require increased friction reduction against the supporting surface, for example due to physical and/or cognitive impairments or an increased risk of shear injuries.

The product can also be used for users for whom dressing is challenging, for example due to reduced muscle tone, spasticity, reduced mobility, or similar conditions.

Contraindication:

Use with caution for users with open wounds or scabs. If possible, it is recommended to cover the wound before transfers or dressing.

Warning:

Always do a risk assessment before using the product especially if used e.g., when sitting on the edge of the bed since this poses a risk of the user sliding/falling on to the floor. Always make sure the user

has sufficient support underneath their feet when sitting if a nylon piece is placed underneath their bottom. Never leave the product on the floor since this poses a risk of falling.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Date of manufacture and Life span:

The product's production date can be found on the product label under 'LOT' and is displayed as YYMMDD - e.g. 251020, which refers to the product being produced on the 20th of September 2025.

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Inspection:

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Sliding material Quiet

Help when putting on shoes (Active/ passive user):



1. Use a piece of the Sliding Material Quiet. Fold it and place it inside and over the heel of the shoe with the fold in the front of the shoe.



2. Place the foot on top of the folded Sliding Material Quiet and slide the foot into the shoe. Make sure to use the user's resources when sliding into the shoe.




3. When the foot is inside the shoe pull the corner of the lower part of the Sliding Material Quiet out. This way it will easily slide out. If the Sliding Material Quiet feels stuck the pull the opposite lower corner.

How to put on pants with Sliding Material Quiet when lying (Less active user):



4. Place the first piece of Sliding Material Quiet. Zig-zag-fold the lower half of the Sliding Material Quiet – keep the upper part straight. Place it from the hip and down to the foot.

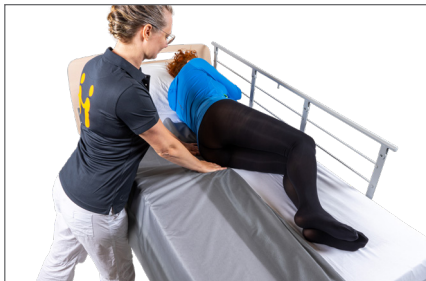

5. Turn the user to the side and place the Sliding Material with the fold under the user. Turn the user to the opposite side and straighten out the Sliding Material. Piece number one is in place.



6. Place two new pieces of Sliding Material Quiet. One under each leg from the buttocks and down with the fold between the legs. A plastic bag can be put around each foot if the Sliding Material is not long enough.



7. Pull up the pants between the two layers of Sliding Material. Pull the pants over the feet and up to the knee.



8. When the pants reach the knee height pull out the sliding Material from around the feet and ankles. Pull the lower/outer Sliding Material towards the hips.



9. Pull by turns pants and Sliding Material. When the pants are pulled up please check for folds on the back before removing all three pieces of Sliding Material from under the user.

User Manual - Sliding material Quiet

Putting on a shirt lying down (Less active user):



10. Place two large pieces of Sliding Material Quiet under the user from the buttocks to the back of the head. A plastic bag can also be placed over each hand.



11. Fold the upper part of the Sliding Material Quiet over the shoulders. Put the shirt on the first arm and up to the shoulder. Gently pull out the plastic bag, if one is used.



12. Pull the shirt over the shoulder and pull it "too far over and down" on the back towards the buttocks on the opposite side. Push the shirt to the other side between the two layers of Sliding Material Quiet. Use the cavity under the loin.



13. Turn the user to side lying position.



14. Pull out the shirt in a slightly downwards direction.



15. Bend the elbow on the opposite arm and put on the sleeve. Remove plastic bag, if one is used. Adjust the shirt before removing the Sliding Material Quiet. Make sure to use the cavities under the body when removing it.

Putting on compression socks (Less activer user):



16. Fold at piece of Sliding Material Quiet with the fold towards the toes. Place it under the user's leg. Fold it around the user's toes – a cloth-peg can be used to keep it in place, if it is at peep-toe sock. Turn the sock inside out from the heel and up and place the sock from the toes to the heel.



17. Put on a pair of rubber coated gloves (to induce the friction). Push the compression sock over the heel and up the leg, with fingers straight. Pull the outer layer of the Sliding Material Quiet so it is removed around the toes and the heel. Pull up the top of the sock over the heel and then push the sock into place using the Sliding Material.



18. Carefully push or pull the sock all the way up with the fingers straight and adjust. Remove the Sliding Material Quiet by pulling the outer layer.

User Manual - Sliding Material Quiet

Technical Specifications

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	SWL:	Product weight:
MC 001-1025 Sliding material Quiet - Roll 100 m	L 100 x W 1,5 m	Not relevant	11,50 kg
MC 001-1026 Sliding material Quiet - Roll 10 m	L 10 x W 1,5 m	Not relevant	1,32 kg
MC 001-1030 Sliding material Quiet - Roll 20 m	L 20 x W 1,5 m	Not relevant	2,08 kg
MC 001-1031 Sliding material Quiet - Roll 50 m	L 50 x W 1,5 m	Not relevant	5,66 kg

Materials:

Polyamide, silicone

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight/ SWL (Safe Working Load):
No maximum weight limit.



Washing:

This product can be washed in a washing machine at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:
Do not iron.



Dry cleaning:
Do not dry clean.



Bleach:
Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

