

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

Quick Guide

(DK) **Light Cover** - til Light Turner

(UK) **Light Cover** - for Light Turner

24_20220524 DK/UK



The Original One-Layer System



001-2654



001-2655



001-2656

Vare nr./Item no.	HMI nr./no.	Navn/Name - Størrelse/Size	UDI-DI
<input type="radio"/> MC 001-2654	131094	Light Cover - S	5744001974124
<input type="radio"/> MC 001-2655	125009	Light Cover - M	5744001971208
<input type="radio"/> MC 001-2656	125026	Light Cover - L	5744001971215

Eftersyn:

Produktet skal kontrolleres jævnlig og gerne inden hver brug.

Kontroller, at der ikke er skader på stof, stropper, sømme mm.

Har produktet høj friktion eller på anden vis er i stykker, skal produktet kasseres.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.




(DK) Efterlad aldrig bruger på produktet på sengekanten.

(UK) Never leave the user on the product when lying on the edge of the bed.



(DK) Efterlad aldrig produktet liggende på gulvet.

(UK) Never leave the product on the floor.

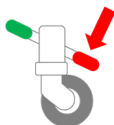

(DK) Sørg for, at alle relevante bremsere er låst, inden forflytning foretages.

(UK) Make sure that all relevant brakes are activated before transferring the user.



(DK) Sørg altid for, at brugeren ikke hænger fast i stropperne.

(UK) Always make sure that the user is not caught in the straps.



(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1
(UK) Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic
Human Handling Concept



(DK) For mere information, brugervejledning, videoer mm., se
(UK) For more information, User Manual, videos etc. visit

www.master-care.dk



Developed &
Manufactured by:



master-care.dk

Light Cover


The **Original** One-Layer System

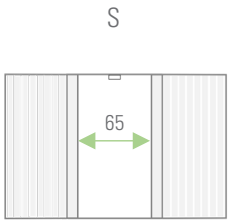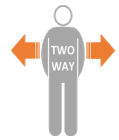

200 kg




97 x 200

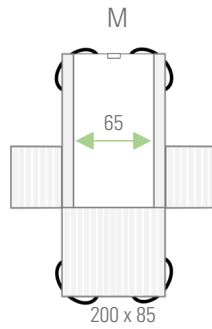

200 kg



200 x 85

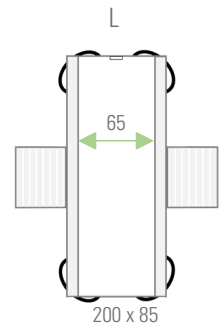


200 kg







200 x 85

(DK) Monter Light Cover / (UK) Mount Light Cover

(DK) Placer vinger / (UK) Place the wings

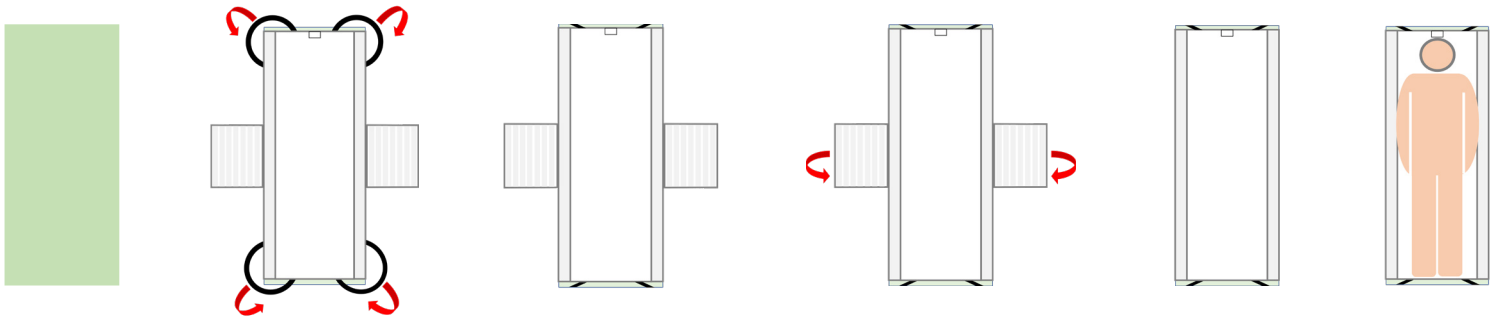

(DK) Anvendelsesmuligheder / (UK) Applications

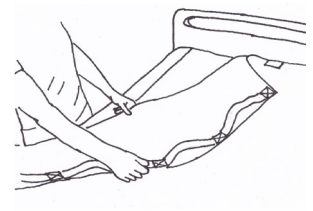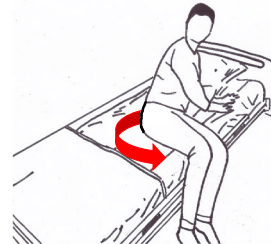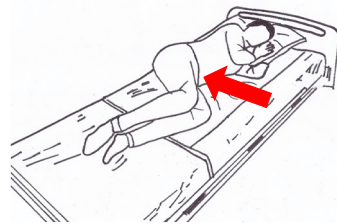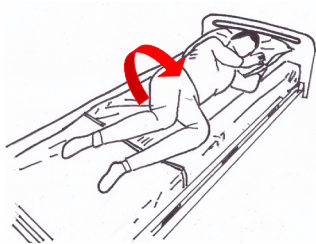

(DK) Placer Light Turner oven på Light Cover for at få 4-vejs glid.

(UK) Place Light Turner on top of Light Cover to gain 4-way glide.



(DK) Skånsom vask og tørretumbling forlænger holdbarheden væsentligt.
(UK) Gentle washing and dry tumbling significantly prolongs the durability.

(DK) Anvend aldrig skyllemiddel, da dette forringer glideegenskaberne væsentligt.
(UK) Never use fabric softener as this will reduce the sliding properties significantly.

(DK) Skånsom stryging forbedrer glideevnen.
(UK) Gentle ironing improves sliding abilities.



master-care.dk

Find us on



master care A/S, Sofienlystvej 3, DK 8340 Malling, Danmark
Tel. +45 86 93 85 85, E-mail: info@master-care.dk

(DK) Vi tager forbehold for evt. trykfejl og mangler. (UK) Reservations are made for printing errors.