MADE IN EUROPE

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Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the costumer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The intended use for the product is to reduce friction between the user and the mattress to make it easier for the helper and gentler for the user whilst turning, moving up/downwards, and moving to sitting position on the edge of the bed. This product should always be used in combination with a Mattress Cover from master care a/s.

Proper use of the product:

The combination of the Master Turner and the Mattress Cover from master care A/S constitutes a turning system, which can be used for users with a decrease in function where transfers in bed and between beds can be difficult.

Warning:

Always make sure not to position the user too close to the edge of the mattress if sitting on top of the Mattress Cover since this poses a risk of sliding off/down. If the user is transferred to a sitting position on the edge of the bed, always make sure the knees have a 90-degree angle, the feet can reach the floor and to have enough non-skid under the user's thighs.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product. master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

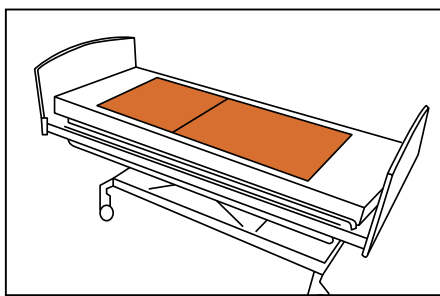
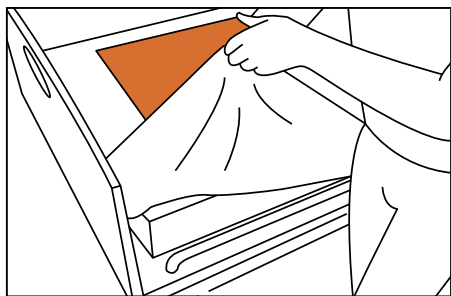
Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

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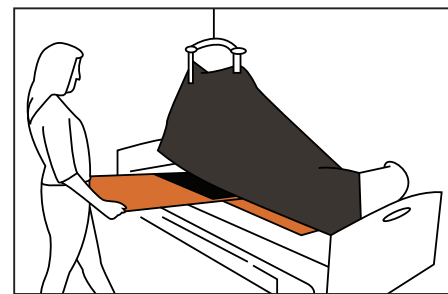
UK

Place the cover on the mattress. See illustration
This also applies for BASIC ST Cover

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UK

Attach the lift to the turn sheet on the side where the sliding flap opening is located. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Insert the sliding flap under the user at the points of least friction (ankle, hollow of the knee, lower back or neck). Slide the straps in so they are pointing forward

Mount the lift on the opposite side, and raise the side up. Pull out the sliding flap

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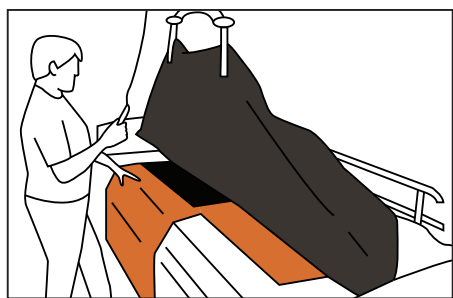
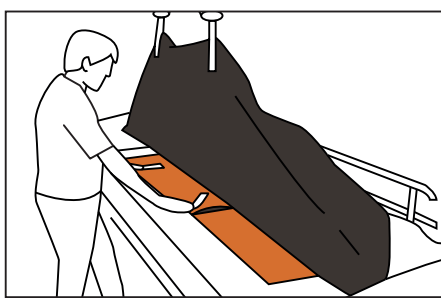
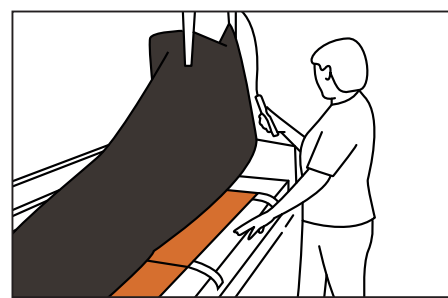
UK Attach the lift to the turn sheet on the opposite side of the sliding flaps opening. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Double up the sliding flap and slide it under the user using both hands. Smooth out the sliding flap so it is spread evenly across the cover's non-slip surface. Finish by maneuvering the straps under the user

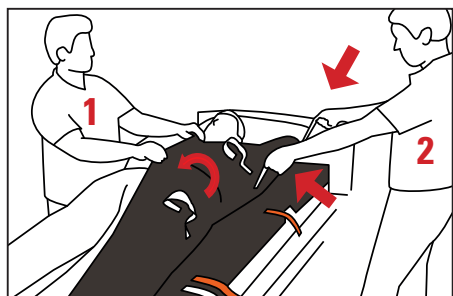
Then, attach the lift on the opposite side. Raise the lift until the user's shoulder and bottom are not touching the surface beneath. The sliding flap straps should now be visible and can be fastened

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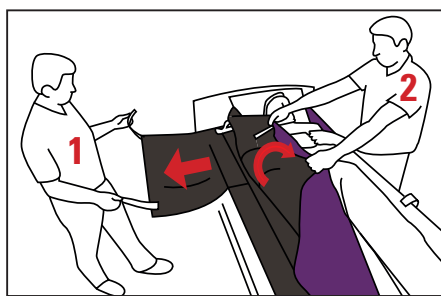
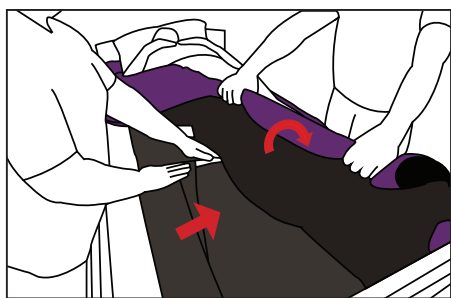
UK Person 1 helps position the user onto their side. See illustration. The sliding flap straps should now be visible under the user

Person 2 loosens the sliding flap's two straps and folds them. Fold the sliding flap under the user with straps at the front

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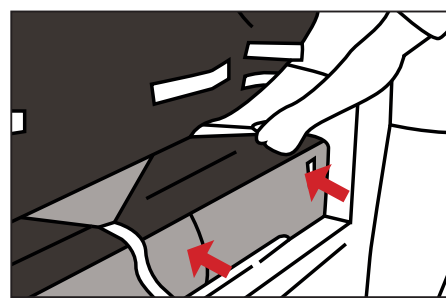
Stropper låses fast på BASIC Cover

UK Person 1 holds the turn sheet and helps position the user on their side. Double up the sliding flap and fold it under the user with straps at the front

Person 2 holds the turn sheet up on the opposite side. Person 1 takes hold of the straps and pulls the sliding flap into position under the user

Fasten the straps onto the BASIC Cover

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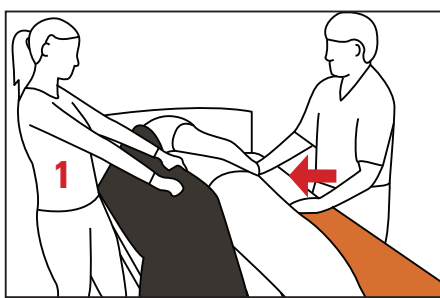
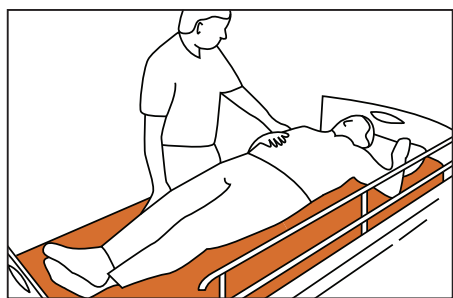
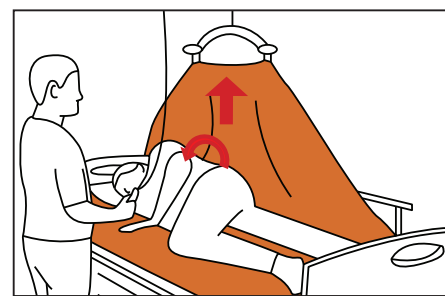
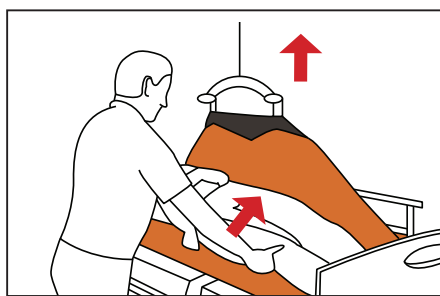
UK Cross the user's legs. Lead the user's hand to touch their opposite shoulder. Slide the pillow and shoulder toward the side that you need to turn the user

Person 1 pulls the turn sheet and helps position the user on their side

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UK Cross the user's legs and pull their pillow and shoulder toward the side that you need to turn the user. Attach the turn sheet's straps to one side of the lift

Lift the user slightly above the surface beneath. Person 1 then positions the user toward the bed guard. Use the Master Comfort Stomach Wedge for additional relief and support during turn

Raise the lift. Turn the user over onto a lateral position

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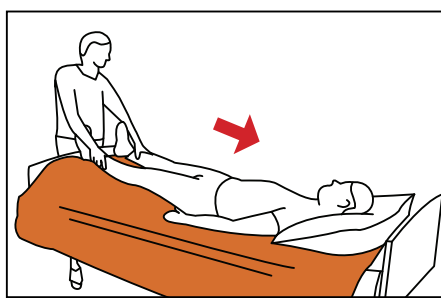
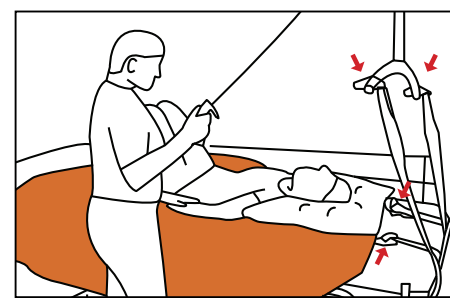
Person 1 stands at the foot of the bed and makes a weight transfer of the user's ankle/calf. The user then slides toward the foot of the bed

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
Alternatively, dismount the bed's headboard. Attach the Master Turner® pull straps to the turn sheet's straps at the head of the bed, pulled under the bed-frame and hooked onto the lift. Raise the lift to pull the user into an upright position in bed

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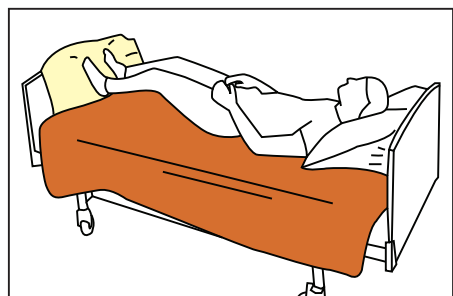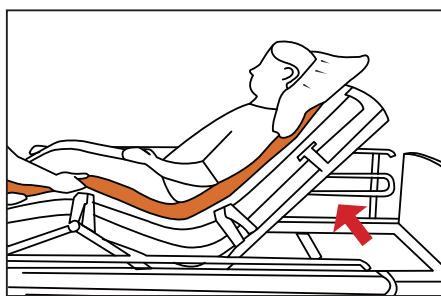
Aktiver knæ-knæk. Placer Master Comfort vaskekile i fodende

UK

Activate the adjustable bed under the user's knees. Place the Master Comfort Washing Wedge at the foot of the bed

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

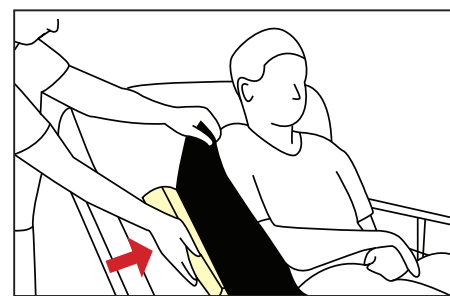
Place the Master Comfort Support Wedges under the turning sheet on either side of the user

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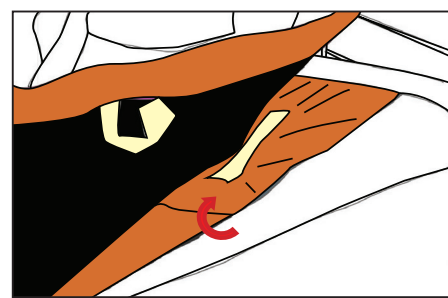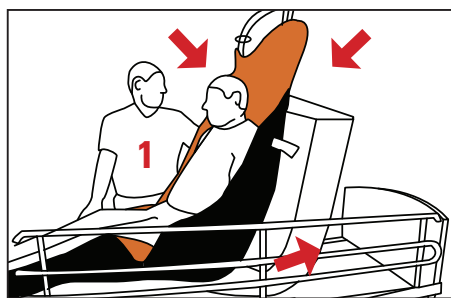UK

When this method of transfer is used, person 1 must ensure that the sliding flap's opening is facing the helper. Elevate the adjustable bed under the user's knees. Elevate the head of the bed. Attach the turning sheet's straps to the main section of the lift and raise until it fits snugly behind the user's back

Position the user's legs over the edge of the bed

The user can then slide into the desired seating position. In order to avoid sliding forward, the BASIC Cover sliding flap can be pulled back to make the BASIC Cover anti-slip visible

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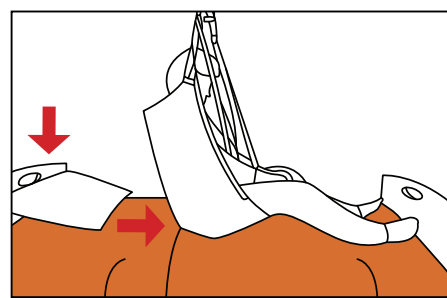
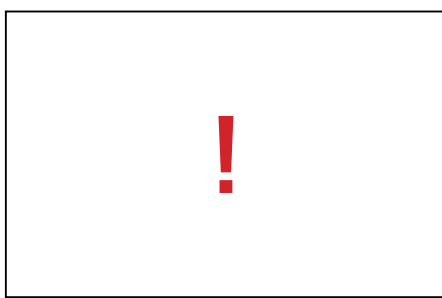
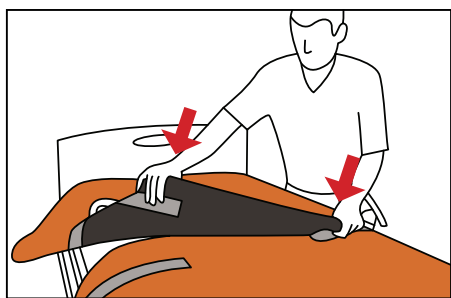
UK Position the bottom piece with the sliding part against the cover. The two soft straps must be facing upward toward the middle of the bed. Place the top piece with the sliding part against the cover

Remove the washing straps and save for later use. Attach the straps of the top piece to the bottom piece. Be careful not to work over the middle of the bed and that there are no unintended folds in the system. Pull up the top and bottom pieces until they are touching the headboard

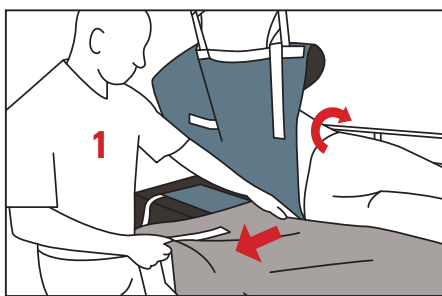
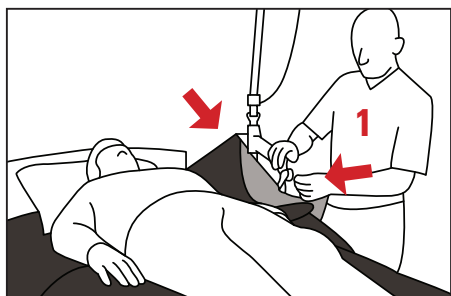
Position the user so their lower back is positioned over the system in the middle of the turning sheet. See illustration

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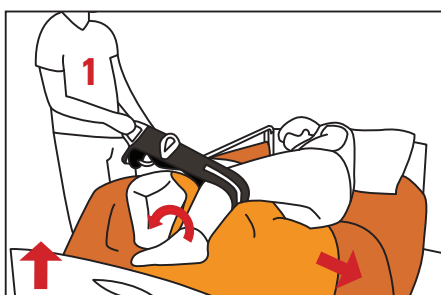
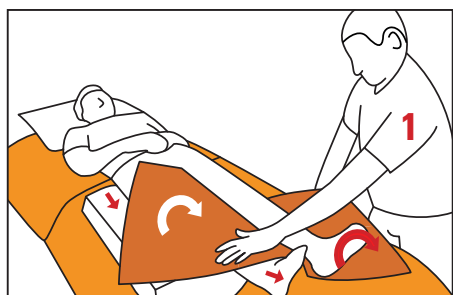
UK Fasten the straps on the one side of the top piece to the lift. Follow the instructions provided in 2.2 on page 5

If the user is heavy or uncooperative, the top piece can be fastened to a lift to help keep the user in a lateral position

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UK

Before the user is moved into a lateral position, position the Master Comfort Washing Wedge from the user's hip and down on the side you wish to turn the user. Fold a Nylon Master slide piece across the washing wedge and in between the user's legs

Turn the user as outlined in 2.2 on page 5. Extend the pull arm by positioning the Master Grip under the hollow of the knee. Use the bed guard to keep the washing wedge in place and pull the Master Grip until the leg has slid up on the Washing Wedge. Remove the Nylon Master slide piece

Remove the slide piece once the user is in a stable position. Detach the bottom piece from the top piece. Then, keep the user in a lateral position using the lift and top piece

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Materials:

Cotton, polyester, polyamide, nylon.

For Master Turner Duo Inco: Cotton, polyester, polyamide, polyurethane, nylon.

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	Handles/ Flaps:	Max. User Weight:
MC 001-1303 BASIC - Top	L 90 x B 150 cm	4	200 kg
MC 001-1305 BASIC - Bottom	L 140 x B 150 cm	2	200 kg
MC 001-1301 BASIC Cover 1 - L	L 200 x B 80 cm (130 x 60)	1	200 kg
MC 001-1314 BASIC Cover ST - L	L 200 x B 80 cm (110 x 60)	0	200 kg
MC 001-1316 BASIC Cover XL - 1	L 200 x B 120 cm (130 x 100)	1	200 kg

WARNINGS

Never leave the user on the product when lying on the edge of the bed.

Never leave the product on the floor.

Always make sure that the user is not caught in the straps.

Never leave the user lying on the product whilst connected to a hoist.

Do not use as a sling sheet. Can be combined with a hoist for e.g. repositioning in bed.

Make sure that all relevant brakes are activated before transferring the user.

CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

User Weight:

Recommended max. user weight: 200 kg.

Washing:

This product can be washed in a washing machine at max. 85 °C.

Products with Hook/Loop band can be washed at max 60 °C. Gentle washing and dry tumbling significantly prolongs the durability.

Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.

Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.

Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.

Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.

Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.

Ironing:

Do not iron.

Dry cleaning:

Do not dry clean.

Bleach:

Do not bleach.

Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.

User Manual:

Always read the User Manual thoroughly before using the product.

Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

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