

MADE IN  
EUROPE




This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

## Multi Rack

36\_20250103\_UK

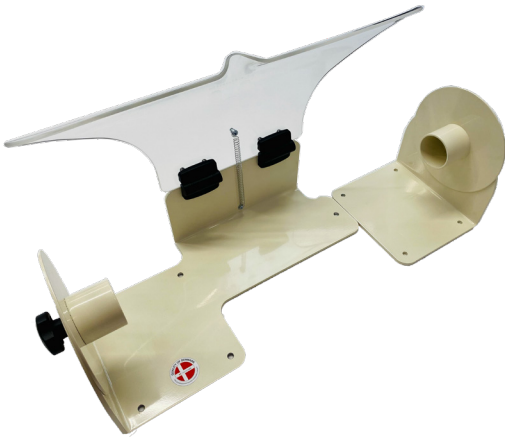

MC 002-1287



Article no.	Name - Size
MC 002-1287	Multi Rack



# User Manual - Multi Rack



Multi Rack is an adjustable rack for storing SPU covers and nylon sliding material.



## User Manual:

We recommend reading the user manual thoroughly before using the product (-s).



## Mounting the Rack:

Multi Rack is mounted using screws in the pre-drilled holes in the 2 fittings.

**Always** mount the Multi Rack using screws and rawplugs suitable for the specific wall on which it is mounted (e.g. drywall, concrete wall). Multi Rack is delivered without screws and rawplugs

We recommend a minimum of 30 cm from the floor to the Multi Rack due to hygiene.



## Safe working load:

We recommend a max SWL of 30 kg.  
If the SWL of the walls is lower, this will always apply.



## Visual inspection:

Make visual inspection frequently and check that all parts are fully functional and intact. Make sure that screws are tightened and the Multi Rack is securely fastened.  
Products with deficiencies must be discarded.



## Cleaning:

Multi Rack can be cleaned with warm water and a soap solution or if necessary a solution of 70% ethanol, 45% isopropanol or similar.



## Storage:

Multi Rack can be mounted in rooms with a temperature between 5-60°C.



## Lifespan:

Estimated lifespan of the product is 10 years with normal wear and tear



## Disposal:

The product can be disposed as PMMA (plastic) and metal. Always make sure to follow local guidelines when disposing or discarding the product.



## Warranty:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

## For more information and newest user manual:




# User Manual - Multi Rack


## Mounting horizontally:



1. Place Fitting 1 against the wall. Adjust with a spirit level. Choose the appropriate height and mark out the wholes on the wall.

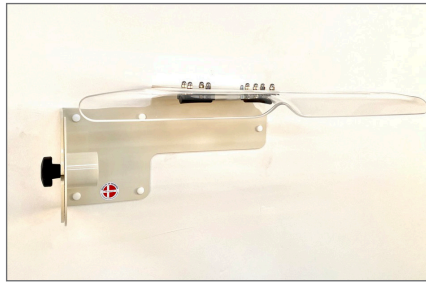


2. Mount the Fitting. Be aware always to use solid screws and if necessary rawlplugs.

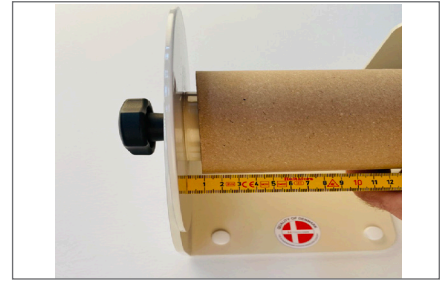

3. Measure the total length of the cardboard tube /roll your foil is fitted on. Add 2 cm to the measurement to have some clearance between the cardboard tube and the two Roll Holders.



4. Measure from the inside of Fitting 1 to the right to find the correct distance for Fitting 2. Use the measurement from text 3.

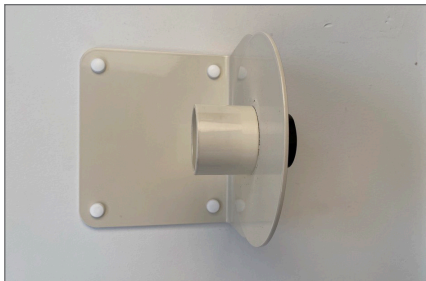

5. Place Fitting 2 level with Fitting 1. Mark the wholes on the wall and mount Fitting 2. Always use suitable screws and if necessary rawlplugs.



6. Place the Roll Holder B in the track of Fitting 2. Tighten the screw sufficiently.



7. Place one end of the cardboard tube over the Roll Holder B on Fitting 2.




8. Place the lose Roll Holder A on the opposite end of the cardboard tube.




9. Place Roll Holder A on the track of the Fitting 1. Tighten the screw sufficiently.

# User Manual - Multi Rack

## Mounting horizontally (continuation):

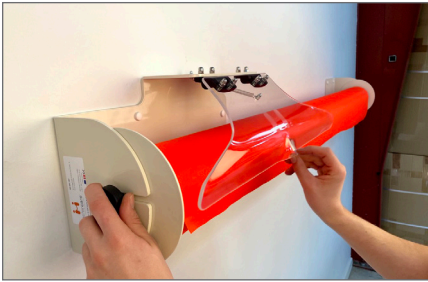

10. Place the Foil Holder upon the Foil or Nylon – this way the material stays in place.

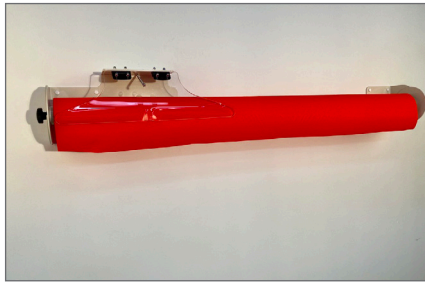

11. Multi Rack is mounted and ready for use.

## Mounting vertically:




12. Place Fitting 2 on the wall. Use a spirit level to adjust the position. Choose the appropriate height, mark out the wholes upon the wall and mount it.




13. According to current hygiene standards it is recommended to place Fitting 2 at least 30 cm above the floor.



14. Make sure to place the Roll Holder B in the track of Fitting 2 and tighten the screw.

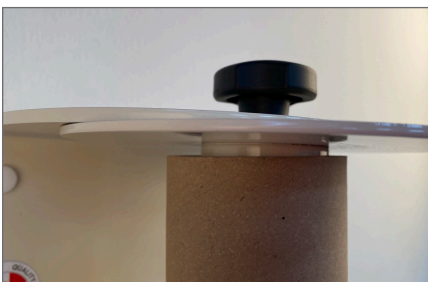

15. Measure the total length of the cardboard tube /roll your foil is fitted on. Add 2 cm to the measurement to have some clearance between the cardboard tube and the two Roll Holders.

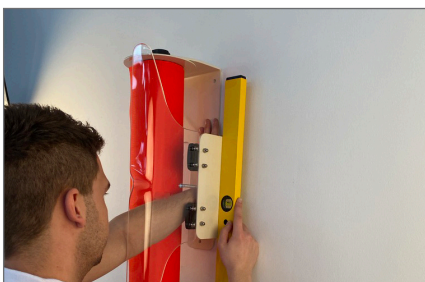

16. Place Fitting 1 level with Fitting 2. Mark the wholes on the wall and mount Fitting 1. Always use suitable screws and if necessary rawlplugs.




17. Place one end of the cardboard tube over the Roll Holder B on Fitting 2.



# User Manual - Multi Rack


## Mounting vertically (continuation):

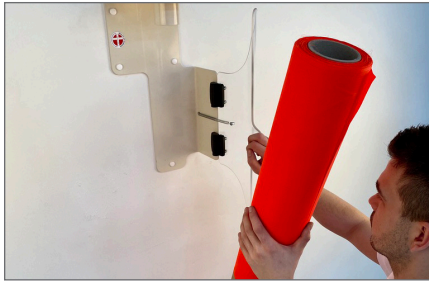

18. Open the Foil Holder on Fitting 1.



19. Place the loose Roll Holder A on the opposite end of the cardboard tube.

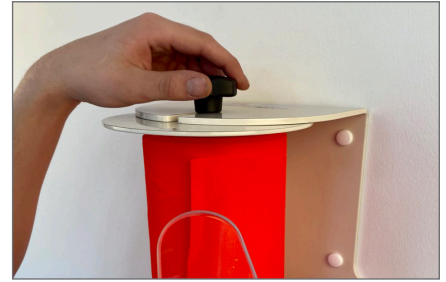

20. Place Roll Holder A on the track of Fitting 1. Tighten the screw sufficiently.




21. Place the Foil Holder upon the Foil or Nylon – this way the material stays in place and is ready for use.

## Replacing the Foil/Nylon Roll:



23. Open the Foil Holder.

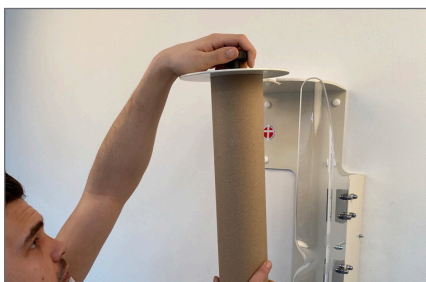

24. Loosen the screw on the Roll Holder A.



25. Slide out the cardboard tube with the Roll Holder A from Fitting 1. Lift off the Cardboard tube and replace with a new one. Reposition the Roll Holder A and fit into the track on Fitting 1. Tighten screw.

# User Manual - Multi Rack

## Technical Specifications

**Article no. - Product Name:**

MC 002-1287 Multi Rack incl. Roll Holder  
Roll Holder 5,8  
Roll Holder 7,2

**Size:**

L 70-170 x H 25 x D 32  
Cardboard inner measurements min. 5,8 cm  
Cardboard inner measurements min. 7,2 cm

**Materials:** Steel (S235) and PMMA.



For more information, videos  
Quick Guide etc., visit  
[www.master-care.dk](http://www.master-care.dk)