

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

38_20250114_UK

Master Comfort Sleep

The perfect cushion for sleeping



Master Comfort Sleep



Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic
Human Handling Concept



SCAN ME

Developed &
Manufactured by:



master-care.dk

REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-8085	136598	Master Comfort Sleep	05744001974650
MC 001-8086	136637	Master Comfort Sleep - Cover	05744001974667

Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the costumer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The products can be used by people who have impaired or partial impaired capacity to change body position due to restricted movement or body perception damage and therefore are unable to provide a stable body positioning without support. Furthermore, the products can prevent/ reduce injuries caused by pressure or shearing forces for users who are in risk of pressure or shear ulcers.

The products can also have a stabilizing and calming effect on people with sensory or cognitive impairments and have a pain-relieving effect.

The products increase the support surface, which gives a better distribution of the pressure on the tissue thereby reducing the risk of pressure injuries.

Proper use of the product:

The Master Comfort products can be placed between the user and the area of support and thereby reduce the pressure on the skin. Alternatively, the products can be used to support the sitting position or when lying on the side to prevent falling forward or backwards.

Indications:

The product is suitable for users with impaired physical and/or cognitive function who need support in a lying position. It can also be used to help prevent pressure ulcers in the user.

Contraindications:

Don't use the products in directly contact with open wounds. Otherwise there are no known contraindications for the use of the product.

Warning:

It is very important for the patient to reposition themselves, or is repositioned, on a regular basis. This relieves pressure which helps prevent both tissue compression and potential pressure injury formation. Skin redness is one clinical indicator for skin breakdown. Regularly inspect/monitor if your skin develops any redness. Contact a healthcare professional immediately, if the skin develops redness.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of the products is 1 - 5 years depending on the use, user's load and how often it is washed and dried.

Products which no longer provide sufficient support, with defect seams, has broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Comfort Sleep

Placing the Master Comfort Sleep:



1. Master Comfort Sleep provides a stabile positioning in bed. Can be placed either in front of the body or behind the back and legs. The top can be used as a pillow.



2. Place the Master Comfort Sleep behind the user's head, back and under the upper leg.



3. The lower part of the Master Comfort Sleep can be modulated to give optimal support to the upper leg or be placed between the legs to provide pressure relief.

Seated positioning:



4. Master Comfort Sleep can be used to support/ increase body perception and awareness to the upper body when in supine lying. Place it like a "U" along the upper body and behind the head.



5. Place the middle of Master Comfort Sleep under the head. Leave the Bed Rail up to keep the Krøyer balls in place.



6. Place and adjust the left and right side of Master Comfort Sleep along the body. Adjust the balls to comfortably support the forearms and hands.

Cover:



7. The Master Comfort Sleep is delivered with a incontinence surface. The fabric is stretchable and has a long durability. It can be cleaned with soap and water or with alcohol.



8. A soft, comfortable and washable cover is available to protect the cushion.



9. Master Comfort Sleep consists of real Krøyer balls which added to the body and give the body a sense of stability and a good pressure relief.

User Manual - Master Comfort Sleep

Technical Specifications

ISO certifications:

DS/EN ISO 12952-1:2010

Textiles – Assessment of the ignitability of bedding items – Part 1: Ignition source: smouldering cigarette.

DS/EN ISO 12952-2:2010

Textiles – Assessment of the ignitability of bedding items – Part 2: Ignition source: match-flame equivalent.

REF/Article no. - Product Name:	Size:	Max. user weight /SWL:	Contents:	Package weight:
MC 001-8085 Master Comfort Sleep	L 200 x W 50 cm	Not relevant	Krøyer balls: 100% Polystyrene Inco cover: 60% Polyester, 40 % Polyurethane	3,02 kg
MC 001-8086 Master Comfort Sleep - Cover	L 200 x W 50 cm	Not relevant	Blue cover: 70% Cotton, 30% Polyester	0,54 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



The use of pillows and wedges can't replace the need of frequent repositioning and daily skin assessment.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: There is no max user weight on the product.



Washing:

The Blue Cover can be washed in a washing machine at max. 85 °C. Lower washing temperature will prolong the durability. The Master Comfort Sleep can be disinfection with alcohol - do not wash it in a washing machine.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

The Blue Cover may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

The Master Comfort Sleep tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make frequent visual inspections to check that the incontinence surface and all seams are intact before use



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

Find us on:

