

MADE IN
EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

StayUp Strap

26_20250114_UK



	Article no.	HMI no.	Name	UDI-DI/GTIN	Size
○	MC 001-1944	136866	StayUp Strap	5744001974827	L 75 x W 3,6 cm
○	MC 001-1943	-	StayUp Strap - 10 pcs	5744001974919	L 75 x W 3,6 cm



1. Place the StayUp Strap on top of the pants around the thigh.



2. Pass the strap through the plastic ring.



3. Tighten the strap and close the hook band against the elastic band.



4. Open the pants and release them. They can only fall down to the StayUp Strap. Regardless of whether you stand or sit when going to the toilet, the trousers will not drop to the floor.

The Original Nordic
Human Handling Concept



Developed &
Manufactured by:



User Manual - StayUp Strap

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

Intended purpose:

The intended use of this product is to prevent the pants from dropping to the floor when going to the toilet and thereby minimizing the risk of falling for users with decreased balance.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of the product with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, load and how often it is washed/dried.

Check that the hook binding is sufficient to hold the pants up. If not or if the product is broken it must be discarded.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

ADVARSLER




StayUp Strap must be removed after use to prevent alternations in the circulation.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

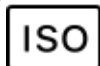

ISO klassificering:

ISO 12952-1:2015
Tekstiler - Vurdering af sengeudstørs
antændelighed - Del 1: glødende
cigaret.

ISO 09 09 09 01

For more information, videos
Quick Guide etc., visit
www.master-care.dk



User Weight:

No max. user weight.



Washing:

Products *with* Hook-and-Loop tape can be washed in a washing machine at max. 60 °C. Gentle washing significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

The expected life span of products with daily wear, tear and washing averagely are 1-3 years. The products life span depends on the use, load and how often it is washed/dried. Check that the hook band has sufficient connection to the elastic band. If the products has defect seams, broken material or other deficiencies it must be discarded. Make visual inspections frequently and preferably before every use.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.