

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

22_20260301_UK

Dress Master - Arm

Pat. BA 2022 00046

Gebrauchsmusters Nr. 20 2023 105 598



REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-0993	141963	Dress Master - Arm	05744001975169

Regulation MDR 2017/745 for medical devices class 1



ISO 12 03 03 20

The Original Nordic Human Handling Concept



Developed & Manufactured by:

master-care.dk

User Manual - Dress Master - Arm

Table of Contents

Important information before using the product(s):	page 3
Using Dress Master - Arm:	page 4
Undressing shirt/jacket:	page 4
Dressing when sitting:	page 5
Cleaning and reusability:	page 5
Technical Specifications	page 6

Thank you for choosing a product from master care A/S.

Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is a sliding tube for dressing. The double sliding aspect reduces the risk of shear injuries and reduces the ergonomic load for the helper during dressing.

Correct use of the product:

Slide the Dress Master on the user's leg(-s) /arm(-s) before sliding the clothes over the Dress Master. When the clothes is on remove the Dress Master by pulling lightly in the bands.

Indication:

The product is used when the user is unable to dress them self and friction is high against the skin due to e.g. lack of mobility, decreased physical or cognitive function.

Contraindications:

Do not use the product if the patient has open wounds. Always dress wounds before using the Dress Master.

Warning:

Never leave the Dress Master on the floor due to risk of falling.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for dressing the user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Production and Lifespan:

The product's date of manufacture can be found on the product label under "LOT" and is shown as YYMMDD – for example, 250920, which indicates that the product was manufactured on September 20, 2025.

The product is expected to have a service life of 1–3 years under normal wear and washing conditions. The service life of the product depends on usage, load, and how and how often the product is washed.

Inspection:

Make visual inspections frequently and preferably before every use. Products that no longer provide sufficient friction reduction to ensure a health-safe transfer for both caregiver and user should be discarded. Products should be inspected regularly, and if defective seams or damaged surfaces are found, the product should be discarded.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Dress Master - Arm

Using Dress Master - Arm:



1. Place a piece of sliding material under the user's upper body. Place the Dress Master - Arm on the user's arm so that it covers the shoulder.



2. Pull the shirt/jacket over the Dress Master - Arm and well up towards the neck.



3. Turn the user to the side and pull/push the opposite side of the shirt/jacket down towards the hip and underneath the user so that the shoulder seam is towards the user's back.



4. Turn the user back to supine position, bend opposite arm and slide it into the sleeve. Straighten and close the shirt/jacket.



5. Dress Master - Arm can be pulled out by alternately pulling the white bands. Remove the sliding piece under the user.

Undressing shirt/jacket:



6. Place a sliding piece under the user's upper body. Slide the Dress Master - Arm onto both of the user's arms and under the user's back so that they overlap and cover the user's shoulders.



7. Fold the collar backwards over the shoulders and down between the 2 layers. Pull it down towards the waist.



8. The arms can then easily slide out of the sleeves.

User Manual - Dress Master - Arm



9. If necessary, slide the Dress Master - Arm over the jacket - between the user's lower back and the jacket - before the jacket is pulled out to avoid shear and the shirt is displaced. Remove the sliding piece.

Dressing when sitting:



10. Place the Dress Master - Arm on the user's arm so that it covers the shoulder.



11. Pull the shirt/jacket over the Dress Master - Arm and pull it too far back/down the user's back towards the opposite hip.



12. Place the arm in the armhole on the opposite side and pull the shirt/jacket on. Adjust and close the shirt/jacket.

Cleaning and reusability:



13. Pull out the Dress Master - Arm by pulling alternately on the 2 white bands.



14. The product can be reused by other users if it is washed and clean and if seams, straps, fabric etc. are intact.

User Manual - Dress Master - Arm

Technical Specifications

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	Max. user weight:	Product contents:	Product weight:
001-0993 Dress Master - Arm	L 125 x W 26/36 cm	150 kg	16% Polyester, 76% Polyamide, 8% Silicone.	0,14 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 150 kg



Washing:

This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

