

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

44_20260408_UK

Giwik – Ankle Pad



REF (Article no.)
GW2002

HMI no.
101623

Name
Giwik – Ankle Pad

UDI-DI/GTIN
05744001974735

Size
Length 11 cm,
Outer diameter 46 cm,
Inner diameter 35 cm

The Original Nordic
Human Handling Concept



Scan Me

Developed &
Manufactured by:



User Manual - Giwik - Ankle Pad

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

Intended purpose:

The product is intended to provide pressure relief on pressurized body parts or to relieve joints to reduce the risk of pain.

Indication:

Can be used if the user experiences pain or discomfort during positioning, e.g., in the side-lying position, or is at risk of pressure on the ankle's bony prominence.

Contraindication:

It is recommended that open wounds be dressed before using the pressure-relieving pad. Otherwise, no contraindications are known for the use of the product.

Content: Fabric: 90% cotton, 10% polyester. Foam: 100% polyethylene. Elastic band: 100% rubber

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of the product with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, load and how often it is washed/dried.

Check that the friction on the surface of the product is sufficient for the intended use. If not or if the product is broken it must be discarded.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

ADVARSLER



We recommend that the treatment of pressure sores is carried out in collaboration with healthcare professionals. If pressure marks do not disappear when using the product, professional healthcare advice should also be sought.



CE Certification:

The products are CE-approved, meeting the essential health and safety requirements of the Medical Device Regulation (EU) 2017/745, Class I.



Medical device, compliant with the Medical Device Regulation (MDR) 2017/745.

For more information, videos Quick Guide etc., visit www.master-care.dk



User Weight:

No max. user weight.



Washing:

This product can be hand washed at 60 °C.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

The expected life span of products with daily wear, tear and washing averagely are 1-3 years. The products life span depends on the use, load and how often it is washed/dried. If the products is defect/broken, it must be discarded. Check that the foam still has a bouncing feeling when pressing and releasing pressure on the foam. Make visual inspections frequently and preferably before every use.



Dry Tumbling:

Tumbling drying is NOT recommended since the product can melt/shrink.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.