

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

44\_20250114\_UK

## Giwik - Crutches Pad Adult



REF (Article no.)	HMI no.	Name	UDI-DI/GTIN	Size
<input type="radio"/> GW2008	101620	Giwik - Crutches Pad Child	05744001974889	Length 9 cm, Inner diameter 9 cm
<input type="radio"/> GW2010	101476	Giwik - Crutches Pad Adult	05744001974902	Length 12 cm, Inner diameter 15 cm

The Original Nordic  
Human Handling Concept



Developed &  
Manufactured by:



master-care.dk

# User Manual - Giwik - Crutches Pad

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

## MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## Intended purpose:

The product is intended to provide pressure relief on pressurized body parts or to relieve joints to reduce the risk of pain.

## Indication:

The pressure-relieving pads to minimize pain and discomfort from hand grips and elbow support when walking with crutches.

## Contraindication:

No contraindications are known for the use of this product.

## Mounting of the product:

The “tubes” are mounted around the crutch handles with the seams facing downward and mount the “bag” around the elbow support.

**Content:** Fabric: 90% cotton, 10% polyester. Foam: 100% polyethylene.

## Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

## Life span:

The expected life span of the product with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, load and how often it is washed/dried.

Check that the friction on the surface of the product is sufficient for the intended use. If not or if the product is broken it must be discarded.

Make visual inspections frequently and preferably before every use.

## Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

## ADVARSLER



Ensure the Velcro strap is always securely fastened around the crutch's metal shaft so that the tube does not rotate during walking.

We recommend that treatment of pressure sores takes place in collaboration with healthcare professionals. If pressure marks do not disappear when using the product, professional medical advice should also be sought.



### CE Certification:

The products are CE-approved, meeting the essential health and safety requirements of the Medical Device Regulation (EU) 2017/745, Class I.



Medical device, compliant with the Medical Device Regulation (MDR) 2017/745.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)



### User Weight:

No max. user weight.



### Washing:

This product can be hand washed at 60 °C.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Maintenance:

The expected life span of products with daily wear, tear and washing averagely are 1-3 years. The products life span depends on the use, load and how often it is washed/dried. If the products is defect/broken, it must be discarded. Check that the foam still has a bouncing feeling when pressing and releasing pressure on the foam. Make visual inspections frequently and preferably before every use.



### Dry Tumbling:

Tumbling drying is NOT recommended since the product can melt/shrink.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.