

MADE IN EUROPE



Vores produkter og råvarer er produceret i Europa.

Vælg kvalitet, spar penge og skån miljøet.

Quick Guide



(DK) **Vario Cover** - kombiner med Master Turner
(UK) **Vario Cover** - combine with Master Turner

The Original Master Turner System


09_20260114 DK/UK



MC 001-2830

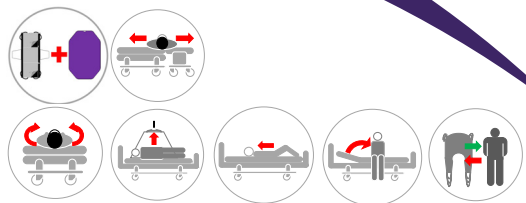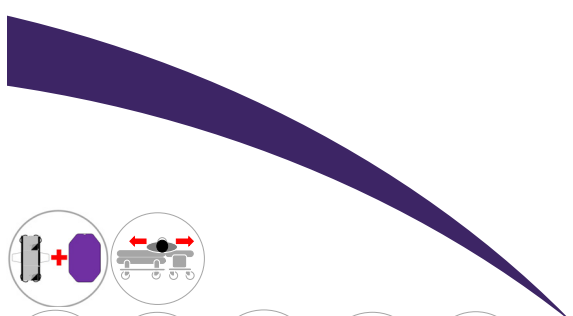

REF (Vare nr./Item no.)	HMI nr./no.	Navn/Name - Størrelse/Size	UDI-DI/GTIN
● MC 001-2830	146580	Vario Cover - L2E 200x88	05744001975527

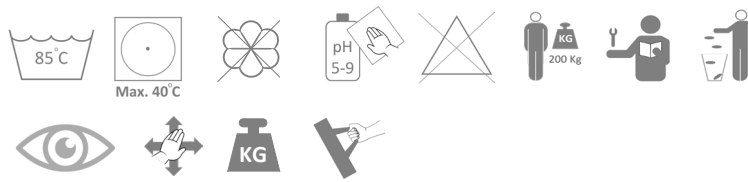

Eftersyn:

Produktet skal kontrolleres jævnlig og gerne inden hver brug. Kontroller, at der ikke er skader på stof, stropper, sømme mm. Har produktet høj friktion eller på anden vis er i stykker, skal produktet kasseres.

Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1
(UK) Regulation MDR 2017/745 for medical devices class 1



(DK) Efterlad aldrig bruger på produktet på sengekanten.
(UK) Never leave the user on the product when lying on the edge of the bed.




(DK) Efterlad aldrig produktet liggende på gulvet.
(UK) Never leave the product on the floor.



(DK) Efterlad aldrig en bruger liggende på produktet, mens dette er koblet på en lift.
(UK) Never leave the user lying on the product whilst connected to a hoist.



(DK) Sørg altid for, at brugeren ikke hænger fast i stropperne.
(UK) Always make sure that the user is not caught in the straps.

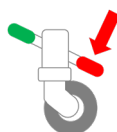

(DK) Sørg for, at alle relevante bremsere er låst, inden forflytning foretages.
(UK) Make sure that all relevant brakes are activated before transferring the user.



(DK) Må ikke bruges som liftsejl – kan med fordel kombineres med lift til fx vendinger i seng.
(UK) Do not use as a sling – can be combined with a hoist for repositioning in bed.

ISO 12 31 03 21

The Original Nordic Human Handling Concept



(DK) Brugervejledning, videoer mm., se
(UK) User Manual, videos etc. visit

Developed & Manufactured by:



master-care.dk

www.master-care.dk