

# MADE IN EUROPE




This product has been developed and manufactured using both raw materials and with production in Europe.


Choose quality, save money, and protect the environment.

## Quick Guide Light Cover Sand


24\_260323 DK/UK

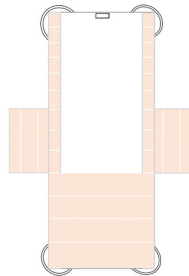


Light Cover Sand - M

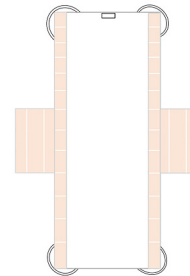

Light Cover Sand - L

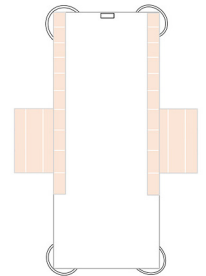

Light Cover Sand - Leg

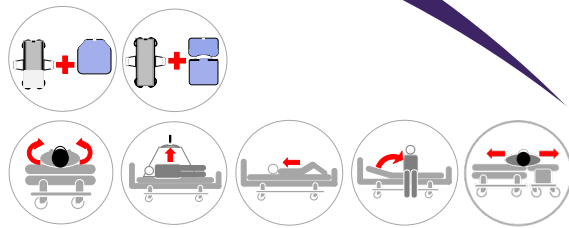

| REF (Vare nr./Item no.) | HMI nr./no. | Navn/Name - Størrelse/Size | UDI-DI         |
|-------------------------|-------------|----------------------------|----------------|
| ○ MC 002-2734           | 145477      | Light Cover Sand - M       | 05744001975411 |
| ○ MC 002-2735           | 145478      | Light Cover Sand - L       | 05744001975428 |
| ○ MC 002-2736           | 145479      | Light Cover Sand - Leg     | 05744001975435 |

### Erklæret formål:

Produktet nedsætter friktionen mod underlaget ved vendinger i seng. Velegnet til brugere med smert-er og bevægelsesnedsættelse i hofte, bække og /-eller ryg.

### Intended use:

The intended use is to reduce friction against the surface while turning in the bed. Suitable for users with pain and reduced movement in hip, pelvic and/or back. The product has 2-way-glide.

### Eftersyn:

Produktet skal kontrolleres jævnlig og gerne inden hver brug. Kontroller, at der ikke er skader på stof, stropper, sømme mm. Har produktet høj friktion eller på anden vis er i stykker, skal produktet kasseres.

### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



**(DK)** Efterlad aldrig bruger på produktet på sengekanten.

**(UK)** Never leave the user on the product when lying on the edge of the bed.



**(DK)** Efterlad aldrig produktet liggende på gulvet.

**(UK)** Never leave the product on the floor.



**(DK)** Sørg for, at alle relevante bremsere er låst, inden forflytning foretages.

**(UK)** Make sure that all relevant brakes are activated before transferring the user.



**(DK)** Sørg altid for, at brugeren ikke hænger fast i stropperne.

**(UK)** Always make sure that the user is not caught in the straps.

**(DK)** Forordning MDR 2017/745 for medicinsk udstyr klasse 1  
**(UK)** Regulation MDR 2017/745 for medical devices class 1  
ISO 12 31 03 21



The Original Nordic  
Human Handling Concept



master care A/S, Sofienlystvej 3, 8340 Malling, Danmark  
Tlf. +45 86 93 85 85, E-mail: info@master-care.dk

(DK) Vi tager forbehold for evt. trykfejl og mangler.  
(UK) Reservations are made for printing errors.

