# MADE IN EUROPE

This product has been developed and manufactured using both raw materials and with production in Europe.

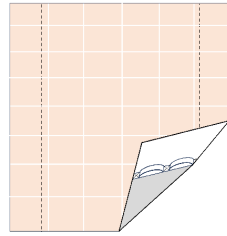
Choose quality, save money, and protect the environment.

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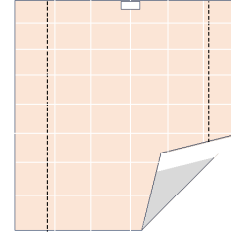
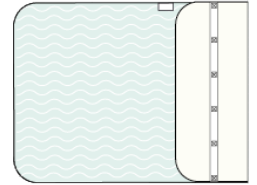
#### Intended use:

The intended use for the product is to reduce friction between the user and the mattress to make it easier for the helper and gentler for the user whilst turning, moving up/ downwards, and moving to sitting position on the edge of the bed. This product should always be used in combination with a Satin Sheet from master care a/s. Combined the two products have 4-way-glide.

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#### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.

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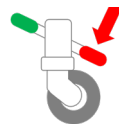
**(UK)** Never leave the user on the product when lying on the edge of the bed.

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**(UK)** Never leave the product on the floor.

**(DK)** Sørg for, at alle relevante bremses er låst, inden forflytning foretages.

**(UK)** Make sure that all relevant brakes are activated before transferring the user.

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**(UK)** Always make sure that the user is not caught in the straps.

**(DK)** Forordning MDR 2017/745 for medicinsk udstyr klasse 1  
**(UK)** Regulation MDR 2017/745 for medical devices class 1  
 ISO 12 31 03 21

The Original Nordic Human Handling Concept

master care A/S, Sofienlystvej 3, 8340 Malling, Danmark  
 Tlf. +45 86 93 85 85, E-mail: info@master-care.dk

**(DK)** Vi tager forbehold for evt. trykfejl og mangler.  
**(UK)** Reservations are made for printing errors.