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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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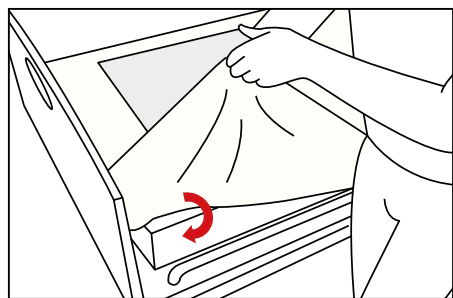
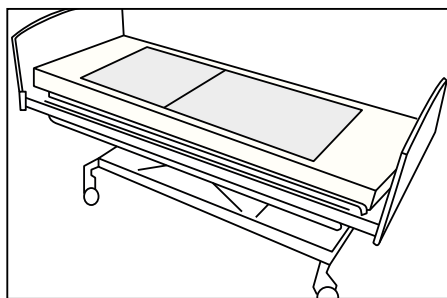
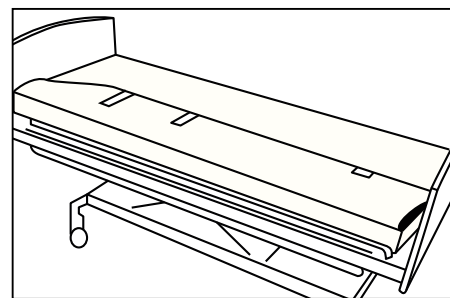
UK Place the One Time mattress cover on the head section of the mattress and fold the stretch sheet down around the four corners of the mattress

The One Time mattress cover fits tightly around the mattress

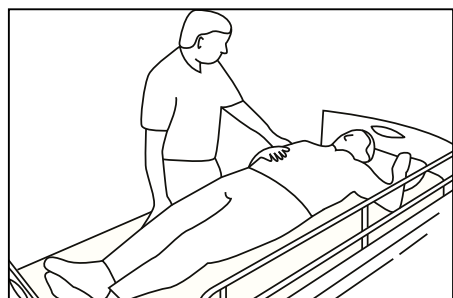
Place the One Time turn sheet on top of the mattress cover

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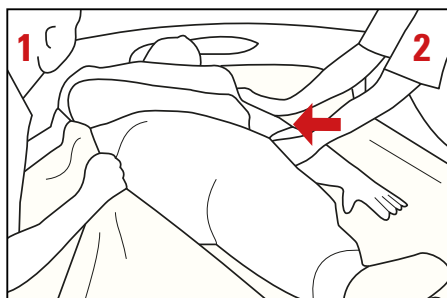
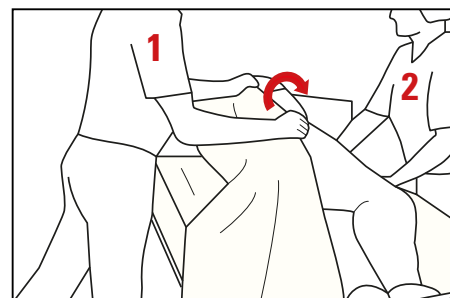
UK Cross the user's legs. Guide the user's hand to the opposite shoulder. Push the pillow and shoulder forward on the side to which the user is going to turn

1 pulls and 2 pushes the user towards the desired edge of the bed

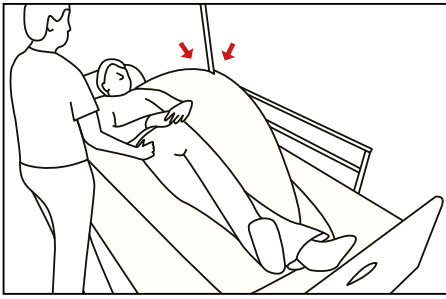
1 pulls the turn sheet and pulls the user over onto the side

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
UK Cross the user's legs and pull the pillow and shoulder forwards on the side to which the user is going to turn. Attach the turn sheet's straps on the one side of the lift

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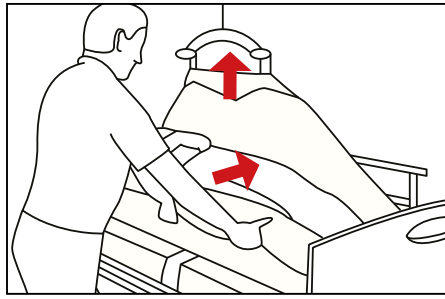
The user is lifted up slightly. The assistant pushes the user over towards the guard rail. Use the Master Comfort stomach wedge to provide greater safety during turning

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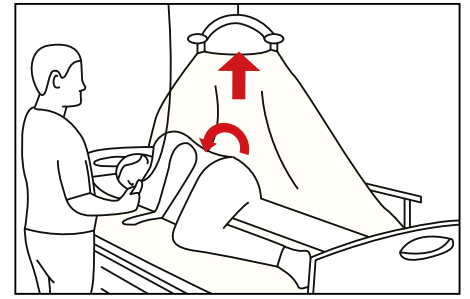
Raise the lift. The user is turned onto the side

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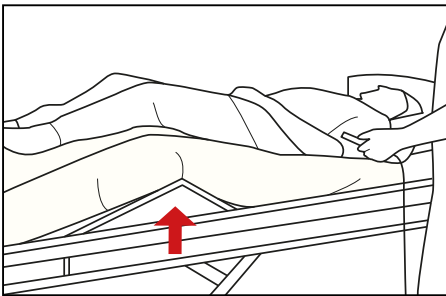
UK The user activates the middle section of the bed

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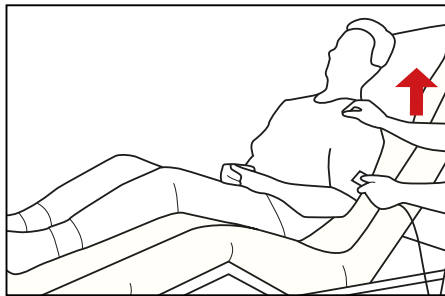
The user elevates the head section of the bed

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
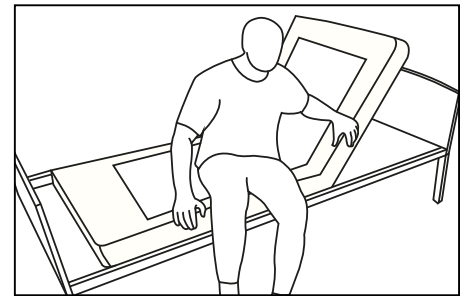
The user can now rotate on the smooth silk section and slide the legs over the edge of the bed

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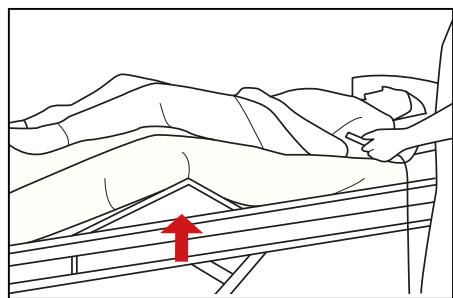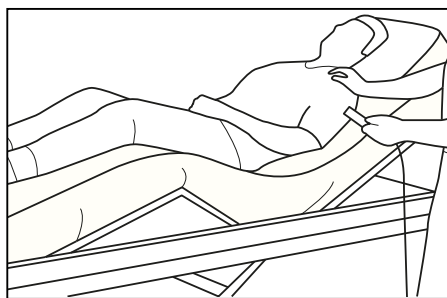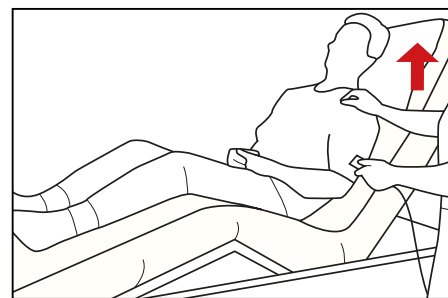
UK The assistant elevates the middle section of the bed under the user's knees

The assistant elevates the head section so that the user slides down the bed and the elevated middle section fits under the user's bent knees

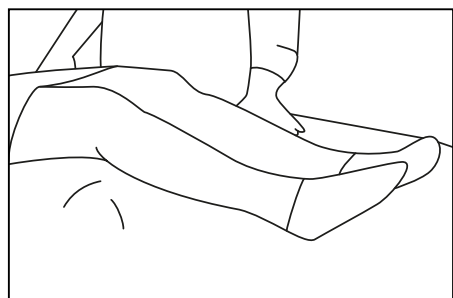
When the user has slid down, the head section is elevated as required and the user is now sitting upright in the bed

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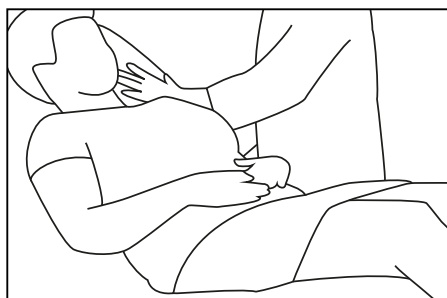
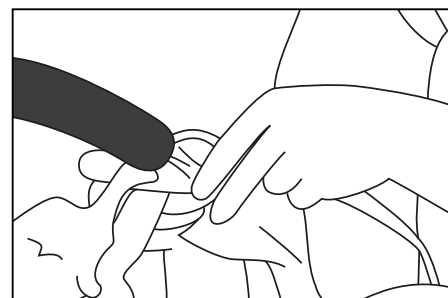
UK Activate the middle section

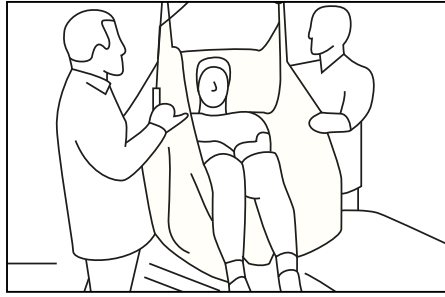
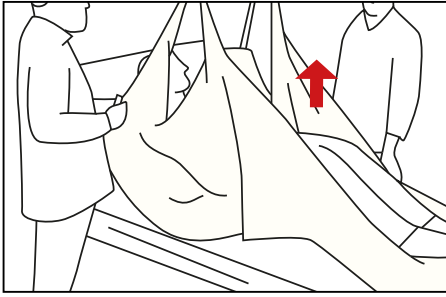
Activate the hip section so the user is sitting at about a 45 degree angle

Please the top straps and leg straps on the lift yoke – Use the outermost straps

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Brugeren kan nu overføres

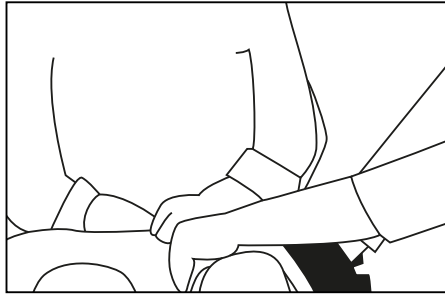
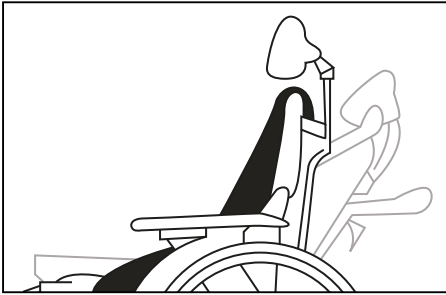
UK Then the bed is lowered and the client is lifted off the underlay

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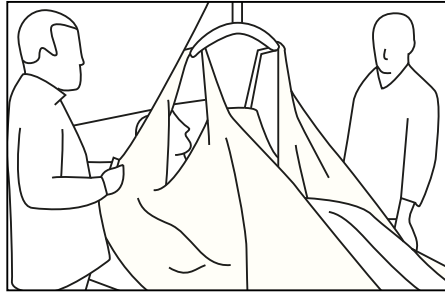
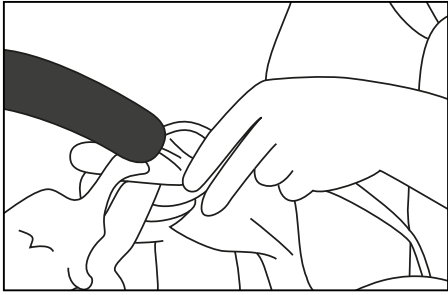
UK Lower the back of the wheelchair so the user can move properly back into the seat

Fold the One Time Lift down along the side of the user. Fold it along the inside. The turn sheet is now comfortably folded away but easy to pull out if there is a need for lifting

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UK Place the top straps and leg straps onto the lift yoke – Use the outermost

Lift from the wheelchair to the bed. Place the hips slightly above the middle section of the bed and elevate the head section of the bed. Remove the top and leg straps and smooth out folds and wrinkles so the user is lying comfortably

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UK

- Inexpensive solution in acute situations
- Ensures high hygienic standards
- Easy to fit on the mattress
- Prevents pressure ulcers

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UK

- Ensures high hygienic standards
- Affords a quick and inexpensive solution in emergency situations
- Can be used several times for the same user
- Can be used with all types of floor or ceiling lifts
- 4 handles for easy handling
- Has excellent frictionless properties, thus preventing contact sores as the risk of shear is decreased

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MC 001-2517 [HMI: 53666]
Master Turner Two In One Soft - **Small**

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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