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I tvivl?: +45 8693 8585

Washing- and tumble instructions

Washing temperature: **40 - 85°C**
Tumble dry: **max 60°C**
Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds

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UK

WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- Velcro straps must be closed by washing and drying
- During wash and drying, the washing straps MUST be applied to the velcro on the toppiece

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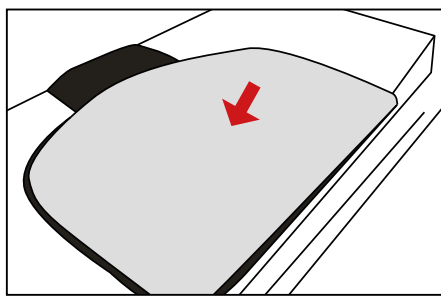
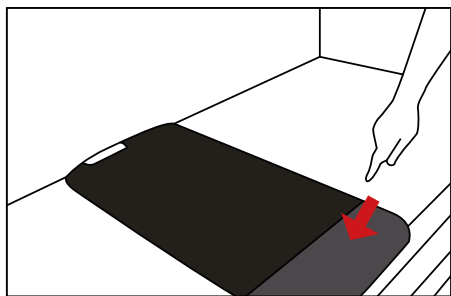
Travel Master Top placeres oven på bundstykket med den bredeste side udaf

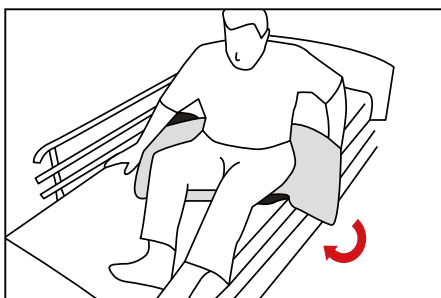
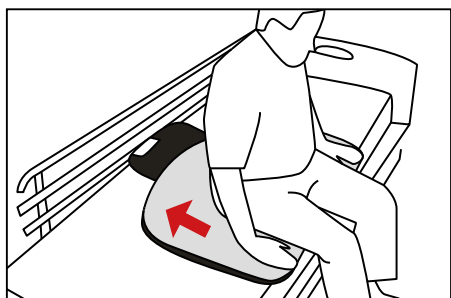
UK Travel Master Bottom should be placed in the centre of the bed. The anti-slip piece should face the edge of the bed. This is so you do not fall out of the bed

Travel Master Top is placed on top of the Bottom with the widest edge facing outwards

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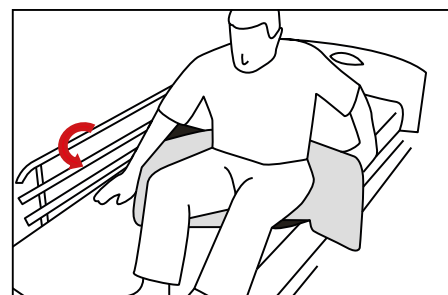
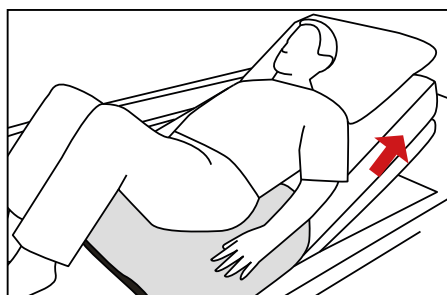
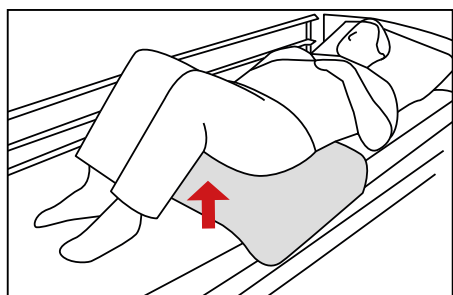
UK You can sit comfortably with Travel Master

Then you can easily turn in the bed, as there is less friction underneath

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Bend your knees

Lift the headrest of the bed

Rotate and move your legs over the site of the bed, with the help of Travel Master

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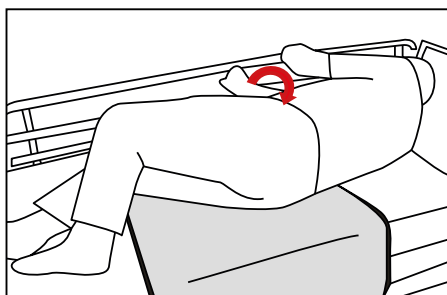
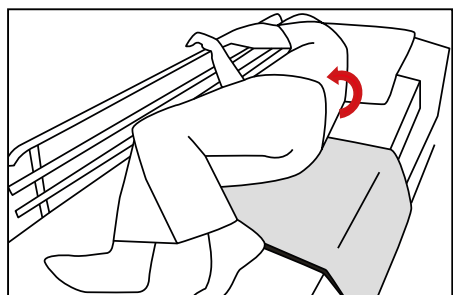
UK

You can bend your knees and rotate the upper body (using the natural movement of the back), with the help of Travel Master, as there is a greater freedom of movement

When you have turned onto the side, you are able to move freely around the pelvis

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UK

- Soft, skin-friendly surface
- Absorbs moisture
- Can also be used in a double bed
- Unique sliding properties (HLF)
- The Bottom piece has a non-slip edge
- Model Two In One can be used for incontinent users
- Easy to carry
- Long shelf life

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

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- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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