

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

17\_20260423\_UK

## Master Turner Bath / Shower


The **Original** Master Turner System



MC 001-1599



MC 001-1600

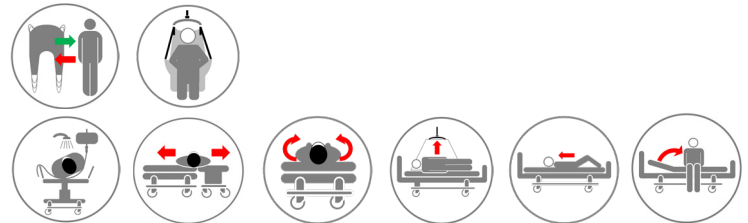

REF (Article no.)	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-1599	103106	Master Turner Shower	05744001970485
MC 001-1600	84811	Master Turner Bath	05744001970492

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic Human Handling Concept



Scan Me

Developed & Manufactured by:



# User Manual - Master Turner Bath and Shower

## Table of Contents

Important information before using the product(s): .....	page 2
Placing the Master Turner Bath/Shower under user in bed: .....	page 3
Transfer sideways to/from the Bath stretcher: .....	page 3
Turning on the bathing unit using the lift: .....	page 3
Removing the Master Turner Bath/Shower: .....	page 4
Only Master Turner Bath - Out of the bathing unit and into bed: .....	page 4
Technical Specifications .....	page 5



# Thank you for choosing a product from master care A/S.

## Important information before using the product(s):

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## The intended use of the product/-s:

The products can be used while transferring a user who has difficulty with or is unable to get into the bathtub or to the bath stretcher. The products can be used manually, with a hoist and with or without sliding material.

Master Turner Bath can also be used as a lifting sling from, e.g. the bath stretcher or -tub to the chair.

## Correct use of the product:

Place the patient directly on the Master Turner Bath or Shower. Use the handles to slide the user to the bath stretcher. If a Master Turner Bath is used it is possible to combine it to a hoist for the transfer to/from bed. Use the handles in the Master Turner Bath/ Shower to mobilize the patient in the bathtub or -stretcher to obtain a more ergonomic working condition and avoid shear injuries.

## Indication:

The products are suitable for bathing situations where the user needs to be repositioned on, for example, a shower trolley, and where this is ergonomically demanding and/or if the user is at risk of shear injuries during repositioning on the shower trolley.

## Contraindication:

No contraindications for use is known for the products.

## ISO approval:

Master Turner Bath is approved in accordance with ISO 10535:2006 for use with a hoist spreader bar with hook attachment points.

## Warning:

**Master Turner Shower is not approved for lifting.**

## Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

## Date of manufacture and Life span:

The product's production date can be found on the product label under 'LOT' and is displayed as YYMMDD - e.g. 251020, which refers to the product being produced on the 20th of September 2025.

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

## Inspection:

Make visual inspections frequently and preferably before every use. Products that no longer provide sufficient friction reduction to ensure a health-safe transfer for both caregiver and user should be discarded. Products should be inspected regularly, and if defective seams or damaged surfaces are found, the product should be discarded.

## Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.


master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.

# User Manual - Master Turner Bath and Shower

## Placing the Master Turner Bath/Shower under user in bed:



1. Fold the bathing sling down the middle. The label "This side up and against the headrest" should point upwards. Take hold of the half that is hanging down and fold it up. Repeat this until there is as little fabric as possible on one side.

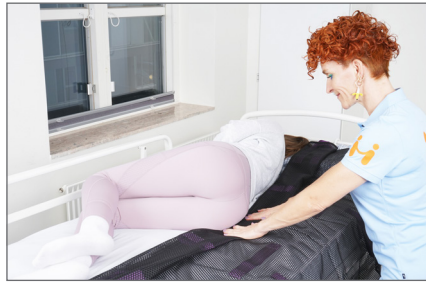


2. Turn the user onto his or her side. Place the bathing sling up along the user's back from top to bottom. The label "This side up and toward the back/head area" should be at the back of the user's head. The user's spine should be aligned with the center of the bathing sling. Slide the straps a little under the user.



3. The user is laid back down on the bathing sling and turned to the other side. The assistant pulls on the bathing sling straps to unfold it. The user is laid on his or her back again.

## Transfer sideways to/from the Bath stretcher:

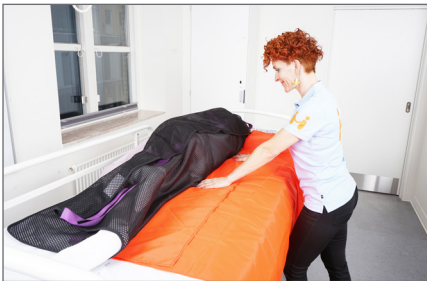

4. Turn the user to their side. Place a folded piece of sliding material alongside the user from head to feet.




5. Turn the user to supine. Gentle pull/push the user to the opposite gurney.




6. If the distance is large, one of the helpers can place their hands under the user's shoulders and help sliding the user the opposite side.

## Turning on the bathing unit using the lift:

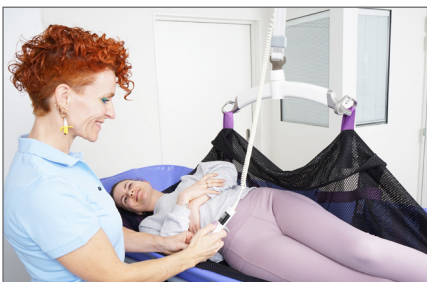

7. Hook the straps at the head and the hip area of the bathing sling onto the lifting beam on one side. Raise the hoist slightly.

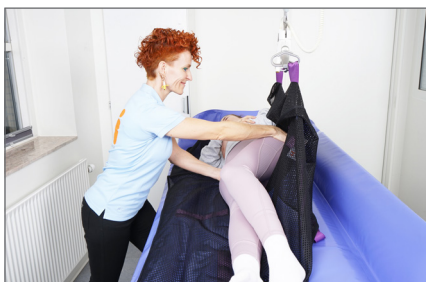


8. Slide the user into the middle of the unit to make room for the turn. Rise the lift and turn the user to full side lying position.



9. Remove the straps from the lifting beam. The Master Comfort washing wedge with a plastic protection bag can be placed between the user's knees to help stabilize the hip.

# User Manual - Master Turner Bath and Shower

## Removing the Master Turner Bath/Shower:

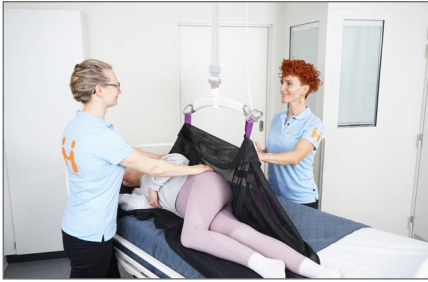

10. Lower the knee rest and the head rest. The straps at the head and the hip area of the bathing sling are hooked onto the lifting beam on one side. Raise the lift slightly till the user is lying on their side.



11. Remove the straps from the lifting beam. Roll the bathing sling and the Inko Master incontinence sheet up as far as possible under the user. Turn the user on his or her back.



12. Turn the user over to the other side and pull the bathing sling and Inko Master out from under the user.

## Only Master Turner Bath - Out of the bathing unit and into bed:



13. Place the top lifting loops from the Master Turner Bath on the lifting yoke.



14. Place the lower lifting loops on the yoke.



15. Place an Inco Master incontinence sheet on the bed to prevent it from getting wet. Lift the user to the bed and remove the Master Turner Bath.

# User Manual - Master Turner Bath and Shower

## Technical Specifications

REF - Product Name:	Size:	Handles:	SWL:	Content:	Product weight:
MC 001-1599 Master Turner Shower	L 170 x W 110 cm	12	150 kg	98% Polyester, 2% Polyamide	0,70 kg
MC 001-1600 Master Turner Bath	L 150 x W 120 cm	12	150 kg	99% Polyester, 1% Polyamide	1,06 kg

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Never leave the user lying on the product whilst connected to a hoist.



**Master Turner Shower:**  
Do not use as a sling sheet. Can be combined with a hoist for e.g. repositioning in bed



**Master Turner Bath:**  
The product is approved for lifting with hoists with hook attachment points cf. ISO 10535:2006.



Make sure that all relevant brakes are activated before transferring the user.



**CE-certification:**  
This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



**User Weight:**  
Recommended max. user weight: 150 kg.



**Washing:**  
This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



**Detergents pH level:**  
This product tolerates detergents / disinfectants with a pH level between 5-9.



**Fabric softener:**  
Never use fabric softener as this will reduce the sliding properties significantly.



**Dry Tumbling:**  
This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



**Surface cleaning:**  
The surface of this product can be cleaned with a wet cloth and soap.



**Disinfectants:**  
This product tolerates alcoholic disinfectants with a pH level between 5-9.



**Ironing:**  
Do not iron.



**Dry cleaning:**  
Do not dry clean.



**Bleach:**  
Do not bleach.



**Maintenance:**  
Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



**User Manual:**  
Always read the User Manual thoroughly before using the product.



**Disposal:**  
Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)

